

SKINACT DIAMOND MICRODERMABRASION WITH ULTRASONIC AND HOT/COLD HAMMER (SKU16140)



SKIN+ACT

TOLL FREE: 1.800.418.0830

WWW.SKINACT.COM
WWW.SPAANDEQUIPMENT.COM

INTERNATIONAL: 818.834.6640

TABLE OF CONTENTS

DISCLAIMER.....	03
FEATURES.....	04
SPECIFICATIONS.....	04
TOOLS.....	04
TREATMENT FUNCTIONS.....	05
GETTING READY FOR YOUR TREATMENT.....	06
OPERATING STEPS	06
SAFETY INFORMATION	08
WARRANTY AND PRODUCT SPECIFICATION DATA SHEET	09

THANK YOU FOR CHOOSING OUR PRODUCT!

Before using the device, we kindly ask you to carefully read the entire instruction manual provided. Please pay close attention to any warnings outlined in this manual. It's important to keep this instruction manual easily accessible at all times. If you pass the device to another individual, please ensure to provide them with the manual, as it is an integral part of the product. Please note that certain components of the product may differ from the images presented in catalogues, websites, brochures, etc. As part of our commitment to continually improve our products, the manufacturer reserves the right to modify features and/or components without prior notice.

**THANK YOU AGAIN FOR YOUR PURCHASE. WE HOPE YOU ENJOY
USING OUR PRODUCT!**

DISCLAIMER

CORRECT USE OF THE DEVICE

This machine is intended for general wellness purposes only. It is not a medical device and is not intended to diagnose, treat, cure, or prevent any disease. The information provided in this user manual is not a substitute for professional medical advice. Always consult with a physician before using this device, especially if you have any pre-existing medical conditions, are pregnant, or nursing. Discontinue use immediately if you experience any discomfort or adverse effects. The manufacturer and distributor disclaim any liability for damages or injuries resulting from the improper use of this device.

WHO SHOULD NOT USE THIS DEVICE

- Individuals with heart conditions, pacemakers, or irregular heartbeats.
- Pregnant or breastfeeding women.
- People with epilepsy, alcoholism, ulcers, or digestive system disorders.
- Those diagnosed with cancer or undergoing cancer treatment.
- Individuals who have had major surgery in the past year.
- People with asthma, cold, flu, or upset stomach.
- Anyone with a serious illness or disease not listed above.

AREAS TO AVOID TREATMENT

- Sensitive skin.
- Skin rashes, damaged, inflamed, or infected areas.
- Surgical scars less than 12 months old.
- Swollen skin.
- Eyeballs (probes should never point directly at them).
- Genitals.

TREATMENT CAUTIONS

- Always start with the lowest settings and gradually increase based on your skin's reaction.
- Mild swelling and redness may occur after treatment but should disappear within a few hours.
- Overly aggressive techniques can cause temporary flaking or bleeding.
- Avoid using the microdermabrasion function on eyelids (keep them closed and covered during treatment).
- **Certain areas are not suitable for treatment: skin tumors, moles, birthmarks, lips, eyes, herpes, skin cancer, and liver spots.**

DEVICE MAINTENANCE

- Do not modify or attempt to repair the machine yourself.
- Contact a qualified technician for maintenance needs.

FEATURES:

- **Diamond Microdermabrasion:** Steam intensity, timer and ozone option.
- **Ultrasonic:** 5 professional glass electrodes, high frequency handle, intensity level adjustment.
- **Cold/Hot Hammer Treatment:** Adjustable arm, bright cool-tone LED, and magnifying lens.

SPECIFICATIONS

DIAMOND MICRODERMABRASION

- Diamond Microdermabrasion has 3 tip grades
- 3 Large: Fine, Medium, Coarse
- 3 Medium: Fine, Medium, Coarse
- 3 Small: Fine, Medium, Coarse
- ***The optional Diamond Microdermabrasion comes with 9 Tips and 3 Handles with filters***
- ***The Diamond Microdermabrasion on this unit is class 1 grade. It is only for the surface of the skin***


ULTRASONIC

This kit comes with two different ultrasonic handles. The bigger flat handle is for the larger areas like the back or stomach. The smaller probe is for around the eyes


COLD/HOT HAMMER

- Cold Hammer Minimum Temperature: ~52° F / ~11° C
- Hot Hammer Minimum Temperature: ~120° F / ~49° C


Tools

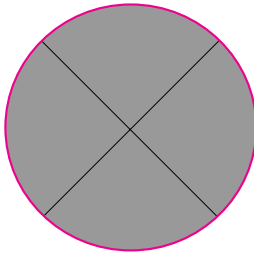

- **Diamond Microdermabrasion:** Diamond microdermabrasion resurfaces the skin non-surgically by using sterile diamond heads to peel and rub off the dead cells of the top skin layer. The microdermabrasion handles are attached to a vacuum. So, the dead skin cells are sucked away.

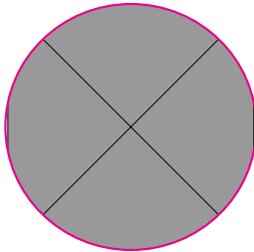

- **Ultrasonic:** Stimulates the skin tissue with acoustic vibrations. Improves cell membrane and regenerates collagen. Reduces wrinkles and fine lines for younger looking and feeling skin. Ultrasonic also generates frictional heat which accelerates blood circulation and helps muscles to relax. Decomposes cellulite and allows deeper product penetration into the skin.

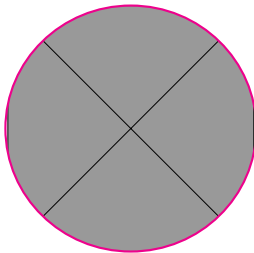

- **Cold/Hot Hammer:** The Cold/Hot Hammer cools and soothes the skin with the cold side and warms the skin on the other side. Cooling the skin is important after a microdermabrasion treatment. Warming the skin opens pores and helps products to penetrate more deeply.

TREATMENT FUNCTIONS

BEFORE YOU BEGIN

- **CONSULTATION.** It is highly recommended to consult with a dermatologist before using this device, especially if you have any pre-existing skin conditions, allergies, or concerns.
- **SKIN PREPARATION.** Always cleanse and thoroughly dry your face before using any of the system's functions. Remove any makeup or skincare products.
- **REVIEW INSTRUCTIONS.** Carefully review the separate attachment guide for detailed instructions on proper usage, recommended settings, and cleaning procedures for each applicator.

1. MICRODERMABRASION

Microdermabrasion gently exfoliates the top layer of dead skin cells, revealing a smoother, brighter, and healthier-looking surface.

THIS TREATMENT EFFECTIVELY ADDRESSES:

- Fine lines and wrinkles
- Crow's feet
- Age spots
- Acne scars

BENEFITS:

- Provides a fresh, healthy glow
- Non-surgical, safe, and quick "lunchtime" treatment
- Suitable for most skin types with proper settings (consult attachment guide)
- No anesthesia required (ideal for those with sensitive skin)
- Allows immediate resumption of normal activities

2. HOT & COLD THERAPY

This dual-function therapy provides targeted benefits for your skin:

- **Cold Therapy:** Soothes and calms the skin after microdermabrasion or other exfoliating treatments, minimizing redness and irritation.
- **Hot Therapy:** Opens pores to enhance the absorption of skincare products, maximizing their effectiveness.

3. ULTRASONIC FUNCTION

Ultrasonic waves gently stimulate the skin, promoting collagen and elastin production, the building blocks of youthful skin.

THIS RESULTS IN:

- Reduced fine lines and wrinkles
- Improved skin tone and texture
- Enhanced blood circulation and lymphatic drainage
- Increased hydration and plumping of the skin
- Deeper penetration of skincare products

TREATMENT GUIDELINES

Start with lower settings and gradually increase intensity as your skin tolerates. Discontinue use if you experience any discomfort or irritation.

GETTING READY FOR YOUR TREATMENT

Here's a simplified guide to prepare for your Supra 5 in 1 treatment and how to use each function:

BEFORE WE BEGIN:

1. **Skin Check:** A trained professional will examine your skin and create a personalized treatment plan.
2. **Protection:** The operator will wear gloves and a mask for hygiene.
3. **Clean Up:** For oily skin, a steamer or degreasing lotion may be used before cleansing. All skin types will be cleansed with a wipe and dried.

1. MICRODERMABRASION:

1. **Attach the Handpiece:** Connect the handpiece and tube to the machine.
2. **Diamond Tip Prep:** Place a cotton filter in the diamond tip, screw on the headpiece, and remove the filter after treatment.
3. **Power Up:** Select the desired power level (low, medium, high), treatment time, and start the machine.
4. **Treatment Technique:** Glide the handpiece in slow, even motions across the treatment area, avoiding staying in one spot too long. Stretch the skin gently for better coverage.
5. **Moisturize:** After treatment, dry your skin and apply a soothing gel.
6. **Clean Up:** Use a toothbrush or cotton swab with alcohol to clean the diamond tip. Disinfect it with UV light or high heat.

OPERATING STEPS:

1. Push in the movable ring of the copper nozzle at the handpiece, and pull out the 6x4 size vacuum hose. Then push in the black plastic ring on the vacuum hose.
2. Insert a cotton swab into the Diamond Head, then tighten the Diamond Head on the handpiece. Remove the cotton swab after every treatment.
3. Turn on the power switch.
4. Adjust the vacuum intensity using the Vacuum Regulator according to the reference table provided. The intensity will be displayed on the Vacuum gauge; ensure it is comfortable for the client.
5. Once the treatment is completed, cool the treated skin surface by applying cold towels. Continue until the skin becomes cool.
6. After the treatment, the skin may experience some redness and warmth, which is normal after an abrasive treatment.
7. Advise clients not to use any other abrasive products on the skin to avoid adverse reactions.
8. Adjust the intensity according to the client's comfort level.
Note: Due to the nature of its treatment, it is to be used ONLY by certified personnel. Any violations of the above will void the warranty, and operators will be responsible for any damages to the systems or clients.
9. Use the inclined area of the handpiece to contact the skin surface. Proceed with a slow sliding motion and avoid placing the handpiece at one spot for too long, as it may cause harm to the skin.
10. Adjust the intensity knob on the vacuum regulator slowly from minimum to maximum. Only set the flow rate to maximum if necessary, ensuring better comfort intensity.
11. Gently stretch the skin with your thumb and finger, then glide the handpiece 2-3 times over the treated area. Cross over the area in different directions to prevent streaking lines, as shown in the picture. Brush strokes or polishing motions may be used in different areas.
12. After treatment, brush off any remaining dead skin with a soft brush and rinse with damp sponges or a damp, cool towel.
13. Allow the skin to dry, then apply the desired skincare product.

14. Remember to apply sunscreen lotion after treatment to protect the skin from UV light.
15. To clean the Diamond Head, use a toothbrush or cotton swab dampened with a little alcohol to wipe off any dirt or impurities. Then, use UV light or high temperature for sterilization.

Recommended Vacuum Intensity

Treatment Area	Normal Skin (in HG)	Sensitive Skin(in Hg)
Face	12-22	8-12
Forehead	12-22	10-12
Chest	15-20	15-18
Nose	10-15	10-14
Neck	18-18	10-15
Near Eye Area	8-12	8-10

Treatment Scheduled Table

Treatment	Frequency (days)	Total Visits
Face Peeling	6-10	3-5
Fine Lines	7-13	6-8
Neck	12-14	3-5
Acne Scars	7-12	8-10
Hyper pigmentation	7-12	8-10

2. HOT & COLD THERAPY:

Soothe & Enhance: This dual function offers targeted benefits.

- **Cold Treatment:** Use this function to calm and reduce redness after microdermabrasion or other exfoliating treatments. It's especially helpful as your skin is more sensitive after peeling.
- **Hot Treatment:** Use this function to open pores for better absorption of skincare products, maximizing their effectiveness.
- **Temperature Selection:** The professional will help you choose the appropriate setting (hot or cold) based on your treatment needs.

Targeted Application: The professional will gently apply the attachment to the desired area for the recommended treatment duration.

3. ULTRASONIC:

1. **Function Selection:** Choose "Eye" or "Face" mode, set power level and treatment time, then start the machine.
2. **Power Adjustments:** You can adjust power during treatment. To switch modes, pause, select the new mode, and resume.

A. FACE TREATMENT:

1. Use the larger probe.
2. Choose your desired intensity level.
3. Select pulse or continuous wave mode.
4. Gently move the probe around your face, avoiding the eye area.

B. EYE TREATMENT:

1. Use the smaller probe.
2. Choose your desired intensity level.
3. Select pulse or continuous wave mode.
4. Gently move the probe around the eye area.

SAFETY INFORMATION

Contraindications: Do not use this device if you have any of the following:

- Active skin conditions (e.g., rosacea, eczema)
- Open wounds or infections
- Pacemaker or other implanted electronic devices
- Pregnancy or breastfeeding
- **Sunburn:** Do not use this device on sunburned skin.
- **Skin Sensitivity:** Individuals with very sensitive skin should consult a dermatologist before use and perform a patch test on a small area of the face to assess tolerance.
- **Post-Treatment Care:** Always follow recommended post-treatment care instructions, such as applying sunscreen with SPF 30 or higher after using the LED Light Therapy function.

MAINTENANCE: Regularly clean and disinfect the applicators according to the instructions in the attachment guide.

IMPORTANT NOTICE OF CAUTION BEFORE USING THIS MACHINE:

Never operate this equipment on a person who has or suffers from the following medical conditions described below. (The salon or operator may consider having a client sign a waiver stating that he/she is free of any health conditions.)

- Heart Disease
- Heart Murmur or Irregular Heart Rate
- Has a pacemaker or other type of heart rate balancer
- Pregnancy
- Epilepsy
- Alcoholism
- Ulcers
- Disease of Stomach, Intestine, Liver, or Pancreas
- Breast Cancer
- Undergoing treatment for any form of Cancer
- Had major surgery within the past year
- Asthma
- Cold, Flu, or upset stomach
- Any serious illness or disease not listed above

CAUTIONS

Always begin treatment conservatively and observe the effects by touch and appearance of the skin. Vacuum power that is too high or too large may cause an inflammatory response.

Edema and erythema can occur following treatment but will resolve in a few hours.

Overly aggressive techniques can cause mild flaking over the next 3-4 days. Epidermis that is treated too aggressively can lead to bleeding in the superficial dermis.

Diamond Dermabrasion should not be used on sensitive areas such as eyelids. The eyelids should be closed at all times during the procedure and may be covered with damp cotton pads.

The following areas are considered inappropriate for treatment: Skin tumors, moles, birthmarks, lips, eyes, herpes, skin cancer, and liver spots.

Do not modify or attempt to repair the machine; contact the technician for maintenance.

SpaandEquipment offers a 1-year limited warranty on most equipment, covering malfunctions and manufacturer defects only (parts only, shipping not included). Warranty is void for improper use, wear & tear, alterations, or lack of professional installation for plumbing & electrical equipment (proof required). Customers are responsible for return shipping costs and damage during return shipment voids warranty (ship with insurance).

For details or questions, contact SpaandEquipment.com at 1.800.418.0830.

For further details regarding warranty coverage, please refer to our Warranty Information Policy on our website.

You can find it under the “Return/Shipping/Warranty” section.

SPECIFICATIONS AND TECHNICAL DATA

PRODUCT NAME:

SKINACT Diamond Microdermabrasion With Ultrasonic and Hot/Cold Hammer (SKU16140)

FUNCTION:

This revolutionary equipment features a Diamond Microdermabrasion, Ultrasonic, and Cold/Hot Hammer treatment capabilities all in one convenient device.

SPECIFICATIONS:

Dimensions: 11”L x 15.5” W x 4.25” H