

SKIN+ACT



Facial Spa Skin Scope

USER MANUAL



Function:

The Skin Scanner enables both the client and esthetician to thoroughly analyze the current condition of the skin, providing a clear and detailed visual representation of underlying issues and potential problem areas. By highlighting these concerns, the device facilitates more informed discussions between the esthetician and the client. This advanced analysis tool allows the esthetician to effectively explain the importance of targeted treatments and personalized skincare regimens, ensuring the client fully understands the benefits of proper care. As a result, the client is empowered to make more informed decisions about their skincare journey, leading to better long-term outcomes.

Key Features:

- **Comprehensive Skin Analysis:** Provides a detailed visual representation of the skin's condition, including pigmentation, texture, and moisture levels.
- **Identification of Problem Areas:** Easily highlights specific areas of concern, allowing for targeted treatment recommendations.
- **Enhanced Communication:** Facilitates clear and effective discussions between the esthetician and client regarding skincare needs and solutions.
- **Personalized Treatment Plans:** Enables the development of customized skincare regimens based on the individual analysis of the client's skin.
- **Empowerment Through Knowledge:** Educates clients about their skin health, fostering informed decision-making regarding their skincare choices.

This Skin Scanner is an essential tool in modern esthetics, enhancing the client experience and optimizing treatment outcomes.



Note:

1. Ensure that the local voltage matches the rated voltage of the appliance before use.
2. Only a qualified electrician is authorized to disassemble or replace any components or accessories of this product. Do not attempt to modify the unit yourself.
3. For any operational issues, please contact the supplier or distributor for assistance.
4. Avoid prolonged direct exposure to ultraviolet bulbs to prevent eye damage.
5. Clean the appliance thoroughly after each use to maintain its condition.
6. This appliance is intended for use in beauty salons or similar environments.
7. The appliance must be used in accordance with the instructions. The company is not liable for any malfunction or issues arising from improper use.
8. This machine is not suitable for individuals with physical or mental disabilities, slow reactions, or respiratory issues (including children), unless they are supervised by a responsible person who ensures safe usage.
9. Children should be supervised to ensure they do not interact with or operate the appliance.
10. This product is not classified as medical equipment and should not be used for the treatment of any medical conditions.

Color Chart

| Analysis lamp | skin |
|----------------------------|-------------------------|
| 1 White | spots mature skin area |
| 2 Blue-White | normal and healthy skin |
| 3 White fluorescence | Thick corneum layer |
| 4 Yellow or something pink | oily, acne skins |
| 5 bright fluorescent | Hydrate skin |
| 6 Purple | sensitive skin |
| 7 violet | Dehydrated skin |
| 8 Brown | pigments |



Operation Method for the Skin Scope

To ensure optimal functionality and accurate skin analysis using the Skin Scope, please follow these detailed steps:

1. Preparation and Setup:

- Position the Skin Scope on a stable table or cart at eye level to facilitate easy access and visibility during use.

2. Machine Activation:

- Lift the skirted door fully to the top and turn on the machine to prepare it for operation.

3. Client Preparation:

- Ensure the client's skin is thoroughly cleansed to provide an accurate analysis.
- Position the client's head under the skirt, ensuring the fabric covers the entire head down to the shoulders. The skirt allows the Skin Scanner to function without requiring a darkroom, enhancing usability in various settings.
- Adjust the mirror to the appropriate angle for optimal viewing.

4. Comprehensive Examination:

- Carefully examine the client's entire face, taking detailed notes throughout the analysis to ensure proper treatment recommendations. Utilize the provided color chart for reference and accurate assessment of skin conditions.

5. Initiate the Analysis:

- Turn on the machine to begin the testing phase. The cosmetologist can view the client's facial skin through the viewing window, enabling a direct comparison of the client's skin tone with the color card provided.

6. Skin Analysis:

- Analyze the client's skin condition, identifying specific issues such as pigmentation irregularities, texture variations, and overall skin health.

7. Post-Use Procedures:

- After completing the analysis, turn off the unit to conserve energy and maintain equipment longevity.
- When the Skin Scope is not in use, ensure the skirt is neatly tucked into the machine to protect the device and maintain a tidy workspace.

By following these steps, cosmetologists can maximize the effectiveness of the Skin Scope, providing clients with thorough and insightful skin evaluations that lead to tailored treatment plans and improved skincare outcomes.



Specifications:

1. Voltage: 220V-240V
2. Frequency: 50Hz/60Hz
3. Power: 100W

Note: This appliance has two technical data depends on different country. Please subject to the technical data sticker on the appliance.

What's Included:

1. Main Unit
2. Power Cord
3. User Instructions