

# SKIN+ACT



**Microcurrent And Magic Glove BioLift Skin Care Unit**


**SKU25360**

## FRONT PANEL



## Instruction of Use

- . Connect the power supply via the rear panel
- . Turn on the POWER button

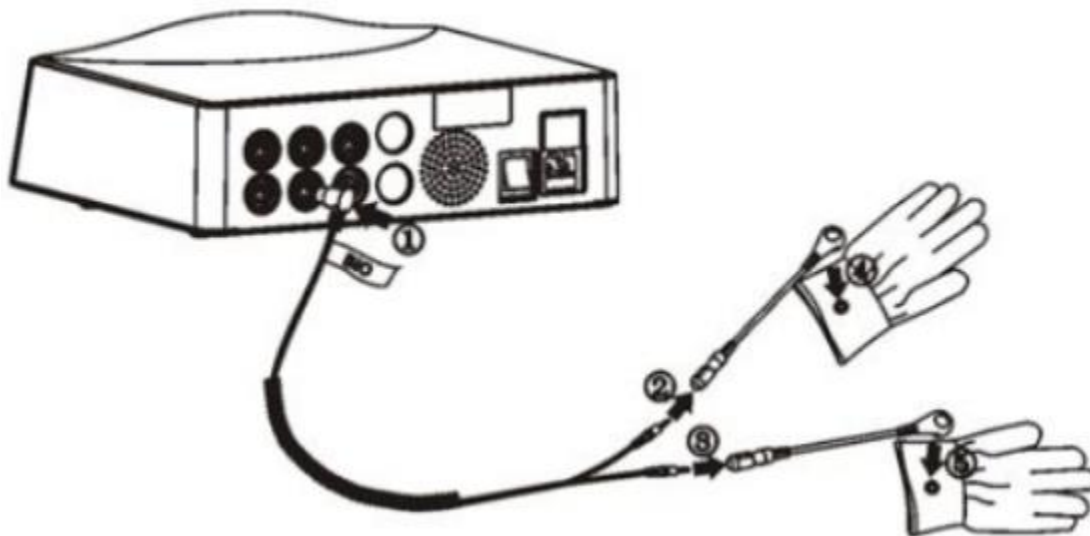

## MAGIC GLOVES INSTRUCTIONS OF USE

- The beautician must put isolating gloves under the magic gloves.
- Make sure that the "ENERGY" level is at the minimum. You can adjust the output level by pressing "ENERGY +" or "ENERGY -".

- Make sure that the “FREQUENCY” level is adjusted at 0.1 Hz. You can adjust the frequency level by pressing “FREQ +” or “FREQ -”.
- Select the desired wave state by pressing “STATE”. There are 4 kinds of waves.
- Select the time of operating by pressing “TIMER +” or “TIMER -”.
- Press “START/PAUSE” to start the treatment.
- The beautician must massage the customer with the magic gloves. Adjust the “ENERGY” and “FREQUENCY” level according to the customer’s feelings. At this moment, the beautician don’t have to feel any effect in their hands, but the customer must feel a micro current and some warm in the part where the treatment are applied.

## **NOTES**

- The delivered magic gloves have a length of 20 cm. The isolating gloves must have a length of at least 23 cm.
- If there are contact between the beautician’s skin and the magic gloves, there’s no effect with the treatment.
- During the treatment, if the beautician breaks the contact with the customer’s skin, the device stops working suddenly. The device restarts when the contact between the magic gloves and the customer’s skin returns.


## **CAUTIONS**

- Pacemaker (any electrical or rhythmic heart-regulating device)
- Pregnancy
- Thrombosis/Phlebitis
- Skin diseases
- Operations (within the last 6 months)
- Silicone implants
- Hypersensitive or vascular skin

## **BIO-LIFTING INSTRUCTIONS OF USE**

- Connect one side of the wire in the rear panel, in the plug named “BIO WAVE MICROCURRENT”.

- Connect the other two sides of the wires in the handles of BIO-lifting. There are two kinds of handles:
  - (1)The ones that need cotton swabs are used in the eyes part. Is needed to include two cotton swabs in the hole of the handle.
  - (2)The ones with only one pole are used to the full body.
- Apply enough conductive gel to the treatment area.
- Make sure that the “ENERGY” level is at the minimum. You can adjust the output level by pressing “ENERGY +/-” button.
- Make sure that the “FREQUENCY” level is adjusted at 0.1 Hz. You can adjust the frequency level by pressing “FREQ +/-” button.
- Select the desired wave state by pressing “STATE”. There are 4 kinds of waves.
- Select the time of operating by pressing “TIMER +/-”.
- Press “START/PAUSE” to start the treatment.

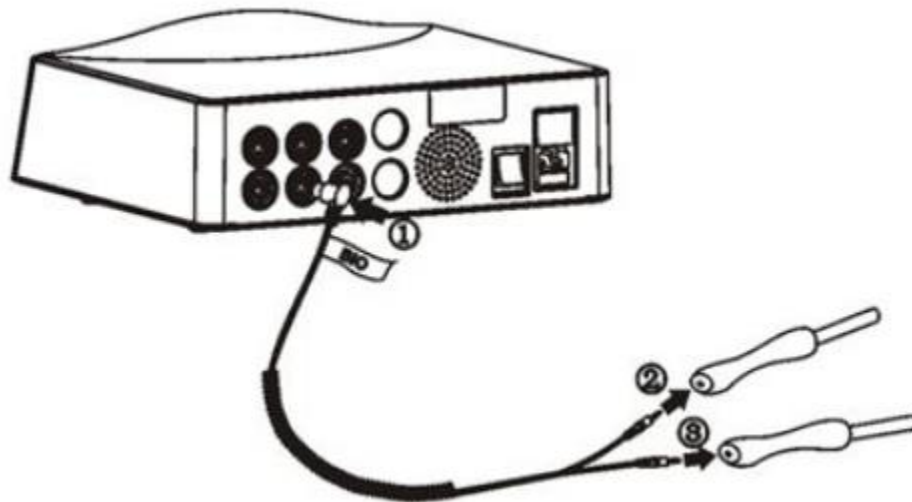

## MAINTENANCE

After each treatment, the magic gloves and the BIO-lifting handles must be cleaned, sterilized and properly saved in a dry place.

## TECHNICAL SHEET

Weights and sizes	Product size (cm)	20.7×22×7
	Gross weight (kg)	3.5
Electrical characteristics	Rated voltage	AC 100-240V
	Frecuency	50/60Hz
	Rated power	4 W

1-800-418-0830 TOLL FREE

1-818-834-6640 INTERNATIONAL

Address: 9667 Canoga Ave, Chatsworth, CA 91311

# SKIN+ACT