

Pro 10-Function Skincare System (SKU16147)



SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

SKIN+ACT

Toll Free: 1.800.418.0830	www.skinact.com www.spaandequipment.com	International: 818.834.6640
---------------------------	--	-----------------------------

USER GUIDE

A. Vaporizer

The facial vaporizer is considered one of the most important devices used in esthetics. Lukewarm vapor helps soften the skin's surface, opening pores and aiding in deep cleansing.

Benefits of the vaporizer:

1. Softens dead surface cells so they can be exfoliated more easily.
2. Opens follicles for deeper cleansing.
3. Helps loosen deposits of grease, blackheads, and makeup so debris can be removed.
4. Assists in eliminating toxins through pore cleansing.
5. Improves cell metabolism.

Caution:

- Always use distilled water. Tap water may cause malfunction due to calcium/mineral deposits.
- The vaporizer will automatically shut off when the water level drops below 1/3 full.

Care of the vaporizer:

- Clean the glass jar and heating elements weekly with a mixture of distilled water and white vinegar (1 cup vinegar to 1 cup water). Allow to soak overnight, rinse, and dry before reuse.

Operating steps:

1. Open the rubber cap, fill the jar with distilled water, and close securely.
2. Set the timer (7–12 minutes).
3. Switch on the unit and adjust steam nozzle direction according to skin condition.

B. Wood's Lamp

Used by estheticians to analyze skin conditions under deep violet rays.

Skin condition under lamp:

- Thick, horny layer: Fluorescence
- Healthy skin: White spots
- Thin/dehydrated skin: Blue-white
- Oily areas & comedones: Brown
- Dehydrated skin: Purple
- Hydrated skin: Pink/yellow tones

Operating steps:

- 1.Press red button (lamp turns on).
- 2.Press black button (lamp turns off).

Caution:

- Do not overheat. Avoid direct contact with the bulb.

C. Peeling

Removes dead skin cells, dirt, and grime.

Instructions:

- 1.Select appropriate brush and lock into place.
- 2.Switch to the correct speed according to client's skin type.
- 3.To change brushes: turn switch "OFF," gently pull off the brush, and insert another.
- 4.Use moist towels or sponges after treatment to remove residue.

Caution:

- Use slow speeds on sensitive skin.
- Avoid areas with broken capillaries.

D. Vacuum

Helps deep-clean pores by removing oil, dirt, and grease. It also improves blood circulation and absorption.

Operating steps:

1. Insert appropriate vent-hole.
2. Adjust suction by covering/uncovering vent with finger.
3. Apply gently with cleansing lotion for lubrication.
4. Clean glass suction cups after each use.

E. Spray

Delivers a fine mist to soothe and refresh the skin. Helps soften pores and remove excess products.

Operating steps:

1. Fill spray bottle with 2 parts distilled water and 1 part astringent.
2. Attach bottle, turn power on, and spray evenly over face.

F. High-Frequency

Benefits:

1. Aids elimination of toxins.
2. Increases gland activity.
3. Stimulates blood circulation.
4. Germicidal action.
5. Increases metabolism.
6. Improves product penetration.

Operating steps:

1. Insert electrode.
2. Switch "ON" and adjust intensity.
3. Turn "OFF" before removing electrode.

Caution:

Do not use on clients with pacemakers, pregnant women, or those with metal implants.



G. Galvanic

Introduces skincare products deep into the skin using iontophoresis.

- Positive pole: Closes pores, reduces redness, prevents inflammation.
- Negative pole: Stimulates blood circulation, forces desincrustation (deep cleansing).

Operating steps:

1. Apply skin care lotion.
2. Switch "ON" and set polarity (+ or -).
3. Adjust intensity to client's comfort.

Caution:

- Do not use galvanic current on broken capillaries or sensitive conditions.

H. Hot Cabinet

- Used for sterilizing towels at ~80°C.
- Capacity: up to 24 towels.

I. Magnifying Lamp

- Adjustable arm and light for magnified skin analysis.

J. Bowl

- Two bowls below the steamer for products or accessories.

General Cautions

1. Machine has been tested and inspected. If malfunction occurs, contact a qualified technician.
2. For professional use only.
3. Follow instructions carefully. The company is not responsible for damage caused by misuse.
4. Not suitable for children or individuals with weak constitution, slow reaction, or psychosis.
5. Children should not play with the machine and must be supervised at all times.

Dimensions:

39.4"H x 29.5"W x 22.4"D

Hot Towel Cabi Dimensions: 6"L x 6"W x 13"H

Weight: 75 lbs.

