

Ozone Steamer, 5 Diopter Magnifying Lamp And High Frequency



SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

SKIN+ACT

Toll Free: 1.800.418.0830	www.skinact.com www.spaandequipment.com	International: 818.834.6640
---------------------------	--	-----------------------------

Ozone Steamer + 5X Magnifying Lamp + High Frequency

Safety Precautions

For professional use only. Do not leave clients unattended.

Plug into 120V, grounded outlet only. No extension cords.

Always use distilled water.

Never pour cold water into a hot steamer.

Allow unit to cool before draining or refilling.

Do not remove or tamper with safety parts.

Components Overview

. Ozone Facial Steamer

Opens pores, improves circulation, and gives radiant skin.

Adjustable timer (up to 60 min). Rotating steamer arm.

Aromatherapy compartment included.

. 5X Diopter LED Magnifying Lamp

Provides enhanced visibility with cool-tone LED. Adjustable arm.

. High Frequency Device

Stimulates circulation, helps product absorption, kills bacteria.

Comes with 3 electrodes: Mushroom, Spoon, Spot.

Specifications

- . Voltage: 120V
- . Power: 750W
- . Frequency: 60Hz
- . Beaker: 35 oz capacity

Maintenance

- . Always empty and clean after each use.
- . Let unit cool before handling water.
- . Check water level regularly.
- . Clean electrodes and lamp with alcohol wipes.

Warranty

1-Year Limited Warranty

