

# Galvanic Portable Skincare Machine (SKU16190)



**SKIN+ACT**

Toll Free: 1.800.418.0830

[www.skinact.com](http://www.skinact.com)  
[www.spaandequipment.com](http://www.spaandequipment.com)

International: 818.834.6640

## NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

# SKIN+ACT

Toll Free: 1.800.418.0830	<a href="http://www.skinact.com">www.skinact.com</a> <a href="http://www.spaandequipment.com">www.spaandequipment.com</a>	International: 818.834.6640
---------------------------	--	-----------------------------

# Galvanic Instrument User Guide

Thank you for purchasing our Galvanic Instrument. Please read this manual carefully before use. Proper understanding and operation will ensure the best performance and safe use of the device.


## Functions

The Galvanic Instrument is designed for facial treatments and has two primary functions:

- Product Infusion: Introduces water-soluble skincare products into the skin.
- Polarity Treatments: Uses positive (+) and negative (-) polarity to achieve different skincare effects when applying a direct current (DC).

## Positive Pole Functions (+)

1. Close pores.
2. Reduce blood circulation.
3. Firm and tighten the skin.
4. Soothe redness after massage.
5. Infuse positively charged products (e.g., astringents) into the skin.

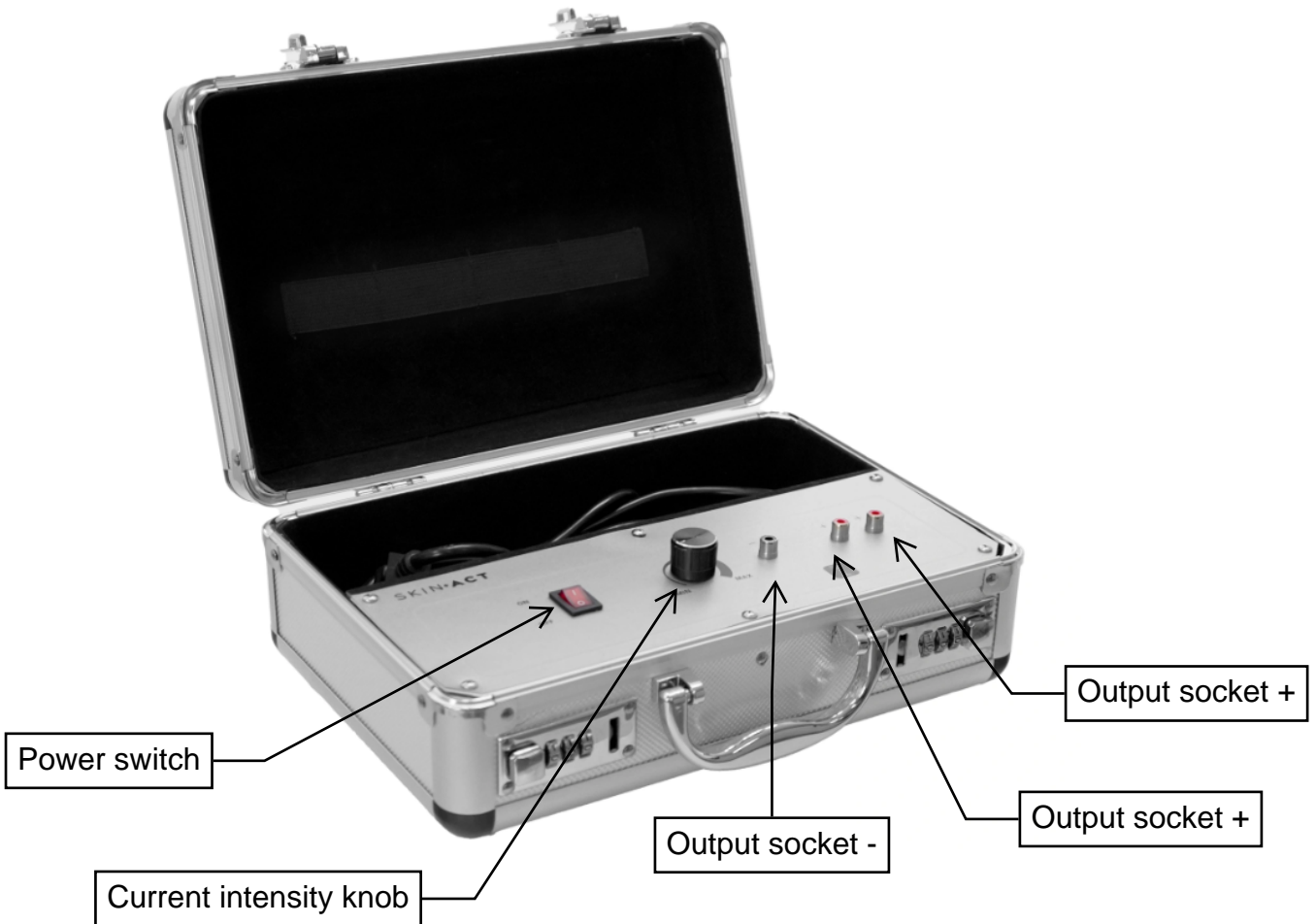

## Negative Pole Functions (–)

1. Soften skin and unclog blocked pores.
2. Infuse alkaline pH products (e.g., desincrustation lotion).
3. Emulsify sebum and aid in deep cleansing.
4. Avoid high current on areas with many broken capillaries.
5. Introduce chemical solutions into intact skin.
6. Cataphoresis: Positive pole introduction of acidic products.
7. Anaphoresis: Negative pole introduction of alkaline products.

## Operating Instructions – Desincrustation

- One electrode is held by the client; the esthetician holds the other.
- Apply a suitable conductive solution or desincrustation lotion to the treatment area.
- Adjust current intensity according to skin type and client comfort.
- Select polarity based on the desired treatment.
- For normal or oily skin, use the appropriate polarity and solution as recommended.
- Always return intensity to “0” before changing polarity or ending treatment.

## Safety Warnings

- Not suitable for individuals with certain health conditions or extreme weakness.
- Only trained personnel should operate this device.
- Keep out of reach of children.
- Always follow safe operating practices.

## Technical Specifications

- Voltage: 120V or 220–240V
- Frequency: 50–60 Hz
- Power: 10 W