

SKIN+ACT

Cleo Electric Spa Treatment Table (Facial Massage Bed)

PRODUCT MANUAL



TABLE OF CONTENTS

Notice & Disclaimer3

Safety and Operating Instructions4

Precautions.....4

Measurements & Angle Specifications5

Remote Control Instructions.....7

Maintenance7

Troubleshooting.....8

Specifications.....9

Warranty Information10

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

This product contains electric adjustment components and moving parts. Improper use may cause injury, pinching, equipment damage, or unsafe operation. Users are responsible for operating the table safely and following all instructions, warnings, and maintenance requirements in this manual. The user is solely responsible for confirming that use of this device is permitted under applicable state/local laws, scope-of-practice rules, licensing requirements, and workplace safety standards. This device is not intended for self-use, home use, or use by untrained individuals.

Safety and Operating Instructions

To ensure safe and proper operation of this electric treatment table, please read the following instructions carefully and retain this manual for future reference.

- **Proper Setup and Grounding:**

This electric treatment table is equipped with a three-prong power cord and must be connected to a properly grounded three-prong outlet. Proper grounding helps reduce the risk of electric shock.

- **Avoid Water Exposure:**

Keep all plugs, controls, power cords, and connecting cables away from water and other liquids.

- **Inspect for Damage:**

Immediately discontinue use if the plug, controls, power cord, or connecting cables are damaged.

- **Usage Frequency:**

Do not operate the electric actuator continuously for more than 2 minutes. Allow it to rest for at least 18 minutes before operating it again.

- **Professional Repairs Only:**

Repairs and service must be performed only by qualified service technicians.

Precautions

1. Health Restrictions:

- Individuals with osteoporosis, heart conditions, injuries, or skin conditions that could be aggravated by positioning or contact with the table should obtain medical clearance before use.
- Pregnant individuals should avoid using this product.

2. Use by Minors and Individuals with Impaired Awareness:

- Children under 14 years of age and individuals with impaired consciousness must not use this product without supervision.

3. Inspection Before Use:

- Inspect the power cord and plug for damage before each use. If damage is found, discontinue use and have the damaged component replaced by a qualified technician or authorized service provider.

4. Voltage Compliance:

- Confirm that the product is connected to the correct voltage as specified. Incorrect voltage can result in product damage.

5. Electrical Safety:

- Do not handle the power plug with wet hands to prevent electric shock.
- Ensure the product is properly installed before powering it on or operating it.

6. Securing the Power Cord:

- Secure the power cord to prevent tripping, pulling, or damage.


7. Cleaning:

- Clean the product's surface with a soft towel dampened with neutral detergent.

8. Unplugging When Not in Use:


- Always unplug the product when it is not in use.

Measurements & Angle Specifications

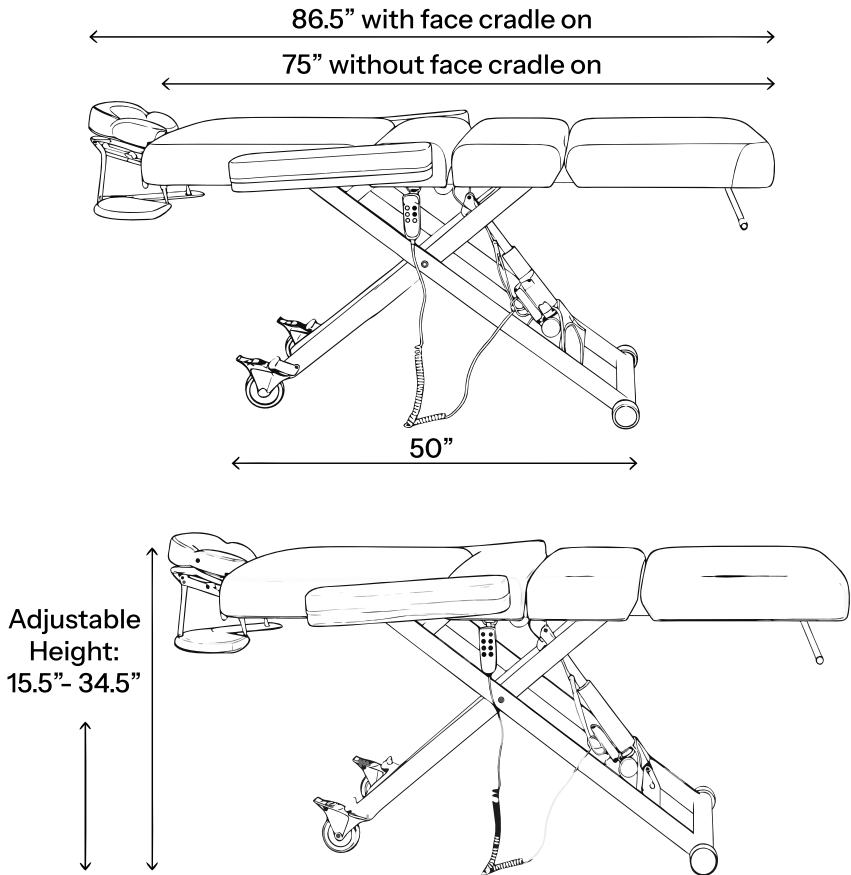


*Measured from the floor to the top of the cushion

*All measurements in this manual have a tolerance of 0.25–0.75 in.

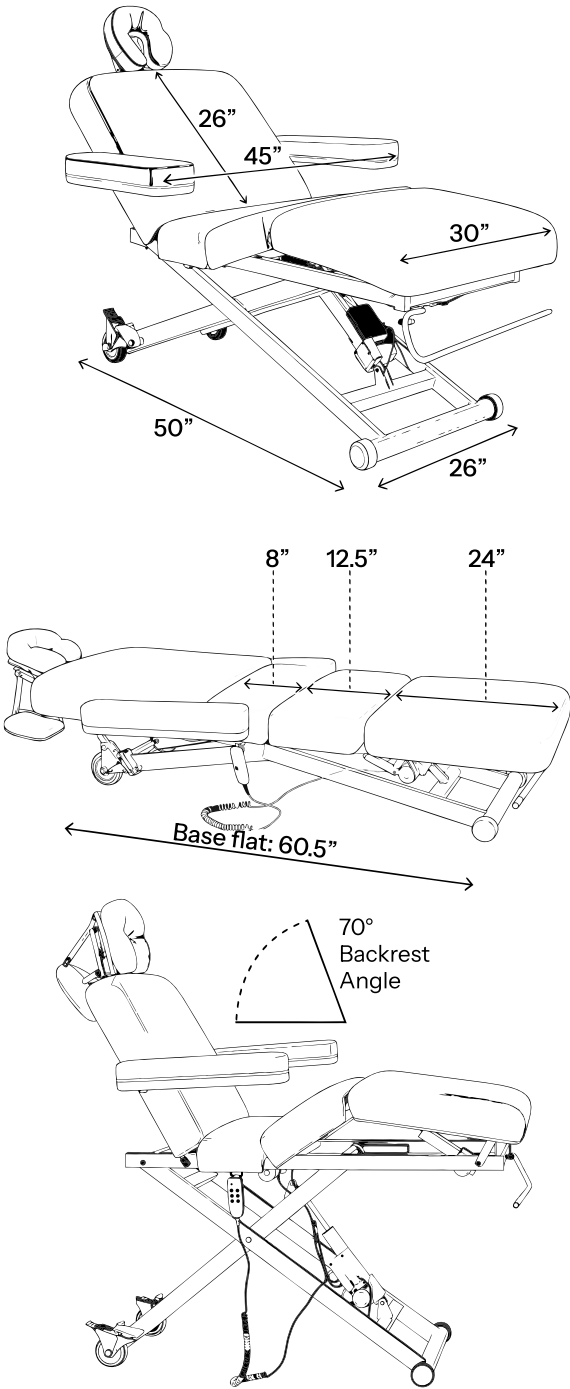

Remote Control Instructions

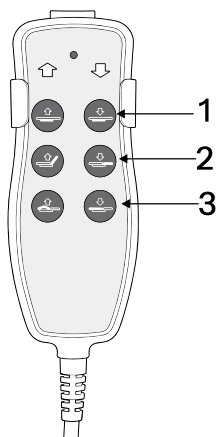

① Raise or Lower the Bed

Press and hold the upper button to raise the table.
Press and hold the lower button to lower the table.

② Adjust the Backrest Angle

Press and hold the upper button to increase the backrest angle.

Press and hold the lower button to decrease the backrest angle.

③ Adjust the Leg Rest Angle

Press and hold the upper button to raise the leg rest.
Press and hold the lower button to lower the leg rest.

Maintenance

- When cleaning and sanitizing, do not use bleach, Barbicide, or other harsh chemicals. Using harsh chemicals can damage the upholstery.
- For best results, we recommend cleaning with **SkinAct All-In-One Upholstery & Disinfecting Multi-Purpose Cleaner** (SKU24261).
- Clean and sanitize the upholstery after each treatment.
- Inspect all accessible components of the unit every six months. If there are any abnormal noises or vibrations during operation, discontinue use and contact SkinAct at 1-800-418-0830.

Troubleshooting

Motor Does Not Operate Properly

Solution: Check that the hand controller or foot controller is securely connected and responding correctly. If the controller does not respond, discontinue use and contact the supplier or service center for inspection.

Control Box or Motor Malfunction

Solution: If the motor fails to operate after confirming the controller connection and power supply, stop using the product. Contact SkinAct/Spa & Equipment for troubleshooting or replacement parts.

Abnormal Noise During Operation

Solution: Apply lubricating oil to moving parts such as hinges, lift areas, or footrest joints. If the noise continues after lubrication, stop using the product and contact SkinAct/Spa & Equipment.

Power Display or Control Box Indicator Does Not Light Up

Solution: If the power display or control box indicator does not light up during normal use, the control box may need to be inspected or replaced. Contact SkinAct/Spa & Equipment for further assistance.

Specifications

Product Name:	Cleo Electric Spa Treatment Table (Facial Massage Bed)
Voltage:	120 V AC
Frequency:	60 Hz
Rated Power:	200 W
Height Range:	15.5”- 34.5”
Length without Face Cradle:	75”
Length with Face Cradle:	86.5”
Width without Arm Rest:	30”
Width with Arm Rest:	45”
Backrest adjustment:	0° flat to 70° incline
Base Dimensions (L x W):	50”-60.5”x 26”
Weight Capacity:	400 lbs

*Length of base changes according to table height

Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor’s discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

Warranty does not cover damage, contamination, upholstery damage, sanitation-related issues, injury claims, or equipment issues resulting from improper cleaning, improper maintenance, misuse, use by untrained users, or use outside applicable safety guidelines. This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

SKIN+ACT

Phone:1-800-418-0830
Email:info@skinact.com
Website:www.skinact.com