

SKIN+ACT

Palma Electric Medical Spa Treatment Chair/Table (With Swivel)

PRODUCT MANUAL



TABLE OF CONTENTS

Notice & Disclaimer 3

Safety and Operating Instructions 4

Precautions 4

Installation 5

Dimensions 6

Measurements 7

Angle Specifications 8

Operation Cautions 9

How to Tighten the Brake Block 10

Remote Control Instructions 11

Foot Pedal Controls 12

Chair Controls 12

Motor Controls 13

Maintenance 14

Troubleshooting 15

Symbols & Safety Labels 16

Specifications 17

Warranty Information 18

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

This product contains electric adjustment components and moving parts. Improper use may cause injury, pinching, equipment damage, or unsafe operation. Users are responsible for operating the table safely and following all instructions, warnings, and maintenance requirements in this manual. The user is solely responsible for confirming that use of this device is permitted under applicable state/local laws, scope-of-practice rules, licensing requirements, and workplace safety standards. This device is not intended for self-use, home use, or use by untrained individuals.

Safety and Operating Instructions

To ensure safe and proper operation of this electric treatment table, please read the following instructions carefully and retain this manual for future reference.



- **Proper Setup and Grounding:**
 - This electric treatment table requires a three-wire power cable and must be connected to a grounded three-prong power outlet. This ensures proper grounding, prevents accidental electric shock, and supports normal operation.
- **Avoid Water Exposure:**
 - All plugs, buttons, and connecting cables must not come into contact with water or any other liquids during use.
- **Inspect for Damage:**
 - Immediately discontinue use of the table if any plugs, buttons, or connecting cables are damaged.
- **Usage Frequency:**
 - The electric push rod of the treatment table should only be operated for up to 6 minutes per hour.
- **Professional Repairs Only:**
 - Repairs and maintenance must be conducted exclusively by trained and qualified professionals.

Precautions


- 1. Health Restrictions:**
 - This product should not be used by individuals with osteoporosis or heart conditions.
 - Pregnant individuals should avoid using this product.
 - Those with physical injuries or skin conditions are advised not to use this product.
- 2. Use by Minors and Individuals with Impaired Awareness:**
 - Children under 14 years of age and individuals with impaired consciousness must not use this product without supervision.
- 3. Inspection Before Use:**
 - Inspect the power cord and plug for damage before each use. If damage is found, they must be replaced by a qualified operator or authorized service provider.
- 4. Voltage Compliance:**
 - Confirm that the product is connected to the correct voltage as specified. Incorrect voltage can result in product damage.

Installation



1. Place the product in the desired location according to the overall room layout, lighting, and operation conditions of the spa, salon, or treatment room. Keep the unit in a clean, dry, and cool environment.
2. Open the packaging and check that the product is intact. Confirm that all included accessories and spare parts are complete. Contact the distributor if anything is missing or damaged.
3. Position the product on a firm, flat, and stable surface. Make sure the unit is placed securely before use. If the product rocks or does not sit evenly, adjust its placement until it is stable.
4. Connect the power plug to the proper power supply as shown in the reference image.
5. Disconnect the plug from the main power supply before moving or servicing the product.

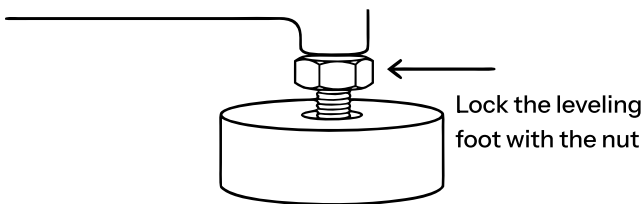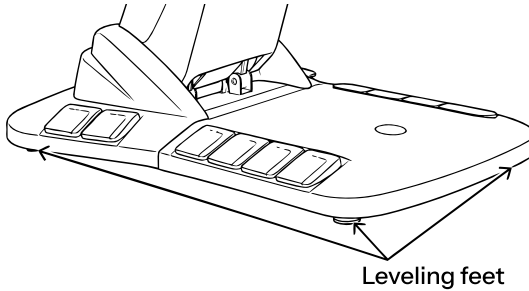


Dimensions

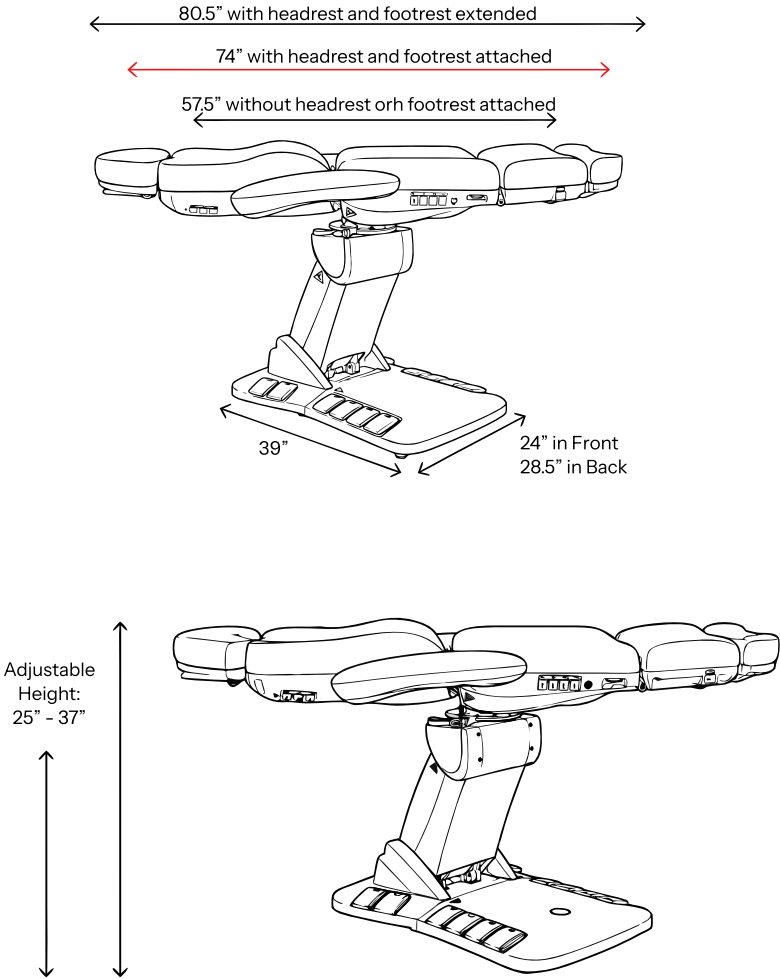

Measurements

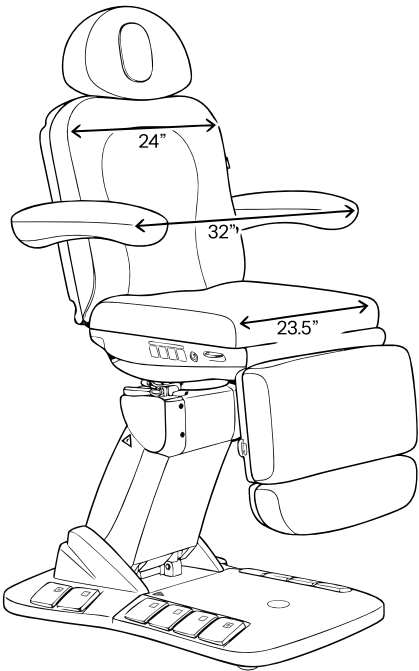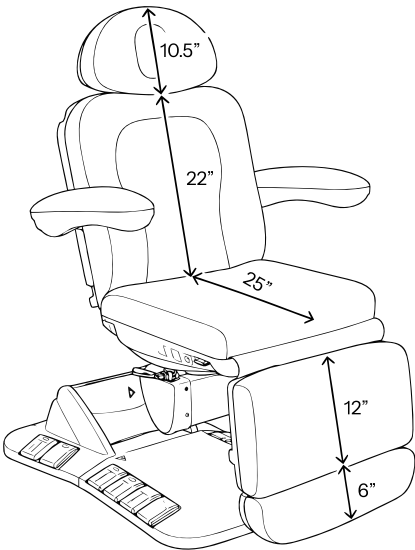


Angle Specifications

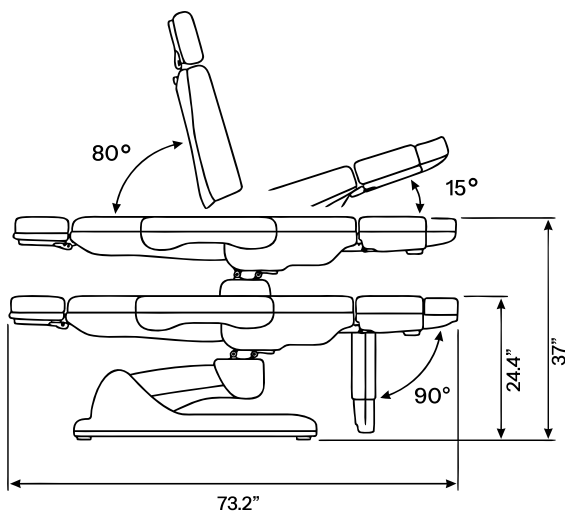


Rotation

-60° to +60°

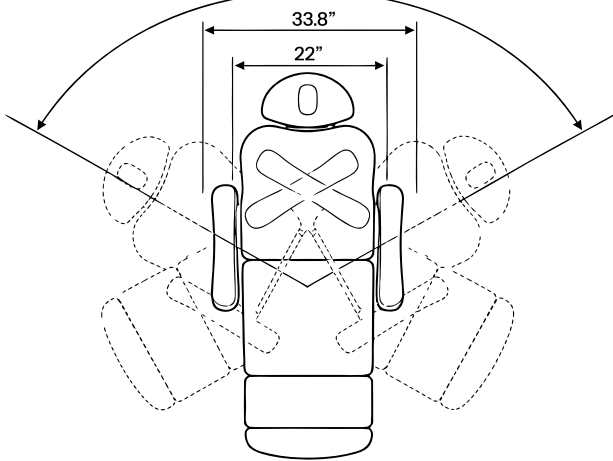

Operation Cautions

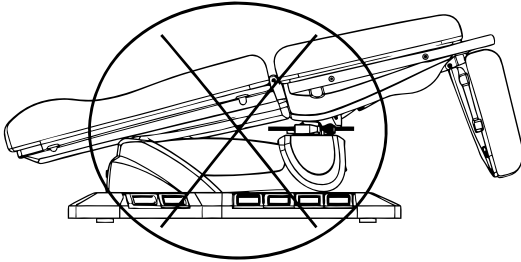

Do not tilt the seat cushion while the chair is in its lowest position.




Warning: Do Not Sit or Apply Excess Force

Do not sit, stand, jump, or place excessive weight on the backrest or footrest. Doing so may cause the product to tip, damage the backrest/footrest joints, or result in injury.




If the headrest support rod becomes loose, follow the steps below to secure it before use.

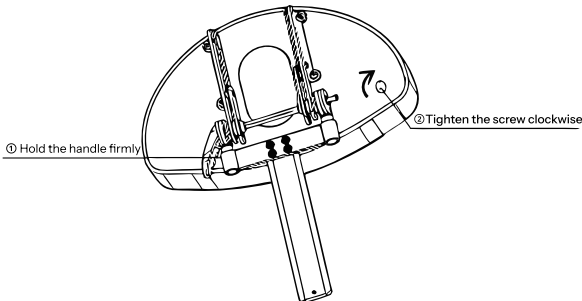


Reset Function Notice

Before starting the reset function, make sure the foot extension is fully closed and returned to its original position. Do not activate the reset function while the foot extension is open or partially extended, as this may affect the chair's movement or cause improper positioning.


How to Tighten the Brake Block

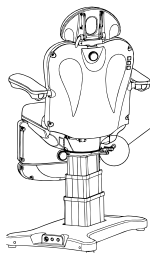


The brake block is located underneath the seat of the treatment chair/table on the right side.

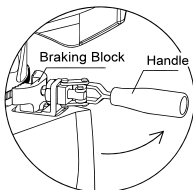


1. Raise the backrest to its highest position. Then, release the locking block by pushing the handle in the counterclockwise direction, as shown in the image above.

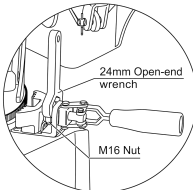


2. Hold the handle securely with your right hand. While holding the handle in place, use an open-end wrench to loosen the M16 nut by turning it clockwise.

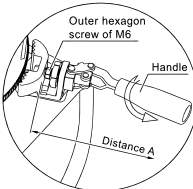

3. Hold the M6 hexagon screw with your left hand to prevent it from rotating. At the same time, rotate the handle 360° counterclockwise with your right hand to increase the clearance.

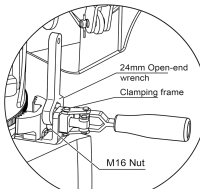


4. Keep the clamping frame level by holding the handle with your right hand. While holding it in place, use an open-end wrench to tighten the M16 nut by turning it counterclockwise.

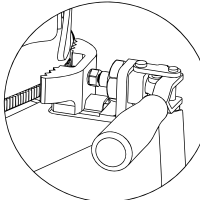

5. Tighten the handle by turning it clockwise, as shown in the image.


Remote Control Instructions

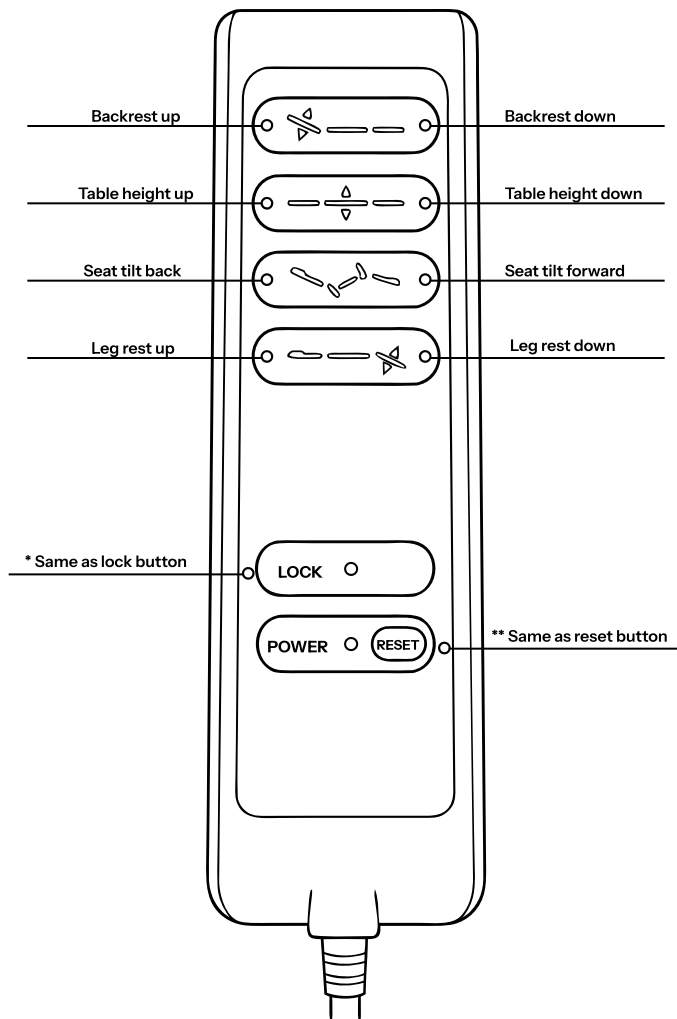


* Press and hold the lock button for 3 seconds. The motors will be locked and no buttons will move the chair/table. Press and hold the lock button again to unlock the motors.

** The reset button automatically returns the unit to the seated chair position and lowest height. It is recommended the chair be in this position when a client gets on or off the chair.

Foot Pedal Controls

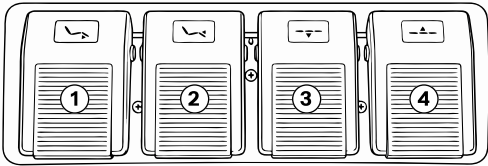


- 1. Leg Rest Down ↓
- 2. Leg Rest Up ↑
- 3. Table Height Down ↓
- 4. Table Height Up ↑

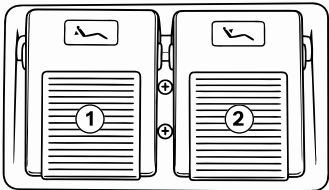


- 1. Backrest Down ↓
- 2. Backrest Up ↑


Chair Controls

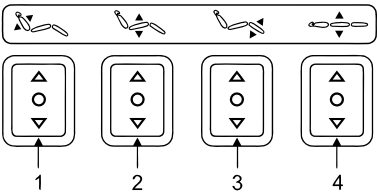

- 1. Backrest Up & Down ↑↓
- 2. Seat Tilt Forward and Back →←
- 3. Leg Rest Up & Down ↑↓
- 4. Table Height Up and Down ↑↓

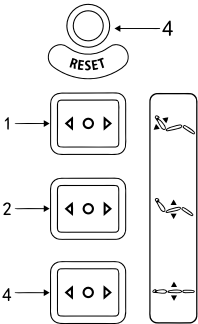



- 1. Backrest Up & Down ↑↓
- 2. Seat Tilt Forward and Back →←
- 3. Table Height Up and Down ↑↓
- 4. Reset Position

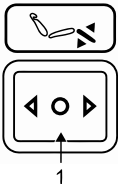

- 1. Leg Rest Up & Down ↑↓

Motor Controls

- The backrest includes controls for the backrest angle, seat tilt, and table height.
- A reset button is located above the backrest controls. When pressed, the unit automatically returns to the seated chair position and lowers to the lowest height. This position is recommended when a client is getting on or off the chair.
- Before a client gets on or off, make sure the chair rotation is locked.
- To make entry and exit easier, swing one armrest out of the way as needed.
- The seat cushion includes controls for the backrest, seat tilt, leg rest angle, and table height.
- A lock button is located next to the seat cushion controls. Press and hold the button for 3 seconds to lock all chair motors. While locked, the motors will not move the chair.
- Press and hold the lock button again for 3 seconds to unlock the motors and restore all motor functions.
- The leg cushion includes a button for adjusting the leg rest angle.

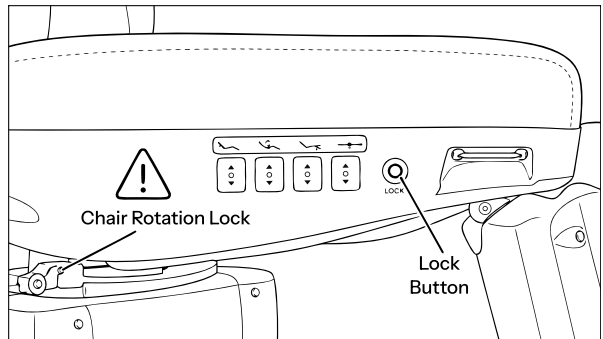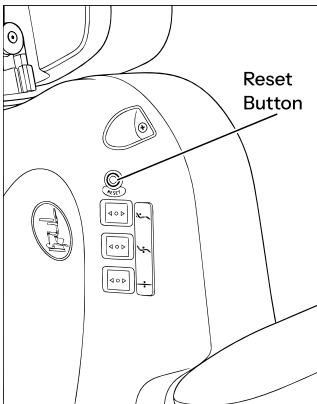

Maintenance

- When cleaning and sanitizing, do not use bleach, Barricade, or other harsh chemicals. Using harsh chemicals can damage the upholstery.
- For best results, clean with **SkinAct All-In-One Upholstery & Disinfecting Multi-Purpose Cleaner** (SKU24261).
- Clean and sanitize the upholstery after each treatment.
- Every six months, visually inspect all visible parts of the unit. If there are any abnormal noises or vibrations during operation, call Spa & Equipment/SkinAct at 1-800-418-0830.

Troubleshooting

Motorized Sections Do Not Move

Solution: Confirm that the hand or foot remote is securely connected and responding properly. If the remote does not respond, replace the remote or contact SkinAct/Spa & Equipment technical support for assistance.

Control Box Not Working

Solution: If the motorized sections do not move after confirming the remote and power connection, discontinue use and contact Spa & Equipment/SkinAct.

Abnormal Noise During Operation

Solution: Apply lubricating oil to rotating or moving parts such as hinges, lift areas, or footrest joints. If the noise continues after lubrication, stop using the product and contact SkinAct/Spa & Equipment.

Motor Performance Issue

Solution: If the motor operates irregularly, makes unusual noise, or fails to move smoothly, discontinue use and contact Spa & Equipment/SkinAct.

 Refer to instruction manual	 Manufacturer
 WEEE	 CE
 Date of Manufacture	 Type B Applied Part
 Warning	 Serial No.
 Atmospheric Pressure for Use	 Temperature Limit
 Keep dry / Protect from Rain	 Fragile, handle with care
 Humidity Limitation	 Stacking limit by mass of 250kg
 Keep away from sunlight	 Do Not Tumble
 This Side Up	 Handle with Care
 = kg Maximum patient weight	 Class II equipment
 UL Mark	 Warning: Hand Crush Hazard
 Warning: Mechanical Injury Hazard	 Do Not Sit

Specifications

Product Name:	Palma Electric Medical Spa Treatment Chair/Table
Voltage:	AC 100-120V
Frequency:	50/60Hz
Power Consumption:	200 W
Backrest Angle:	0° flat to 80° incline
Leg Rest Angle:	0° flat to 85° decline
Height Range:	25”- 37”
Seat Tilt Angle:	0° to 10°
Rotation:	120° (60° to the left, 60° to the right)
Base Dimensions (L x W):	39” x 28.5”
Width with armrest:	32”
Width without armrest:	24”
Length with leg rest & headrest extended out:	80.5”
Length with leg rest & headrest not extended out:	74”
Length without leg rest or headrest attached:	57.5”
Weight Capacity:	440 lbs
Net Weight:	274 lbs

Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor’s discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

Warranty does not cover damage, contamination, upholstery damage, sanitation-related issues, injury claims, or equipment issues resulting from improper cleaning, improper maintenance, misuse, use by untrained users, or use outside applicable safety guidelines. This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

SKIN+ACT

Phone:1-800-418-0830
Email:info@skinact.com
Website:www.skinact.com