


SKIN+ACT

**Ultra 6 in 1 Microdermabrasion + Skin
Scrubber + Bipolar RF + Mesotherapy
+ Cooling Treatment + Bio-Lift**

PRODUCT MANUAL

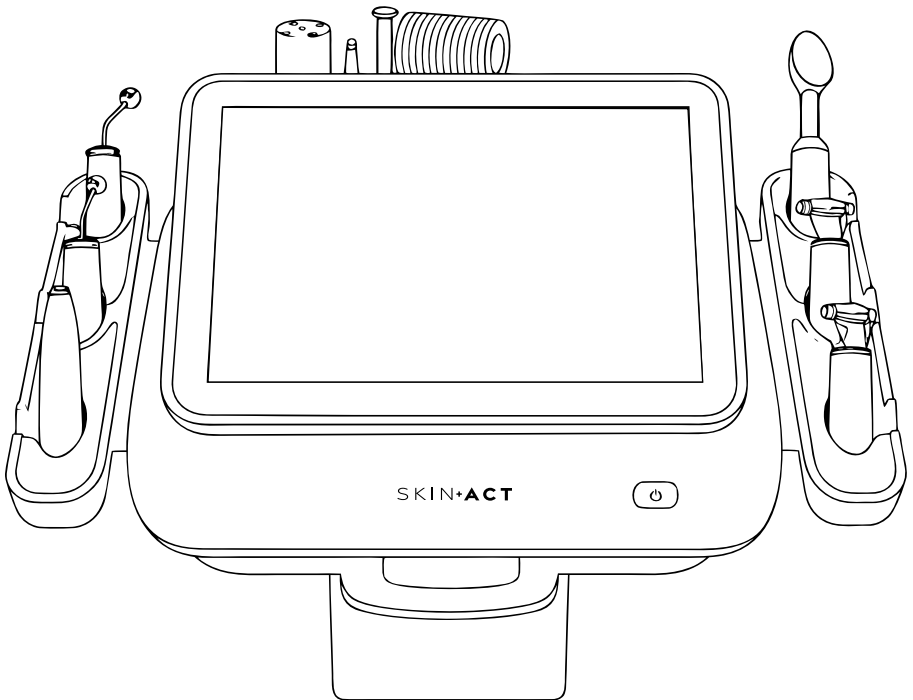

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

Copyright © 2026 SkinAct, All Rights Reserved

TABLE OF CONTENTS

Disclaimer 4

Contraindications & Precautions 5

Areas to Avoid & Important Safety Notes 6

Included Items 7

Benefits of Each Handle 8

Operating Instructions 9

Treatment Guidelines

1. Microdermabrasion 11

2. Skin Scrubber 12

3. Bipolar RF 13

4. Needleless Mesotherapy Facial 14

5. Needleless Mesotherapy Eye 15

6. Cooling Treatment 16

7. BIO Round Microcurrent Eye 17

8. BIO Roller Microcurrent Face 18

Important Notes 20

Troubleshooting 21

Specifications 22

Warranty Information 23

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH:

- Pacemakers, metallic implants, or other electronic medical devices.
- Heart conditions or a history of heart disease.
- Pregnancy.
- Inflamed, irritated, or damaged skin.
- Eczema, dermatitis, swelling, or sunburn.
- Clients who have received hydroxy acid treatments, laser resurfacing, chemical peels, or wound-healing procedures within the past 3 months.
- Over areas with permanent or temporary dermal fillers unless approved by a physician.
- Clients with skin cancer, suspicious lesions, or precancerous conditions.
- Clients currently using blood-thinning medications or who have taken isotretinoin (retinoic acid) within the past 6 - 12 months.
- Clients with asthma, cold, flu or upset stomach.
- Clients with epilepsy, alcoholism, ulcers or digestive system disorders.

Precautions

- **Inspect the Device:** Check for any visible damage to the device or its power cable before and after each use.
- **Unplug When Not in Use:** Always disconnect the device from the power source before cleaning, storing, or performing maintenance.
- **Avoid Humidity:** Do not operate the device in excessively humid environments.

- **Position Cables Safely:** Ensure the power cable is positioned to prevent tripping hazards and is plugged into a grounded socket.
- **Keep Away from Children:** The device and its accessories should be stored securely out of reach.
- **Report Issues Promptly:** If you notice abnormal functioning, contact your distributor or an authorized technician immediately.
- **Remove Jewelry:** All jewelry and metal piercings must be removed prior to performing microcurrent treatments.

Areas To Avoid During Treatment


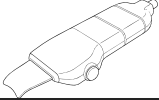

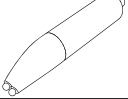
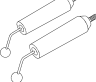
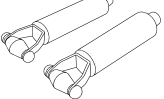
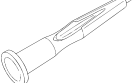
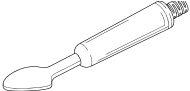
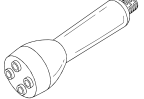
- Sensitive Skin
- Skin rashes, damaged, inflamed, or infected areas
- Surgical Scars less than 12 months old
- Swollen Skin
- Eyes and eyelid area (probes should never point directly at them)

▪ Genitals






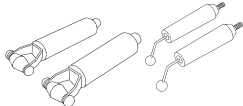
Important

- Always start with the lowest settings and gradually increase based on your skin's reaction.
- Mild swelling and redness may occur after treatment but should disappear within a few hours.
- Overly aggressive techniques can cause temporary flaking or bleeding.
- Avoid microdermabrasion function on eyelids (keep them closed and covered during treatment).
- Certain areas are not suitable for treatment: skin tumors, moles, birthmarks, lips, eyes, herpes, skin cancer and liver spots.

Included Items

Qty	Part Name	Image
1	Host Machine	
1	Skin Scrubber	
1	Cooling Therapy Hammer	
1	Mesotherapy Eye Tool	
2	BIO Round	
1	BIO Roller	
1	Microdermabrasion Pen	
1	Bipolar RF	
1	Mesotherapy Face Tool	

Benefits of Each Handle

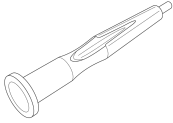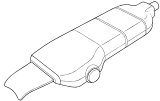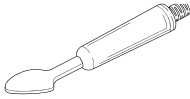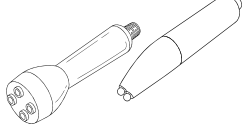
<p>Microdermabrasion Gently exfoliates the skin to remove dead cells and improve skin texture.</p>	
<p>Skin Scrubber Helps cleanse the skin and remove surface buildup, excess oil, and debris.</p>	
<p>Bipolar RF (Radio Frequency) Helps improve the appearance of firmer, smoother looking skin. Delivers controlled radio frequency energy for firmer-looking skin.</p>	
<p>Needleless Mesotherapy Helps support product application and the appearance of hydrated-looking skin. Two different tools for larger areas and for around the eyes.</p>	
<p>Cooling Treatment Soothes and calms the skin, reducing redness, inflammation, and irritation after other treatments or exposure to environmental stressors.</p>	
<p>Microcurrent BIO lift Helps improve skin elasticity and support product application during treatment. Two different tools for larger areas and for around the eyes.</p>	

Operating Instructions

1. Locate the ports on the back of the main unit and connect each handle to its corresponding port, matching the handle to the icon displayed next to it.
2. Turn on the main power switch located on the back of the unit. Then press the front power button to activate the screen.
3. The home screen will display the following available handles:
 - Dermabrasion, Scrubber, Bipolar RF, Mesotherapy Face, Mesotherapy Eye, Cooling treatment, BIO Roller and BIO Round



3. Select your desired handle attachment. The treatment screen will appear, showing the following controls:
 - Treatment Area: Face or Neck
 - Function Mode: Lifting, Firming, Repair, or Anti-Aging
 - Intensity Level: Adjust according to client comfort and skin condition.
 - Treatment Time: Set the desired session duration using the timer controls.
4. Configure your session settings and press Start to start the treatment.

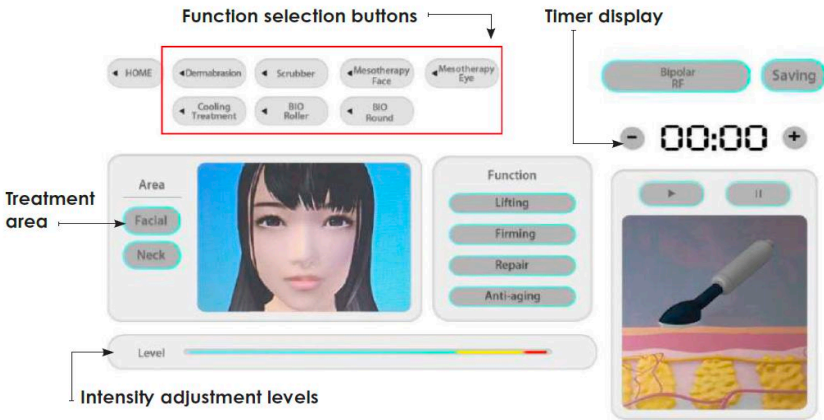


5. Tap **Saving** to store your current settings. When prompted with “Sure to save?”, select Yes to confirm.



Treatment Guidelines

1. Microdermabrasion

Follow the movement directions shown in the examples.

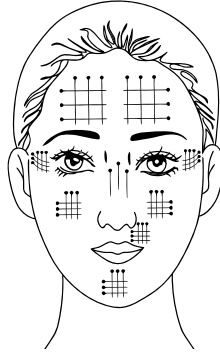

Preparing for Treatment

- 1. Choose the Dermabrasion Attachment:** Select the appropriate attachment for the desired treatment area.
- 2. Set Treatment Time:** Set the treatment time for 15 - 20 minutes.
- 3. Adjust Vacuum Level:** The vacuum level should be adjusted based on your client's comfort level. Start with a lower suction for sensitive areas like the eye area.
- 4. Ensure the filters are clean and in place for proper suction**

Treatment

- **Test Suction**

Before applying to the face, test the suction level on your arm to ensure it's comfortable.

- **Treatment Technique**

Move the tip in smooth, circular motions, avoiding any area for more than 15 seconds. This helps prevent irritation or bruising.

▪ Post-Treatment Care

Clean the diamond tip with a brush and water after use. Disinfect it by soaking it in a 75% alcohol solution for 15 minutes, followed by wiping it dry and placing it in a UV sterilizer.

Important Note: These are general guidelines. Communicate with your client throughout the treatment and adjust settings based on their comfort and specific needs.

2. Skin Scrubber



Follow the movement directions shown in the examples.




Treatment

- 1. Choose Scrubber Attachment:** Select the appropriate attachment for your desired treatment.
- 2. Adjust Settings:** Set the treatment time for 15 - 20 minutes.
- 3. Choose the treatment area, function, mode, and intensity level based on your client's needs and preferences.**
 - Operating at 28,000 Hz, the device helps cleanse the skin and improve the appearance of smoother, refreshed looking skin.

Important Note: These are general guidelines. Communicate with your client throughout the treatment and adjust settings based on their comfort and specific needs.

3. Bipolar RF

Follow the movement directions shown in the examples.

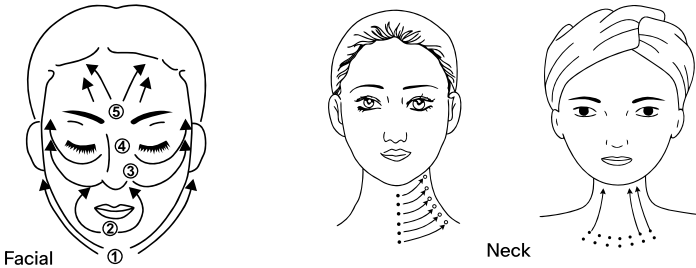

Facial

1. **Choose Bipolar RF:** Select the appropriate attachment for facial treatment.
2. **Set Treatment Time:** Set the treatment time for 15 - 20 minutes.
3. **Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
4. **Apply Medium:** Apply a conductive medium to the treatment area.
5. **Move Probe:** Glide the probe in the correct areas shown above in smooth, circular motions.

Neck

1. **Set Treatment Time:** Set the treatment time for 15 - 20 minutes.
2. **Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
3. **Apply Medium:** Apply a conductive medium to the neck area. Avoid using over the thyroid.
4. **Move Probe:** Glide the probe in the correct areas in smooth, upward motions.


IMPORTANT

Avoid Prolonged Contact: Do not keep the probe stationary in one place for an extended period.

Monitor Client Comfort: Continuously communicate with your client during the treatment and adjust.

4. Needleless Mesotherapy Facial

Follow the movement directions shown in the examples.

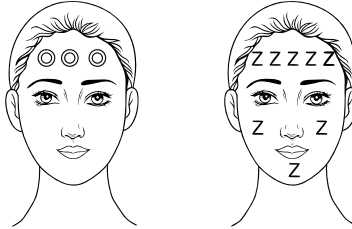

- 1. Choose Mesotherapy:** Select the appropriate attachment for your desired treatment.
- 2. Set Treatment Time:** Set the treatment time for 15 - 20 minutes.
- 3. Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
- 4. Apply Product:** Apply a suitable skincare product directly to the treatment area, or fill the handle's bottle with skincare product by twisting the handle to remove the bottle, filling it, and re-attaching securely before use.
- 5. Move Probe:** Glide the probe over facial skin in smooth, circular motions in the directions shown in the figure.

IMPORTANT


Monitor Client Comfort: Continuously communicate with your client during the treatment and adjust intensity.

Avoid Prolonged Contact: Do not keep the probe stationary in one place for an extended period.

Consult with a Dermatologist: If you have any concerns about using mesotherapy on your clients, it is recommended to consult with a dermatologist.

5. Needleless Mesotherapy Eye

Follow the movement directions shown in the examples.

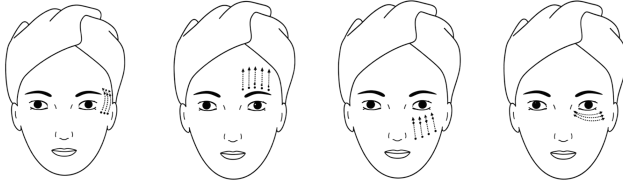

1. **Choose Mesotherapy:** Select the appropriate attachment for your desired treatment.
2. **Set Treatment Time:** Set the treatment time for 10 - 15 minutes.
3. **Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
4. **Apply Product:** Apply a skincare product to the treatment area.
5. **Move Probe:** Glide the probe over facial skin in smooth, circular motions in the direction shown in the figure above.

IMPORTANT

Monitor Client Comfort: Continuously communicate with your client during the treatment and adjust.

Avoid Prolonged Contact: Do not keep the probe stationary in one place for an extended period. Avoid direct contact with the eyeball.

Consult with a Dermatologist: If you have any concerns about using mesotherapy on your clients, it is recommended to consult with a dermatologist.

6. Cooling Treatment

Follow the movement directions shown in the examples.



- 1. Choose Cooling Hammer:** Select the appropriate attachment for facial cooling treatment.
- 2. Set Treatment Time:** Set the treatment time for 10 - 15 minutes.
- 3. Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
- 4. Move Probe:** Glide the probe in the correct areas shown in the figure above with smooth, circular motions.


IMPORTANT

Monitor Client Comfort: Continuously communicate with your client during the treatment and adjust settings as needed.

Avoid Prolonged Contact: Do not keep the probe stationary in one place for an extended period.

7. BIO Round Microcurrent Eye

Follow the movement directions shown in the examples.

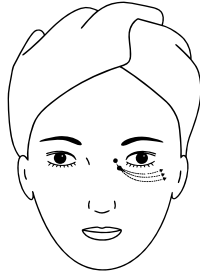

- 1. Choose BIO Round:** Select the appropriate attachment for BIO Round treatment.
- 2. Set Treatment Time:** Set the treatment time for 10 - 15 minutes.
- 3. Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
- 4. Position Probe:** Begin at the inner corner of the eye, along the orbital bone.
- 5. Glide Outward:** Move the probe smoothly outward and upward along the orbital bone towards the temple engaging the levator muscles to lift and tone the eye area.
- 6. Apply Light Pressure:** Use gentle, consistent pressure throughout the movement. For the eyebrow area, an optional pinch technique can be applied to target the depressor muscles.


IMPORTANT

Monitor Client Comfort: Continuously communicate with your client during the treatment and adjust settings as needed.

Avoid Eye Contact: Keep the probe on the orbital bone at all times and avoid direct contact with the eyeball.

7. BIO Roller Microcurrent Face

Follow the movement directions shown in the examples.

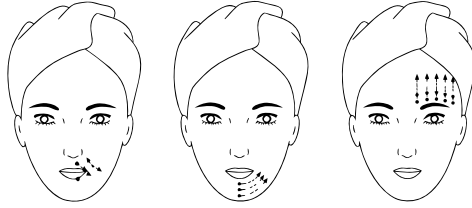

1. **Choose BIO Roller:** Select the appropriate attachment for BIO Roller treatment.
2. **Set Treatment Time:** Set the treatment time for 10 - 15 minutes.
3. **Adjust Intensity:** Adjust the intensity level based on your client's comfort and skin sensitivity.
4. **Apply Product:** Apply a skincare product to the treatment area.

Nasolabial Folds:


- **Lift and sculpt:** Glide the probe diagonally upward from the sides of the nostrils toward the cheekbones, engaging the levator labii superioris to smooth and lift the nasolabial fold area.

Jawline:

- **Position Probe:** Begin at the center of the chin along the jawline.
- **Glide Upward:** Move the probe smoothly upward along the jawline toward the back of the ear, stimulating the depressor anguli oris to improve definition and contour.
- **Pinch Technique:** Apply the optional pinch technique along the jawline to further engage the underlying muscles and enhance definition.

Forehead:

- **Glide Upward:** Beginning at the center of the eyebrows, move the probe smoothly upward towards the hairline, engaging the frontalis muscle to lift and smooth the forehead and reduce the appearance of horizontal lines.

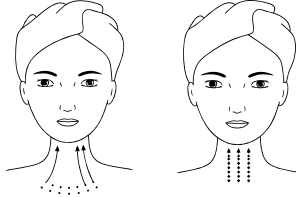**Neck:**

- **Stroke Upward:** Using low to moderate intensity, move the probe in smooth upward strokes from the base of the neck toward the jawline, targeting the platysma muscle to lift and firm the appearance of the skin.
- **Apply Gentle Pressure:** Maintain light, consistent pressure throughout due to the sensitivity of the neck area.

IMPORTANT: Avoid using the device over the Thyroid area, as stimulation in this region may pose serious health risks.

Important Notes

- **Cleanse the skin:** Before treatment, ensure the skin is clean and free of makeup or other products.
- **Apply a serum:** For enhanced results, apply a serum or a water-based conductive gel before using both microcurrent handles. SkinAct serums are recommended for use with this treatment.
- **Monitor Client Comfort:** Continuously check with your client to ensure they are comfortable throughout the treatment.
- **Avoid sensitive areas:** Avoid rolling the device over areas with broken skin, moles, warts or other skin conditions.
- **Clean and disinfect:** After the treatment, clean and disinfect the device and attachments.

Troubleshooting

Machine does not respond when powered on

Solution: Ensure the power supply is securely connected to the machine. Confirm the power switch is turned on.

No Output

Solution: Check that the attachment is properly and fully connected to the correct output port.

Machine overheats during use

Solution: Power off the unit immediately and allow it to cool down naturally before resuming use. If overheating persists, have a qualified technician check that the voltage supply meets the required specifications.

No Power

Inspect the fuse and replace it if damaged.

Specifications

Product Name:	Ultra 6 in 1 Microdermabrasion + Skin Scrubber + Bipolar RF + Mesotherapy + Cooling Treatment + Bio-Lift
Voltage:	AC 110-120V
Frequency:	60Hz
Power Consumption:	75W
Dimensions(L*W*H):	14.2 x 12 x 6.5"
Net Weight:	27 lbs

Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor’s discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

SKIN+ACT

Phone:1-800-418-0830

Email:info@skinact.com

Website:www.skinact.com