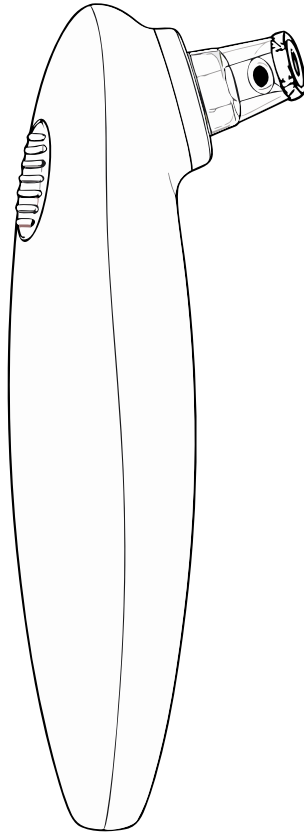


SKIN+ACT

HydroSculpt Glow

Diamond Microdermabrasion + Blue LED Light Therapy

PRODUCT MANUAL & PROTOCOL GUIDE



NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

TABLE OF CONTENTS

Disclaimer & Contraindications.....4

Precautions.....5

Included Items.....6

Device Components.....7

Interchangeable Tip Guide.....8

Installation & Setup.....9

Professional Treatment Protocol.....10

Recommended Treatment Times.....12

Cleansing, Disinfection & Maintenance.....13

Troubleshooting.....14

Specifications.....15

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH:

- Active acne lesions (inflamed pustules, papules, or cystic acne).
- Open wounds or broken skin, sunburned skin.
- Active herpes simplex/cold sores.
- Rosacea flare-ups or highly reactive rosacea skin.
- Severe telangiectasia/couperose skin.
- Eczema, dermatitis, or psoriasis in the treatment area.
- Recent chemical peel or aggressive exfoliation.
- Recent laser resurfacing or IPL treatment.

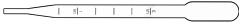
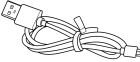
- Recent waxing, sugaring, or depilatory use on the area.
- Use of isotretinoin (Accutane) within the past 6–12 months.
- Fragile, sensitized, or impaired skin barrier.
- Active skin infections (bacterial, fungal, or viral).
- Suspicious lesions or undiagnosed skin irregularities.
- Skin cancer in the treatment area.
- Uncontrolled diabetes with impaired wound healing.
- Blood clotting disorders or anticoagulant therapy causing excessive bruising risk.
- Severe inflammation or irritation of the skin.
- Recent facial surgery or unhealed scar tissue.
- Keloid or hypertrophic scar tendency.
- Extremely thin or atrophic skin.
- Recent injectable treatments unless medically cleared and fully healed.
- Active allergic skin reactions or rashes.
- Severe couperose or highly vascular skin conditions.
- Clients unable to communicate excessive discomfort or sensitivity during treatment.
- Blood Thinning medications.

Precautions

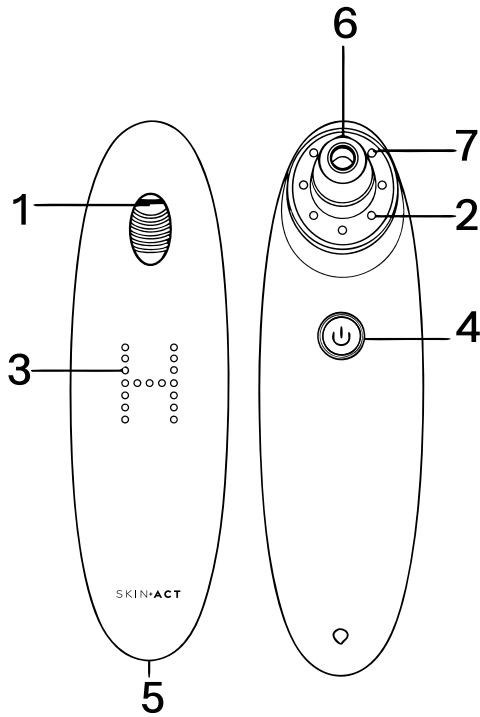
- **No Multi-Client Component Sharing:** To prevent cross-contamination and serious skin infections, never reuse a tip on another client without proper sterilization.
- **Avoid Abrasive Combinations:** Do not use physical scrubs, mechanical brushes, or aggressive chemical exfoliants immediately before or after a HydroSculpt Glow treatment. Wait 14–21 days after a chemical peel, or 7–14 days after other exfoliating treatments.
- **Avoid Excess Pressure:** Do not press the diamond tip firmly into the skin. Allow the vacuum suction to gently draw the skin into contact with the diamond tip for proper exfoliation.

- **Electrical Warning:** Keep your hands completely dry during operation and while connecting the device to prevent water intrusion through the charging port.
- **Post-Treatment Erythema:** Mild redness, warmth and superficial tingling are normal responses due to accelerated microcirculation. This typically resolves on its own; Focus on hydration & barrier support at this time. Advise client to keep the skin cool, hydrated, and protected with a broad-spectrum SPF.

Included Items

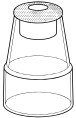
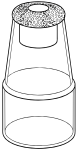
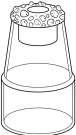
Qty	Part Name	Image
1	HydroSculpt Device	
1	Fine Diamond Microdermabrasion Tip	
1	Medium Diamond Microdermabrasion Tip	
1	Coarse Diamond Microdermabrasion Tip	
1	Small Round Suction Tip	
1	Oval Suction Tip	
1	Water Dropper	
1	USB Charging Cable	
2	Sponge Filter	

Device Components



- | | |
|-------------------------|--------------------------------|
| 1. Water Injection Hole | 2. Blue Light Therapy LED |
| 3. Intensity Indicator | 4. Suction Adjust/Power Button |
| 5. Charging Port | 6. Mist Nozzle |
| 7. Diamond/Suction Tip | |

Interchangeable Tip Guide

Fine Diamond Microdermabrasion Tip An ultra-fine, low-abrasion diamond particle matrix. Optimized for delicate, thin, or sensitive skin profiles. Delivers a mild epidermal sweep to minimize pores and brighten skin tone without causing excessive stress or lingering erythema.	
Medium Diamond Microdermabrasion Tip An intermediate-gauge diamond abrasive particle matrix. This is the standard selection for general skin rejuvenation, addressing fine lines, superficial wrinkles, and mild sun damage across normal, combination, and mature skin texture.	
Coarse Diamond Microdermabrasion Tip Features a heavy-gauge, coarse diamond abrasive particle matrix. Engineered for keratinized skin, thick, oil or resilient skin types. Ideal for treating deep textural scarring, severe congestion and stubborn dead skin cells. Avoid on thin or couperose areas.	
Small Round Suction Tip Designed with a narrow, focused tip with smooth contact margins, Concentrates vacuum pressure directly over localized follicular blockages. Ideal for precise extractions on sides of nose, chin cleft and isolated comedones.	
Oval Suction Tip Features an elongated, wide-tip contact head. Provides wide suction distribution across larger surface zones (cheeks, forehead & neck) Ideal for broad fluid extraction, lymphatic drainage pathways, and overall lifting strokes.	

Installation & Setup

- 1. Inspect the Filtration System:** Ensure a clean, dry Sponge Filter is properly placed inside the main unit tip receptacle before attaching any treatment tip.
- 2. Select & Mount the Tip:** Analyze the client's skin condition. Firmly press the chosen Diamond Microdermabrasion or Suction tip onto the device nozzle until it forms an airtight seal.
- 3. Fill the Hydration Reservoir:** Fill the included Water Dropper 3 mL with distilled water or an approved low-viscosity hydration solution. Open the silicone seal covering the Water Injection Hole. Carefully inject 3 mL using the water dropper of fluid into the port. Close the silicone seal securely.
- 4. Turn On: Press and hold the Suction Adjust / Power Button for exactly 3 seconds.** The device will activate, and the LED indicators will light up.
- 5. Adjust Intensity: Tap the Power Button** sequentially to cycle through the intensity tiers: Low - Medium - High. Test the suction on a gloved hand or wrist to confirm proper fluid flow and consistent vacuum pressure before beginning treatment.

Professional Treatment Protocol

For optimal clinical results, always execute a classic five-step hydro-dermabrasion sequence: Cleanse, Exfoliate, Extract, Hydrate, and Serum Infusion.



Step 1: Cleanse & Prep

Remove surface oils, makeup, and debris to clear the path for treatment. Apply a professional cleanser suited to the client's skin type. Remove thoroughly using a facial sponge or soft esthetic wipes. Apply a warm, moist cotton pad or steamed facial towel to the treatment area 2 – 3 minutes to soften the sebum plugs within the pores.

Step 2: Controlled Exfoliation

Mount the Fine, Medium or Coarse Diamond Microdermabrasion tip. Begin with the device set on Low Vacuum Level. Hold skin taut using your index finger and thumb, using your non-dominant hand. Begin on the forehead and progress downward to the lower face, neck, and décolleté. Glide the device using overlapping cross-hatch passes, combining inward and outward movements to ensure even treatment coverage. Remove any - residue & dead skin by using a damp 2x2 cotton pad and toner of choice.

Step 3: Targeted Extraction

Mount the small round suction tip and set the system to Low or Medium vacuum level (depending on tissue tolerance). Focus on congested areas such as the nose, chin and the central forehead. Glide the tip slowly across visible blackheads.

Professional Rule: Keep the treatment tip moving continuously using smooth, overlapping passes. Never hold the vacuum tip stationary on a single area of skin. Continuous movement is required, as remaining in one spot can cause rupture of capillaries (petechiae, telangiectasia, or reddish-purple bruising). If any discoloration appears, stop the treatment immediately.

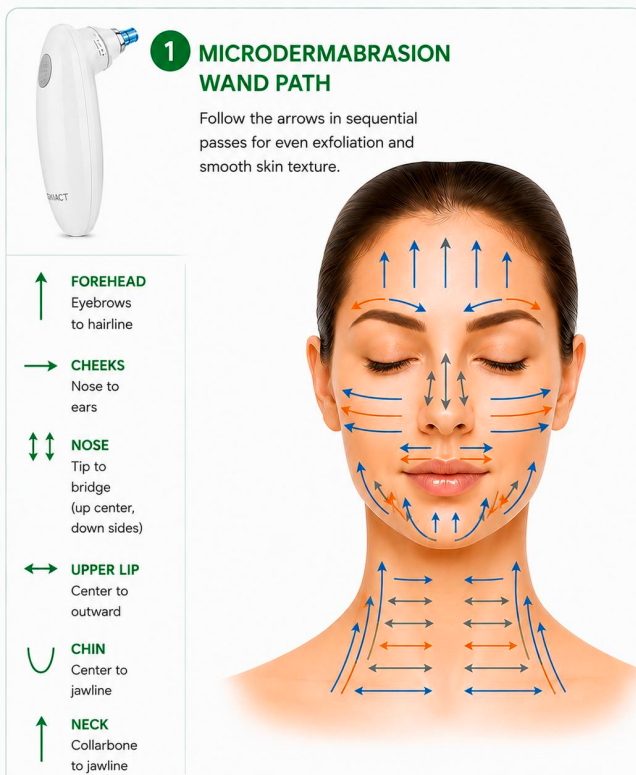
Step 4: Hydration Flush

Mount the Oval Suction Tip and set to Low or Medium vacuum level. Ensure the water reservoir is full. Glide the treatment tip slowly across the cheeks, forehead, and neck using smooth, overlapping motions. This spray operates exclusively when held off the skin, pausing automatically upon contact and resuming when lifted.

Step 5: Professional Serum Infusion

After completing treatment, turn off the device and apply a professional finishing serum such as hyaluronic acid, peptides, or antioxidant formulations. Gently pat or massage into the skin until fully absorbed.

Step 6: Apply the appropriate moisturizer and SPF to conclude the service.



Recommended Treatment Times

To prevent over-exfoliation and minimize excessive post-treatment erythema, strictly adhere to these zonal time limits

Treatment Zone	Maximum Passes	Recommended Stroke Technique
Forehead	1 - 3	Upward and outward sweeping paths
Cheeks	2 - 3	Lateral, upward strokes toward the tragus
Nose	1 - 3	Downward/side sweeps along the bridge and sides of nose
Chin	2 - 3	Circular and upward sweeping glides across the cleft
Neck	1 - 2	Gentle upward & outward glides, completely avoiding the midline thyroid area

Total Face Treatment Time Limit: 10 - 15 Minutes

Note: The presence of erythema will indicate sufficient passes have been completed.

Cleaning, Disinfection & Maintenance

To preserve vacuum integrity and comply with salon sanitation laws, perform the following cleaning and disinfection procedure after every treatment.

Immediately After Every Use:

1. Power down the device, Carefully remove and dispose of the used sponge filter.
2. Rinse the tip thoroughly under warm running water to clear away trapped skin cells and sebum. Use a small brush with warm water and soap for further cleaning. Submerge the tip in an EPA-registered, hospital-grade disinfectant solution (following the manufacturer's exact dilution and contact time rules)
3. Wipe down the main unit chassis using a clean, soft cloth lightly dampened with sanitizer.

Warning: Never rinse or submerge the main device body in water or any other liquid.



Suction Filter Maintenance:

The suction nozzle houses a protective Sponge Filter to block fluid and debris from entering the internal motor chamber. Ensure the sponge filter is dry before inserting within the chamber, to prevent suction blockages. To maintain safety and machine longevity, replace the sponge filter after every client. Use a clean, sharp pair of tweezers to carefully extract the sponge filter from the nozzle head and insert a fresh one from your package.

Long-Term Storage:

Store the device in a cool, dry environment completely free from ambient moisture, steam or direct sunlight. Ensure the unit is kept entirely out of reach of children.

Troubleshooting

Device Does Not Turn On

Solution: Connect the supplied USB cable to a compatible DC 5V/1A power adapter and fully charge the device.

Poor Extraction or Difficulty

Solution: Apply a warm towel, steam, or a skin-softening solution before treatment to help loosen debris and sebum. Use a professional desincrustation lotion or steamer when needed. Make slow, consistent passes and do not force extractions in one session.

Weak or Insufficient Suction

Solution: Fully charge the device before use. If suction remains weak after charging, stop using the device and contact customer service.

No Solution Flow or Hydration Spray

Solution: Open the reservoir port and add 3 mL of pure distilled water. Confirm that the internal sponge filter is dry, properly positioned, and not blocking the fluid line.

Specifications

Product Name:	HydroSculpt Glow
Voltage:	AC100-240V
Frequency:	50/60Hz
Output Power Profile:	DC 5V/1A
Rated Battery Capacity:	500 mAH
Full Charging Duration:	Approximately 2.5 hours
Peak Vacuum Suction Force:	57-65 kPa
Dimensions(L*W*H):	6.2 x 1.8 x 1.9”
Net Weight:	0.3 lbs