


SKIN+ACT

High Frequency Table Top Facial Unit

PRODUCT MANUAL

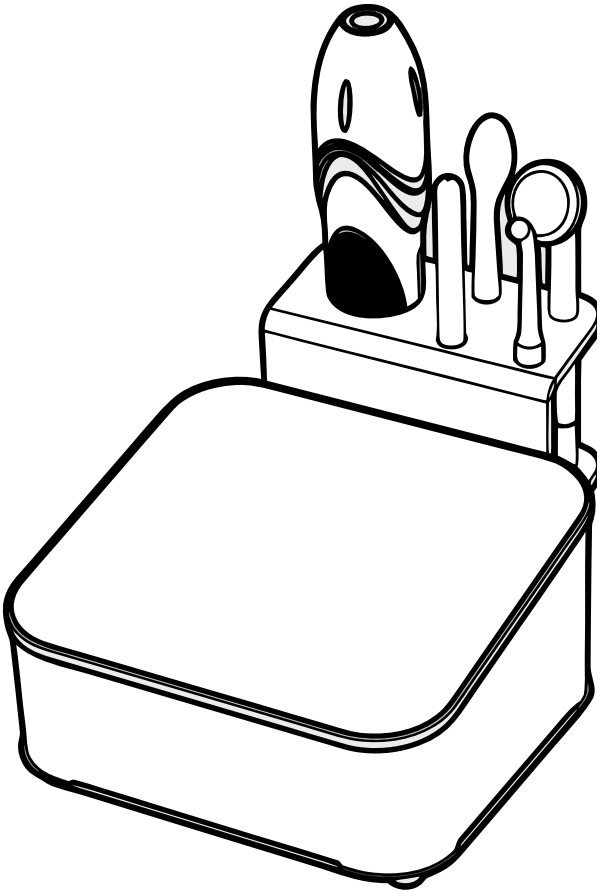

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

TABLE OF CONTENTS

Disclaimer 4

Contraindications & Precautions 5

Electrotherapy Operation Guidelines 6

Safety During Operation 6

Safety Tips 7

Warnings 8

Included Items 9

Device Components 10

Glass Electrotherapy Tubes 11

Touch Panel 12

Device Assembly 13

Direct Electrotherapy 14

Effects of Direct Electrotherapy & Indirect Electrotherapy 15

Effects of Indirect Electrotherapy & Specifications 16

Warranty Information 17

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH:

- Pacemakers, metallic implants, or other electronic medical devices.
- Heart conditions or a history of heart disease.
- Pregnancy.
- Inflamed, irritated, or damaged skin.
- Severe acne rosacea or vascular conditions.
- Epilepsy or other seizure disorders.
- Clients undergoing medical treatments or with underlying health conditions must obtain prior clearance from a physician.

Precautions

- **Inspect the Device:** Check for any visible damage to the device or its power cable before and after each use.
- **Unplug When Not in Use:** Always disconnect the device from the power source before cleaning, storing, or performing maintenance.
- **Avoid Humidity:** Do not operate the device in bathrooms or other excessively humid environments.
- **Position Cables Safely:** Ensure the power cable is positioned to prevent tripping hazards and is plugged into a grounded socket.
- **Keep Away from Children:** The device and its accessories should be stored securely out of reach.
- **Report Issues Promptly:** If you notice abnormal functioning, contact your distributor or an authorized technician immediately.

Electrotherapy Operation Guidelines

- **Normal Operation:** The electrotherapy handle emits a continuous “buzzing” sound during use.
- **Sparking (Direct Use):** Excessive sparking can cause discomfort, irritation, or minor burns. Keep the electrode in contact with the skin and reduce intensity if sparking becomes uncomfortable.
- **Energy Adjustment:** Begin at the lowest energy level before bringing the electrode into contact with the skin, then increase gradually.
- **Sensitive Skin:** If discomfort occurs, place a dry cotton gauze over the treatment area to reduce irritation.
- **Skin Contact:** Ensure the electrode maintains full contact with the skin to prevent excessive sparking.
- **Treatment Duration:**
 - **Oily skin:** approximately 8 minutes
 - **Dry skin:** 3–5 minutes
- **Intensity Level:** Use higher intensity for oily skin and lower intensity for dry skin.
- **Indirect Use:** Maintain contact with the client’s skin during indirect electrotherapy to ensure consistent current flow.

Safety During Operation

- **Start Procedure:** Press Start only after the electrode is in contact with the skin, then gradually increase energy.
- **Removal Procedure:** Before removing the electrode, reduce energy to the minimum and press Off to stop output.
- **Power Control:** Ensure the device is turned off before installing or removing glass electrodes.
- **Electrode Hygiene:** Keep all glass electrodes clean, dry, and disinfected before and after each use.

- **Handle Cleaning:** Clean the electrotherapy handle with a dry cloth only. Do not wash it with water.
- **Oil Removal:** If oil residue is present, wipe with anhydrous alcohol and store in a dry place.









Safety Tips

- **Power Supply:** Ensure the device is connected to a properly installed outlet with the correct voltage.
- **Accessibility:** Use outlets that allow quick disconnection in case of emergency.
- **Unplugging:** Always unplug by holding the plug, not the power cord.
- **Repairs:** Do not attempt to open or repair the device. Only qualified technicians should perform repairs.
- **Damage Check:** Do not use the device if the power cord or unit is visibly damaged.
- **Wet Hands:** Do not touch the device or power cord with wet hands.
- **Skin Check:** Do not use the device on irritated, inflamed, or infected skin areas.
- **Power Stability:** Avoid using the device during power fluctuations or unstable electrical supply.


Warnings

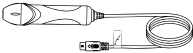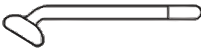- **Metal Objects:** Do not allow clients to touch metal objects during high-frequency electrotherapy. Remove all metal accessories before treatment.
- **Restricted Use:** Do not use this device on individuals with sensitive skin, skin discoloration, rosacea, wounds, pregnant individuals, or those with medical devices such as pacemakers.
- **Product Use:** Do not use alcohol-based or flammable products before or during treatment. Allow all topical products to fully dry before operating the device.
- **Eye Safety:** Never point any electrotherapy electrode directly at the eyes.
- **Device Operation:** Do not touch the electrotherapy unit or output interface while the device is powered on.
- **Professional Use:** This device should be operated by trained professionals only.
- **Interference:** Keep the device away from other electronic equipment to avoid interference during operation.
- **Malfunction:** If any abnormalities occur, stop use immediately and contact a qualified technician. Do not attempt to open or repair the device yourself.
- **Environment:** Do not use the device in bathrooms or damp environments.
- **Storage Safety:** Keep the device away from flammable or explosive materials.

Included Items


Qty	Part Name	Image
1	High Frequency Unit	
1	Attachment Rack	
1	High-Frequency Electrotherapy Handle	
1	Mushroom-Shaped Glass Electrotherapy Tube	
1	Spoon-Shaped Glass Electrotherapy Tube	
1	Spot Glass Electrotherapy Tube	
1	Straight-Bar Glass Electrotherapy Tube	
1	Power Cable	

Device Components

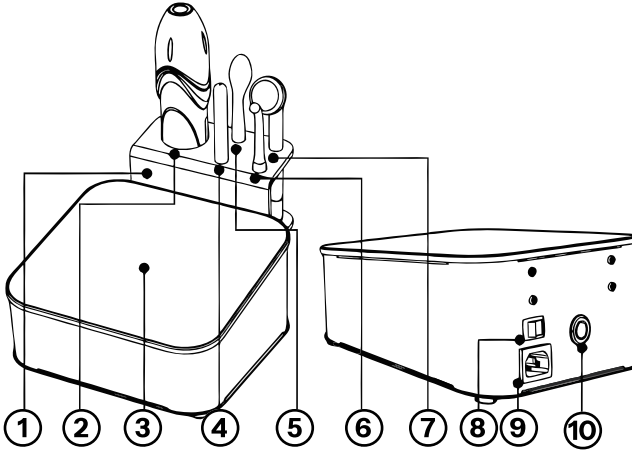



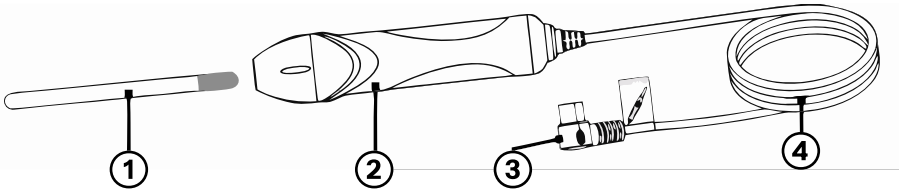

- | | |
|---------------------------|------------------------------|
| 1. Attachment Rack | 6. Spot Electrode |
| 2. High-Frequency Handle | 7. Mushroom-Shaped Electrode |
| 3. Control Panel | 8. Power Switch |
| 4. Straight-Bar Electrode | 9. Power Socket |
| 5. Spoon-Shaped Electrode | 10. Output Socket |

High-Frequency Electrotherapy Handle




1. Electrotherapy Tube
2. Handle
3. Plug for Electrotherapy Handle
4. Electrotherapy handle flexible cable

Glass Electrotherapy Tubes

	Mushroom Electrode: Used for larger areas (Face & Neck)
	Spoon Electrode: Used for medium areas (Nose & Jawline)
	Spot Electrode: Used for small localized areas (Blemishes)
	Rod Electrode: Used for general & contour application areas

Assembly and Disassembly of the Electrotherapy Tube

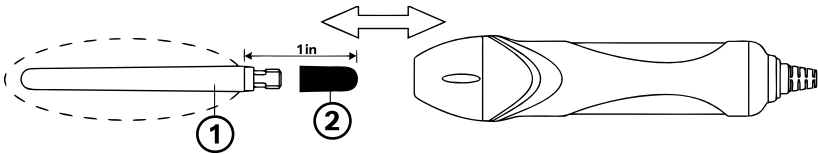


1. Glass Electrode
2. Metal Electrode

Instructions:



- When assembling or disassembling the electrotherapy tube, hold only the areas indicated by the dotted line.
- Keep your hand at least 30 mm away from the metal electrode at all times.

Touch Panel

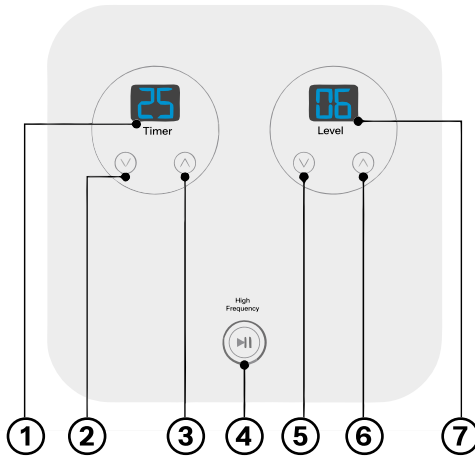


1. Timer Display
2. Decrease Timer
3. Increase Timer
4. Start/Pause
5. Decrease Energy Level
6. Increase Energy Level
7. Power Level Display

Device Assembly

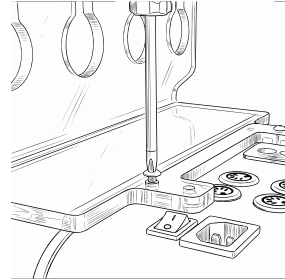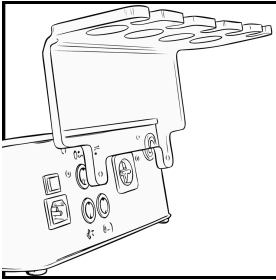

1. Secure the attachment rack to the back of the main unit using screws, following the direction of the arrows.
2. Place each attachment onto the rack.
3. Plug the handles into the corresponding output ports on the front of the main unit. Preparation is now complete.
4. Connect the power cord to the main unit.

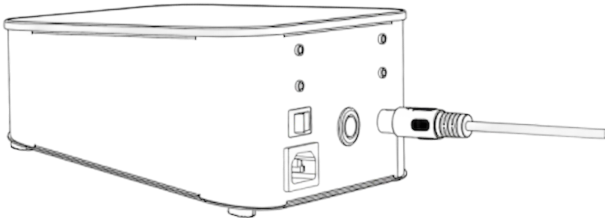


5. Flip the power switch on to enter standby mode. The time display will show “15,” and the energy display will show “1.”
6. In standby mode, press the Start/Pause button to begin operation. Settings can be adjusted while in standby mode.
7. Use the Timer Increase/Decrease buttons to adjust the operating time. The adjustable range is 1 to 30 minutes.
8. Use the Level Increase/Decrease buttons to adjust the energy output. The adjustable range is Level 1 to Level 9.

9. You can start or pause operation at any time by pressing the Start/Pause button while in standby or during operation.

How to Use - Direct Electrotherapy

1. Hold the high-frequency electrotherapy handle by the designated grip area (as shown in the figure). Do not touch the glass tube with your hands, or keep contact to a minimum. Set the energy level to the lowest setting before contact. Place the electrode in contact with the skin, press Start, then gradually increase intensity according to client comfort.

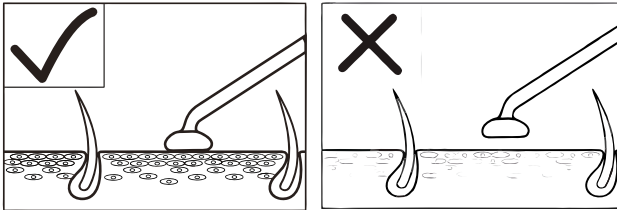

2. Place the glass electrotherapy tube in proper contact with the treatment area, then press the Start button to begin operation.
3. Gradually increase the energy level as needed. The glass tube will generate ionized light and output high-frequency current.

If a skincare product (such as cream or mask) has been applied, gently move the tube across the skin—starting from the neck and moving upward to the forehead, face, cheeks, and nose. Avoid the thyroid area.

Adjust the energy level as needed during use. The generated heat provides a soothing effect, while supporting a refreshed-looking complexion, making it suitable for oily-looking or blemish-prone skin.

4. After completing the treatment, gradually reduce the energy level to the minimum setting and press Pause to stop output. Remove the glass tube from the treatment area.

Effects of Direct Electrotherapy

1. Helps provide a warming, stimulating cosmetic effect.
2. Helps improve the appearance of refreshed-looking skin.
3. May support the appearance of clearer-looking skin.
4. Helps prepare the skin for professional skincare treatments.

Indirect Electrotherapy

1. Instruct the client to hold the straight electrotherapy tube, then press the Start button to begin operation.
2. Gradually adjust the energy level to the desired intensity while massaging the treatment area. Do not touch the electrodes during operation.
3. Before removing the glass electrode from the client's hand, press the Pause button to stop output.

Effects of Indirect Electrotherapy

- Helps create a warming, stimulating cosmetic effect.
- Helps support the appearance of refreshed-looking skin.
- Helps prepare the skin for professional skincare treatments.

Specifications

Product Name:	High-Frequency Table Top Facial Unit HF
Voltage:	AC100-120V/220-240V
Frequency:	50/60Hz
Power Consumption:	20W
Dimensions(L*W*H):	12.0 x 10.0 x 8.9”
Net Weight:	4.6 lbs

Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor’s discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

SKIN+ACT

Phone: 1-800-418-0830

Email: info@skinact.com

Website: www.skinact.com