


SKIN+ACT

# 5 in 1 Skin Care System (Vacuum, Spray, High Frequency, Facial Brush, Galvanic)

PRODUCT MANUAL

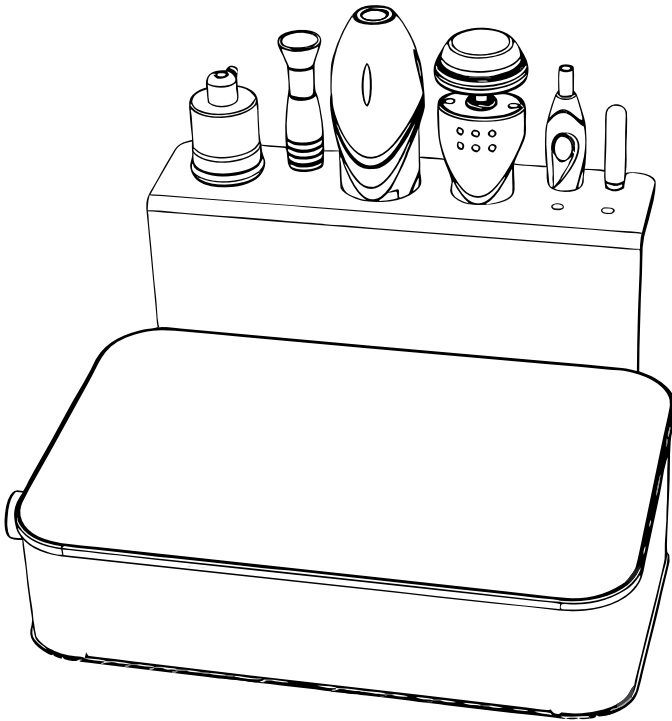

## NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

## Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

Copyright © 2026 SkinAct, All Rights Reserved

## TABLE OF CONTENTS

|   |    |
|---|----|
| Disclaimer & Contraindications .....            | 5  |
| Precautions & Safety Instructions .....         | 6  |
| Probe Safety & Handling .....                   | 7  |
| Included Items .....                            | 8  |
| Device Components .....                         | 10 |
| Accessories to Spray & Vacuum .....             | 11 |
| High Frequency Tool Components .....            | 12 |
| Brush Handle Components .....                   | 13 |
| Galvanic Tool Components .....                  | 14 |
| Device Components (Interface) .....             | 15 |
| Device Assembly & Touch Control Operation ..... | 16 |
| Vacuum Function - Operating Instruction .....   | 17 |
| Vacuum Tips Attachments & Spray Function .....  | 18 |
| Vacuum & Spray Precautions .....                | 19 |
| High-Frequency Electrotherapy Function .....    | 21 |
| Glass Electrotherapy Tubes .....                | 21 |
| How to Use (Direct Electrotherapy) .....        | 22 |
| How to Use (Indirect Electrotherapy) .....      | 23 |
| Important Notes (Electrotherapy) .....          | 24 |
| Daily Maintenance (Electrotherapy) .....        | 25 |

**TABLE OF CONTENTS**

Brush Cleansing Function - Operating Instructions ..... 26

Replacing the Brush Head ..... 27

Brush Cleansing: Precautions ..... 28

Galvanic Function - Operating Instructions ..... 29

Ion Polarity ..... 29

How To Use (Galvanic) ..... 30

Important Notes ..... 32

Daily Maintenance ..... 33

Specifications ..... 34

Warranty Information ..... 35

## DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

## Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH:

- Pacemakers, metallic implants, or other electronic medical devices.
- Heart conditions or a history of heart disease.
- Pregnancy.
- Inflamed, irritated, or damaged skin.
- Severe acne rosacea or vascular conditions.
- Epilepsy or other seizure disorders.
- Clients undergoing medical treatments or with underlying health conditions must obtain prior clearance from a physician.

## Precautions

- **Inspect the Device:** Check for any visible damage to the device or its power cable before and after each use.
- **Unplug When Not in Use:** Always disconnect the device from the power source before cleaning, storing, or performing maintenance.
- **Avoid Humidity:** Do not operate the device in bathrooms or other excessively humid environments.
- **Position Cables Safely:** Ensure the power cable is positioned to prevent tripping hazards and is plugged into a grounded socket.
- **Keep Away from Children:** The device and its accessories should be stored securely out of reach.
- **Report Issues Promptly:** If you notice abnormal functioning, contact your distributor or an authorized technician immediately.


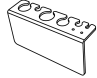
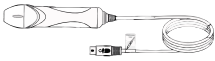






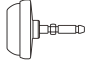
## Safety Instructions










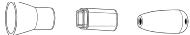

- Plug the device into a properly grounded electrical outlet with the appropriate voltage.
- Do not use the device if it or its power cable is damaged.
- Never touch the device, power plug, or cables with wet hands.
- Avoid attempting to repair or modify the device. Repairs should only be performed by authorized technicians.
- Store the device in a cool, dry place away from direct sunlight and moisture.

## Probe Safety & Handling


- **Continuous Output:** Do not leave the probe in an active output state without performing treatment, as this may cause overheating and reduce its lifespan.
- **Disassembly:** Do not attempt to separate the stainless steel probe from the handle, as this may damage the device.
- **Cleaning:** Do not clean the probe in water or use alcohol on metal components, as this may cause damage.
- **Surface Condition:** Do not use probes with damaged surfaces or severe scratches. Replace any compromised attachments immediately.
- **Malfunction:** If any abnormalities occur, stop use and contact a qualified technician. Do not attempt to open or repair the device yourself.
- **Environment:** Do not use the device in bathrooms or damp environments.
- **Storage Safety:** Keep the device away from flammable or explosive materials.
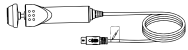
## Included Items
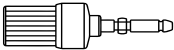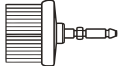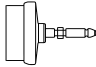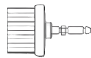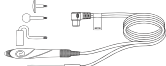
| Qty | Part Name                          | Image   |
|-----|------------------------------------|---|
| 1   | 5 in 1 Skin Care System Unit       |    |
| 1   | Accessory Rack                     |    |
| 1   | High Frequency Electrotherapy Wand |    |
| 1   | Mushroom Electrode                 |    |
| 1   | Spoon-Shaped Electrode             |    |
| 1   | Dot Electrode                      |    |
| 1   | Straight Electrode                 |    |
| 1   | Brush Handle                       |    |
| 1   | Crystallized Small Brush           |    |
| 1   | Crystallized Large Brush           |  |

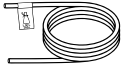
| Qty | Part Name             | Image   |
|-----|-----------------------|---|
| 1   | Small Bristle Brush   |    |
| 1   | Large Bristle Brush   |    |
| 1   | Plastic Bristle Brush |    |
| 1   | Large Soft Brush      |    |
| 1   | Roller Probe          |    |
| 2   | Spray Bottle          |    |
| 1   | Vacuum Handle         |    |
| 1   | Spray Hose Tube       |    |
| 1   | Vacuum Hose Tube      |    |
| 1   | Vacuum Tips           |    |
| 1   | Power Cable           |  |

Device Components

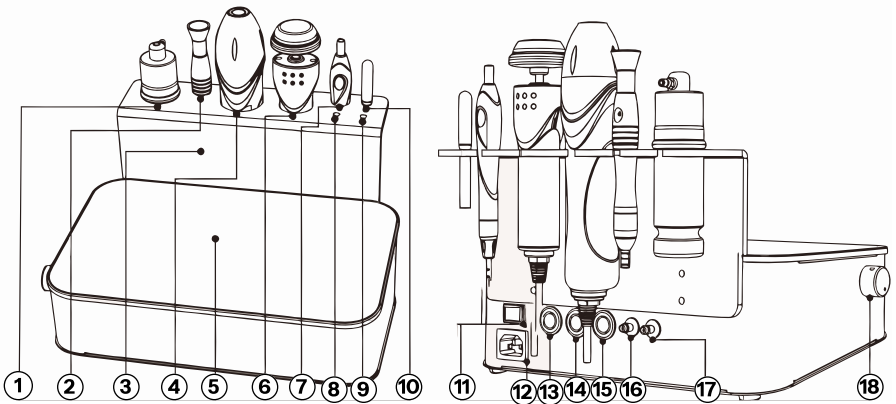


- |   |  |
|---|--|
| 1. Spray Bottle                         | 10. Galvanic Accessories                 |
| 2. Vacuum Handle                        | 11. Power Switch                         |
| 3. Attachment Rack                      | 12. Power Inlet                          |
| 4. High-Frequency Electrotherapy Handle | 13. Galvanic Output Interface            |
| 5. Control Panel                        | 14. Brush Output Interface               |
| 6. Brush Handle                         | 15. High-Frequency Electrotherapy Output |
| 7. Galvanic Handle                      | 17. Spray Output                         |
| 8. Galvanic Round Probe                 | 18. Adjustment Knob                      |
| 9. Galvanic Sphere Probe                |  |

## Accessories to Spray & Vacuum

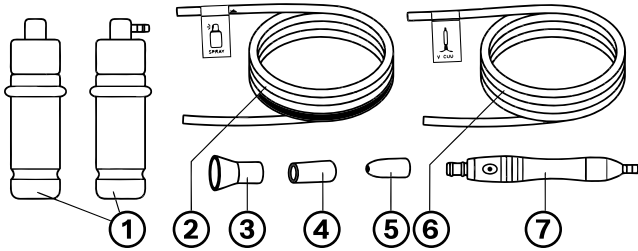


- |                       |                     |
|-----------------------|---------------------|
| 1. Spray Bottle       | 5. Round Vacuum Tip |
| 2. Spray Tube         | 6. Vacuum Tube      |
| 3. Trumpet Vacuum Tip | 7. Vacuum Handle    |
| 4. Flat Vacuum Tip    |                     |

## Vacuum

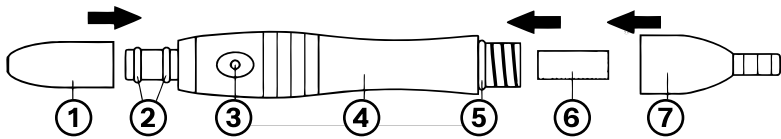

- |                            |                            |
|----------------------------|----------------------------|
| 1. Round Vacuum Tip        | 7. Seal Ring of Handle     |
| 2. Seal Ring of Vacuum Tip | 8. Sponge Filter           |
| 3. Air-Pressure Vent       | 9. Vacuum Handle Connector |
| 4. Vacuum Handle           |                            |

## High Frequency Tool Components

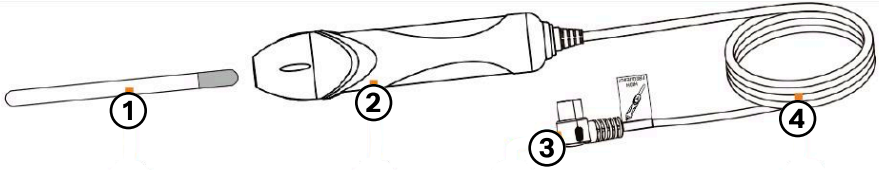


- |                          |                    |
|--------------------------|--------------------|
| 1. Electrotherapy Tube   | 3. Plug for Handle |
| 2. Electrotherapy Handle | 4. Flexible Cable  |

### Selecting The Right Glass Electrotherapy Tube

- **Mushroom Shape:** Used for larger areas (Face & Neck)
- **Spoon Shaped:** Used for medium areas  
(Nose & Jawline)
- **Dot Shape:** Used for small localized areas (Blemishes)
- **Rod Electrode:** Used for general & contour application areas

### Assembly and Disassembly of the Electrotherapy Tube

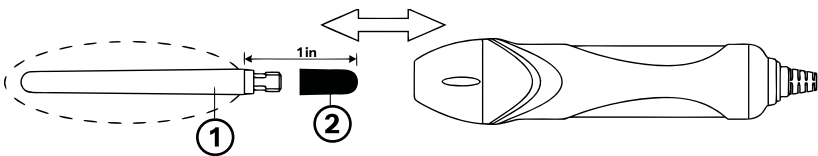


1. Glass Electrode
2. Metal Electrode


#### Instructions:

- When assembling or disassembling the electrotherapy tube, hold only the areas indicated by the dotted line.
- Keep your hand at least 30 mm away from the metal electrode at all times. Do not use alcohol-based products before or during use. Do not use over the eyes, thyroid area, or inflamed skin.

## Brush Handle Components

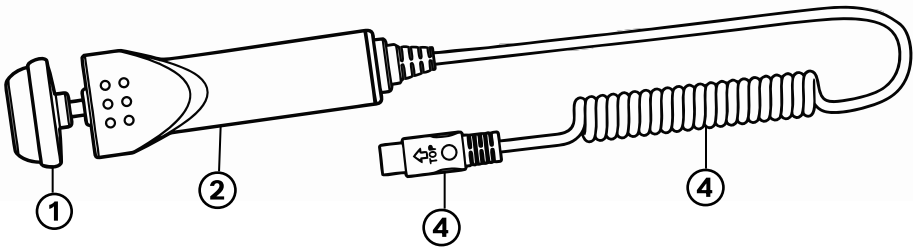


- |                 |                    |
|-----------------|--------------------|
| 1. Brush Head   | 3. Plug for Handle |
| 2. Brush Handle | 4. Flexible Cable  |

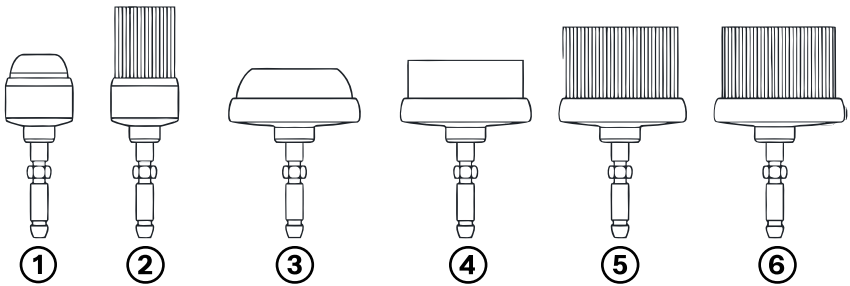


1. **Crystallized Small Brush:** Used to clean small areas of hardened skin and calluses on the hands and feet. Not intended for delicate facial skin.
2. **Small Bristle Brush:** Suitable for gentle cleansing and massage of small facial areas.
3. **Crystallized Big Brush:** Designed for cleaning larger areas of hardened skin and calluses on the hands and feet. Not intended for delicate facial skin.
4. **Plastic Bristle Brush:** Ideal for cleansing and massaging larger areas of the face.
5. **Large Bristle Brush:** Provides gentle cleansing and massage for larger facial areas.
6. **Large Soft Brush:** Used for cleansing and massaging larger areas of the face and other body areas.

## Galvanic Tool Components

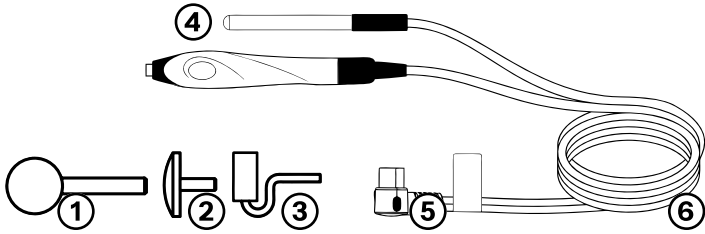


- 1. Round Eye Probe
- 2. Galvanic Sphere
- 3. Roller Probe
- 4. Conduction Probe
- 5. Conduction Probe Plug
- 6. Conduction Probe Cable

## Device Components (Interface)

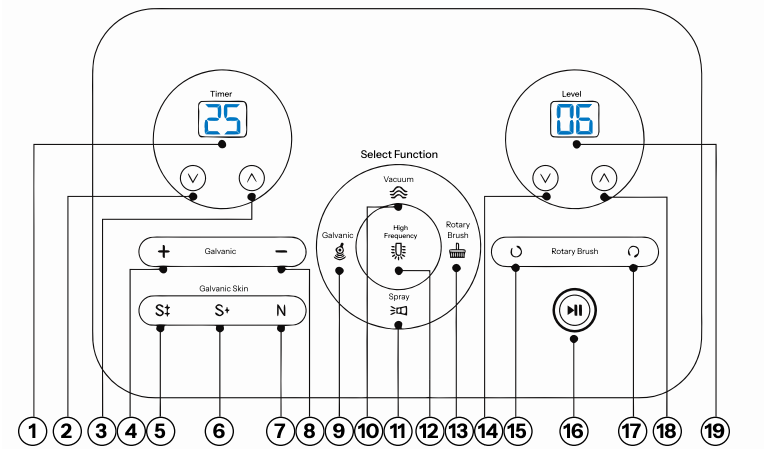

- |                          |                           |
|--------------------------|---------------------------|
| 1. Time display          | 11. Spray Button          |
| 2. Decrease Timer        | 12. High-Frequency Mode   |
| 3. Increase Timer        | 13. Brush Mode            |
| 4. Cation Mode Indicator | 14. Decrease Energy Level |
| 5. Ultra-Sensitive Mode  | 15. Brush Rotate (CCW)    |
| 6. Sensitive Mode        | 16. Start/Pause Button    |
| 7. Normal Mode           | 17. Brush Rotate (CW)     |
| 8. Anion Mode Select     | 18. Increase Energy Level |
| 9. Galvanic Mode         | 19. Energy Display        |
| 10. Vacuum Mode          |                           |

## Assembly Instructions

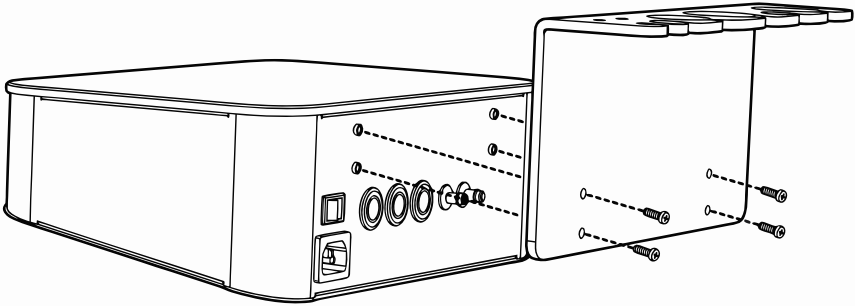

- 1. Attach the Rack:** Secure the attachment rack to the back of the main unit using the provided screws, following the direction of the arrows.
- 2. Place Attachments:** Place each attachment onto the rack.
- 3. Connect Attachments:** Plug all handles into corresponding output ports on the back of the main unit. Handles will not work unless attached to the correct interface.

## Touch Control Operation

- **Function Selection:** This device uses touch-sensitive buttons. Tap the corresponding icon to select the desired function.
- **Time & Energy Adjustment:** In standby mode, use the increase and decrease buttons to adjust treatment time and energy level.
- **Start/Pause:** Press the start/pause button to begin or stop the operation.




## Vacuum Function- Operating Instructions

- **Function Activation:** Select the vacuum icon on the control panel. The indicator light will turn on to confirm activation.
- **Suction Adjustment:** Use the adjustment knob to control suction strength. Turn left to decrease suction and right to increase suction.
- **Time Setting:** Use the increase and decrease buttons to set the treatment time (1–30 minutes), then press the start/pause button to begin.

## How to Use

- **Positioning:** Hold the suction wand like a pen and place the suction head gently against the skin.
- **Activation:** Use your finger to partially cover the pressure relief hole to activate suction.
- **Technique:** Move the wand slowly across the treatment area. Impurities such as blackheads and debris will be extracted.
- **Customization:** Select different suction heads depending on the treatment area.

## Vacuum Tip Attachments

|   |   |
|---|---|
| Small Round Head: Suitable for blackheads and whiteheads on targeted areas. |  |
| Flat Head: Suitable for narrow or hard-to-reach areas.                      |  |
| Trumpet Head: Suitable for larger areas of the skin.                        |  |

## Spray Function- Operating Instructions

- **Function Activation:** Select the spray icon on the control panel. The indicator light will turn on to confirm activation.
- **Time Setting:** Use the increase and decrease buttons to adjust the treatment time (1–30 minutes).
- **Start/Pause:** Press the start/pause button to begin or stop the operation.

## How to Use

- **Preparation:** Ensure the spray bottle is properly filled and securely attached before use.
- **Operation:** Hold the spray bottle firmly and gently press the air hole at the top to release a fine mist.
- **Switching Bottles:** If using multiple solutions, pause the device before switching spray bottles.
- **Application Technique:** Spray evenly across the face, starting from the forehead and moving downward while maintaining a safe distance from the skin.

- **Sensitive Areas:** Avoid direct spraying into the eyes, nose, and mouth. Ask the client to keep eyes and mouth closed during treatment.
- **Post-Treatment:** After spraying, pause the device and use cotton pads or soft tissue to absorb excess liquid from the skin.

## **Vacuum & Spray Function - Precautions**

- **Hose Connection:** Ensure the air hose is connected to the correct outlet according to the label. Do not connect incorrectly.
- **Replacement Safety:** Pause the device before replacing the spray bottle or nozzle.
- **Hose Handling:** Do not bend the hose or block the air nozzle during operation, as this may damage the device.
- **Damage Check:** If the hose is cracked or damaged, trim the affected area and reconnect securely to maintain proper performance.

## Spray Function- Precautions

- **Hose Connection:** Ensure the air hose is connected to the correct outlet according to the label. Do not connect incorrectly.
- **Replacement Safety:** Pause the device before replacing the spray bottle or nozzle.
- **Hose Handling:** Do not bend the hose or block the air nozzle during operation, as this may damage the device.
- **Damage Check:** If the hose is cracked or damaged, trim the affected area and reconnect securely to maintain proper performance.

## Vacuum Function- Precautions





- **Skin Preparation:** Clean and soften the skin before using the vacuum function.
- **Suction Control:** Use moderate suction. Excessive suction may cause bruising or skin damage. Test on your hand before use.
- **Duration Control:** Avoid prolonged suction in one area to prevent bruising.
- **Skin Type Consideration:** For oily skin, suction frequency may be increased. Avoid use on sensitive skin, dilated capillaries, inflamed areas, or around the eyes.
- **Frequency:** Do not overuse the vacuum function, as excessive or repeated suction may irritate the skin, cause temporary redness, bruising, or worsen the appearance of pores.
- **Post-Use Cleaning:** Disassemble, clean, dry, and disinfect the suction tube after each use.

- **Filter Maintenance:** Replace the sponge filter regularly. Do not use the device without the filter, as this may damage the unit.

## High-Frequency Electrotherapy Function

- **Function Activation:** Select the high-frequency icon on the control panel. The indicator light will turn on to confirm activation.
- **Time Setting:** Use the increase and decrease buttons to adjust the treatment time (1-30 minutes).
- **Energy Adjustment:** Use the increase and decrease buttons to set the energy level (1-9).
- **Start/Pause:** Press the start/pause button to begin or stop the operation.
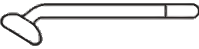
## Glass Electrotherapy Tubes

|   |   |
|---|---|
|     | Mushroom Electrode: Used for larger areas (Face & Neck)     |
|  | Spoon Electrode: Used for medium areas (Nose & Jawline)     |
|   | Spot Electrode: Used for small localized areas (Blemishes)  |
|   | Rod Electrode: Used for general & contour application areas |

## How to Use (Direct Electrotherapy)

- 1. Preparation:** Apply a skincare cream or mask to the treatment area. Hold the high-frequency handle correctly and ensure the device is set to the minimum energy level. Do not touch the glass electrode with your hands or allow it to come into contact with metal objects.
- 2. Initial Contact:** Place the glass electrode in contact with the skin before starting the device. Press the start button to begin operation.
- 3. Energy Adjustment:** Gradually increase the energy level to a comfortable intensity.
- 4. Application:** Glide the electrode slowly across the forehead, cheeks, nose, and chin, keeping continuous contact with the skin.
- 5. Operation:** Move the electrode smoothly and evenly across the treatment area to ensure consistent results.
- 6. Completion:** After treatment, reduce the energy level to the minimum, press the start/pause button to stop the device, and then remove the electrode from the skin.

## Effects of Direct Electrotherapy

- **Soothing Effect:** Produces gentle warmth that calms and relaxes the skin.
- **Refreshing Effect:** Helps improve the appearance of tired-looking skin for a more refreshed-looking complexion.
- **Clarifying Support:** May support the appearance of clearer-looking skin when used as part of a professional facial treatment.
- **Treatment Prep:** Helps prepare the skin for professional skincare treatments and finishing products.

## How to Use (Indirect Electrotherapy)

- 1. Preparation:** Have the client hold the straight electrode securely in their hand.
- 2. Start-Up:** Press the start button and gradually adjust the energy level to a suitable intensity.
- 3. Application:** The operator massages the client's skin while the client holds the electrode. Do not touch the electrode during operation.
- 4. Completion:** Before removing the electrode, press the start/pause button to stop the device.

## Effects of Indirect Electrotherapy

- **Stimulating Effect:** Helps create a gentle, warming cosmetic effect during professional facial treatments.
- **Skin Appearance Support:** Helps support the appearance of refreshed, healthy-looking skin.
- **Treatment Enhancement:** Helps complement professional skincare treatments when used with proper technique and suitable products.

## Important Notes

- **Normal Operation:** The electrotherapy handle emits a continuous sound during normal use.
- **Spark Sensation:** Slight sparks may occur if the electrode is close to but not fully touching the skin, which may cause discomfort. Always start at the minimum energy level before contact. If sensitivity persists, place a dry gauze over the skin (as shown below).



- **Proper Contact:** Ensure the electrode maintains full contact with the skin during direct electrotherapy to avoid unwanted sparking.
- **Treatment Duration:** For oily skin, limit treatment to approximately 8 minutes. For dry or sensitive skin, reduce treatment time to 3–5 minutes. Adjust energy levels accordingly.
- **Indirect Technique:** During indirect electrotherapy, maintain consistent contact with the client to prevent interruption of the current.
- **Start Safety:** Always begin operation only after the electrode is in contact with the skin, then gradually increase the energy level.
- **Removal Procedure:** Before removing the electrode, reduce the energy level to minimum and turn off the device.
- Do not use high-frequency over the thyroid area, eyelids, broken skin, inflamed skin, or areas with reduced sensation.

- **Installation Safety:** Turn off the power before installing or removing glass electrodes.
- **Hygiene:** Keep all electrodes clean, dry, and disinfected before and after each use.
- **Handle Cleaning:** Clean the handle with a dry cloth. Do not wash with water. If needed, use alcohol and store in a dry place.

## Daily Maintenance


- **Handling & Disinfection:** Handle all glass electrodes with care and ensure proper cleaning and disinfection after each use.
- **Connection Check:** Regularly check that electrodes are securely connected. To test, insert the electrode into the handle, place it on a surface without contact, power on the device, and gradually increase the energy level. A visible glow indicates normal function.
- **General Cleaning:** Wipe the device regularly with a dry cloth and store it in a clean, dry, and well-ventilated area.
- **Moisture Protection:** Do not clean the device with water. Avoid exposure to moisture.
- **Power Safety:** After each use, turn off the main power switch and unplug the device.
- **Storage Conditions:** Store the device away from direct sunlight, rain, humidity, dust, and excessive smoke. Keep cables organized and untangled.

## Brush Cleaning Function - Operating Instructions

- **Function Activation:** Select the brush icon on the control panel. The indicator light will turn on to confirm activation.
- **Rotation Direction:** Choose the rotation direction as needed. Use the clockwise or counterclockwise selection button based on treatment preference.
- **Time Setting:** Use the increase and decrease buttons to adjust the treatment time (1–30 minutes).
- **Energy Adjustment:** Use the increase and decrease buttons to set the energy level (1–9).
- **Start/Pause:** Press the start/pause button to begin or stop the operation.



## Replacing the Brush Head

1. **Removal:** Hold the handle firmly with one hand and the plastic base of the brush head with the other. Pull gently in opposite directions until the brush head detaches.
2. **Installation:** Align the brush head with the handle and press firmly until it clicks into place.
3. **Handling:** Always grip the plastic base when removing or installing. Do not pull on the bristles, as this may damage the brush and affect performance.

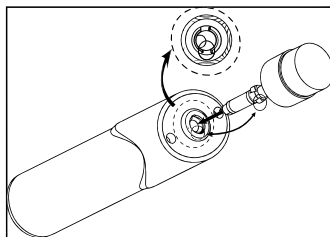

## How to Use (Brush Cleansing)

- 1. Preparation:** Moisten the skin with water or use the spray function to dampen the treatment area.
- 2. Product Application:** Apply a suitable cleansing product (such as cleansing milk or gel) evenly to the skin.
- 3. Brush Selection:** Choose an appropriate brush head for the skin type and treatment area, then attach it securely to the handle.
- 4. Start-Up:** Lightly dampen the brush head with distilled water, turn on the device, and begin cleansing.
- 5. Operation:** Move the brush gently across the skin in circular motions using light pressure. Keep the skin moist throughout the treatment.
- 6. Post-Cleansing:** After treatment, remove any remaining product with a towel or tissue. Then cleanse the skin again using a damp towel or cleansing pad.

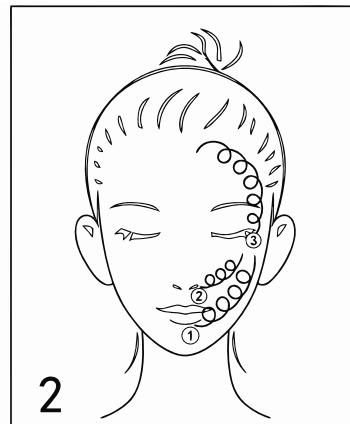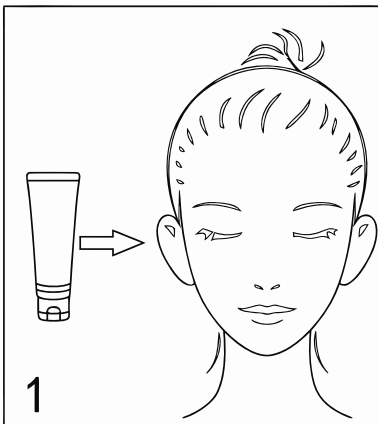

## Brush Cleansing Function - Precautions

- **Attachment Check:** Ensure the brush head is securely attached before use.
- **Initial Contact:** Before the brush touches the skin, start at a lower speed. Use gentle pressure to avoid discomfort.
- **Product Use:** When using cleansing or massage products, reduce the rotation speed to prevent splashing or product dispersion.
- **Replacement Safety:** Pause the device and ensure the brush has completely stopped before replacing the brush head.
- **Cleaning After Use:** Remove and clean all brush heads after each use. Disinfect and store them in a dry place.
- **Handle Care:** Do not immerse the brush handle in water. Clean it with a dry cloth after use and store in a dry environment.

## Galvanic Function - Operating Instructions

- **Function Activation:** Select the galvanic icon on the control panel. The indicator light will turn on to confirm activation.
- **Time Setting:** Use the increase and decrease buttons to adjust the treatment time (1–30 minutes).
- **Energy Adjustment:** Use the increase and decrease buttons to set the energy level (1–9).
- **Start/Pause:** Press the start/pause button to begin or stop the operation.
- **Note:** Positive/negative polarity should be selected based on the product manufacturer's instructions and treatment goal. Do not use products with unknown polarity or products not intended for galvanic use.

### Ion Polarity Selection

- **Mode Switching:** Select the appropriate polarity based on the treatment goal.

|          |  |
|----------|--|
| <b>+</b> | <b>Positive Ion Mode:</b> Used for ion introduction (infusion) treatments. |
| <b>—</b> | <b>Negative Ion Mode:</b> Used for deep cleansing and impurity removal.    |

### Skin Type Selection

|                                  |  |
|----------------------------------|--|
| <b>N</b>                         | <b>Normal Skin(N):</b> Used for ion introduction (infusion) treatments.        |
| <b>S<sup>+</sup></b>             | <b>Sensitive Skin (S):</b> Used for deep cleansing and impurity removal.       |
| <b>S<sub>+</sub><sup>+</sup></b> | <b>Very Sensitive Skin (S+):</b> Used for deep cleansing and impurity removal. |

## How to Use (Galvanic Function)

### A. Monopolar Infusion Method

- 1. Preparation:** Apply half of the skincare essence directly to the treatment area. Soak the remaining essence into a cotton pad and wrap it around the treatment probe.
- 2. Client Setup:** Set the energy level to minimum. Have the client hold the metal electrode rod, while the operator holds the probe and places it on the treatment area.
- 3. Start-Up:** Press the start button to activate the device. Select the appropriate ion polarity (+ or -), then gradually increase the energy level until a slight tingling sensation is felt.
- 4. Application:** Move the probe slowly across the skin in circular, zigzag, or spiral motions while maintaining constant contact. Treatment time is typically 3–5 minutes.
- 5. Completion:** Press the start/pause button to stop the device. Remove the probe and have the client release the electrode rod.

### B. Bipolar Infusion Method (Dual Roller)

- 1. Preparation:** Apply skincare product evenly to the treatment area or use a cotton pad soaked with product.
- 2. Setup:** Connect one roller to the positive pole and the other to the negative pole. Ensure the two rollers do not touch each other.
- 3. Start-Up:** Set the energy level to minimum, then press the start button. Select the appropriate ion polarity based on the product used.

4. **Energy Adjustment:** Gradually increase the energy level until a mild tingling sensation is felt.
5. **Application:** Glide the rollers smoothly across the skin. Move evenly across the face to ensure balanced treatment.
6. **Technique Note:** If only one roller is used at a time, alternate between positive and negative sides to ensure even coverage.

### C. Double Ball Probe

1. **Application Area:** The double ball probe is suitable for small or irregular areas.
2. **Operation:** Follow the same steps as the bipolar (dual roller) method, adjusting movement for smaller, targeted areas.

### D. Impurity Export (Deep Cleansing)

1. **Polarity Setting:** Set the working probe to negative (-) polarity. The client holds the positive (+) electrode rod.
2. **Preparation:** Soak a cotton pad with saline or impurity removal solution and wrap it around the probe head, or apply a thin layer of gauze or mask to the treatment area.
3. **Start-Up:** Turn on the device and select negative ion mode. Begin at a low energy level and increase gradually.
4. **Application:** Move the probe slowly across the treatment area while maintaining full contact with the skin.
5. **Completion:** After treatment, reduce the energy level to minimum, stop the device, and remove the probe.

## Important Notes (Galvanic Function)

- **Treatment Order:** Perform galvanic treatment after cleansing and exfoliation, but before massage. Ensure the skin is clean and dry.
- **Contact Medium:** Always wrap the probe with cotton or use a conductive medium to ensure proper current flow.
- **Energy Adjustment:** Increase energy gradually from low to a comfortable level, maintaining a mild tingling sensation.
- **Handling:** Do not touch the metal parts of the probe during operation.
- **Skin Type Consideration:** Suitable for dry, mature, and oily skin. Avoid heavy oil-based products, as they may reduce effectiveness.
- **Post-Treatment Care:** Clean the probe thoroughly after use and store it in a dry place.
- **Polarity Sensation:** Positive and negative modes may produce different sensations. Adjust according to comfort.
- **Metal Safety:** Remove all metal jewelry before treatment. Do not use on clients with metal implants or similar conditions.
- **Contraindications:** Use caution on sensitive skin and begin at the lowest setting. Do not use on highly reactive, injured, inflamed, irritated, or compromised skin.
- The acidity or alkalinity of the nutrient solution must match the positive and negative poles of the roller.
  - When the roller is **positive (+)**, the nutrient solution should be **acidic (+)**.
  - When the roller is **negative (-)**, the nutrient solution should be **alkaline (-)**.

- Use only water-based products suitable for galvanic treatments. Follow the product manufacturer's polarity instructions. If polarity is unknown, do not use the product with galvanic current.
- During use, do not allow the metal parts of the roller or ball probe to come into direct contact with each other. If contact occurs, the device's protective mechanism will automatically activate. Separate the probes immediately to resume normal operation.
- Keep the probe clean at all times. Clean and disinfect after each use.
  - If using alcohol, only wipe the metal parts of the probe.
  - Do not clean other parts with alcohol, as this may cause damage.

## **Daily Maintenance**

1. Arrange the connecting cable between the probe and device neatly. Avoid repeated twisting.
2. Handle the probe carefully during use. After use, disinfect and dry it thoroughly to keep it clean.
3. Clean the main unit with a dry cloth only. Do not use water.
4. After each use, turn off the main power switch and unplug the device before leaving it unattended.
5. Store the device in a location protected from direct sunlight, rain, humidity, dust, and excessive smoke.

## Specifications

|                           |                                |
|---------------------------|--------------------------------|
| <b>Product Name:</b>      | <b>5 in 1 Skin Care System</b> |
| <b>Voltage:</b>           | <b>AC100-120V/220-240V</b>     |
| <b>Frequency:</b>         | <b>50/60Hz</b>                 |
| <b>Power Consumption:</b> | <b>25W</b>                     |
| <b>Dimensions(L*W*H):</b> | <b>14.57 x 14.17 x 6.50”</b>   |
| <b>Net Weight:</b>        | <b>8.38 lbs</b>                |

## Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor’s discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

### SKIN+ACT

Phone: ..... 1-800-418-0830

Email: ..... info@skinact.com

Website: ..... www.skinact.com