

SKIN+ACT

Venice Electric Medical Spa Treatment Chair/Table (With Swivel)

PRODUCT MANUAL



TABLE OF CONTENTS

Notice & Disclaimer3

Warnings 4

Stability and Rotation Safety 5

Safety and Operating Instructions 6

Precautions 6

Dimensions & Angle Specifications 7

Built-In Unit Controls 8

Remote Control Instructions 9

Foot Pedal Controls 10

Headrest Assembly 10

Power and Controller Connections 11

Maintenance 11

Troubleshooting 12

Specifications 13

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

This product contains electric adjustment components and moving parts. Improper use may cause injury, pinching, equipment damage, or unsafe operation. Users are responsible for operating the table safely and following all instructions, warnings, and maintenance requirements in this manual. The user is solely responsible for confirming that use of this device is permitted under applicable state/local laws, scope-of-practice rules, licensing requirements, and workplace safety standards. This device is not intended for self-use, home use, or use by untrained individuals.

Stability and Rotation Safety

When operating a hydraulic rotating chair or single-post electric treatment table, the operator is responsible for maintaining stability and ensuring client safety. Always follow the instructions provided for the specific model.

To ensure stability

- 1. Place the chair or table on a level, dry, and non-slip surface. Adjust the leveling feet, if equipped, so the base remains stable and does not rock.
- 2. Inspect the base, controls, adjustment mechanisms, and fasteners before each use. Do not operate the unit if any component is loose, damaged, or functioning improperly.
- 3. Keep the client’s weight centered over the base. Position the chair or tabletop in line with the base whenever possible, particularly during treatment.
- 4. Do not allow anyone to sit, stand, lean, or place excessive weight on the headrest, backrest, leg rest, foot section, or armrests.
- 5. Whenever possible, rotate or reposition the unit before the client enters it. Do not rotate or adjust the unit while the client is entering or exiting.
- 6. If rotation is necessary while occupied, place the client in a stable upright position, keep the client centered, support the chair or table securely, and rotate it slowly.
- 7. Keep hands, feet, clothing, cords, and other objects away from moving parts and potential pinch points during rotation or adjustment.
- 8. Ensure all rotation locks and adjustable sections are securely engaged before beginning treatment.
- 9. Do not exceed the stated weight capacity or apply sudden, concentrated, or excessive downward pressure. These products are not intended for massage or procedures that require substantial downward force.
- 10. Do not leave a client unattended while the unit is elevated, reclined, rotating, or being adjusted.
- 11. Stop using the unit immediately if it becomes unstable, shifts unexpectedly, makes unusual sounds, or fails to hold its selected position.



Warning: Do Not Sit or Apply Excess Force



Do not sit, stand, jump, or place excessive weight on the backrest or footrest. Doing so may cause the product to tip, damage the backrest/footrest joints, or result in injury.

30%

Back Cushion

55%

Seat Cushion

15%

Leg Cushion



Warnings

- Disconnect the power supply before replacing electrical components or performing maintenance or cleaning.
- Connect the power cord only to a properly installed and grounded electrical outlet to reduce the risk of fire and electric shock.
- Do not stand on the equipment.
- Keep this equipment out of the reach of children. Do not allow children to play with or operate the equipment. When leaving the equipment unattended, raise the backrest to its highest position.
- Do not use or store the equipment in wet or damp environments, such as bathrooms, as this may result in electric shock or equipment failure. Keep the hand controller and foot controller away from water.
- Do not operate the equipment near flammable paints, coatings, gases, or in oxygen-enriched environments.
- This equipment is not intended for use by persons (including children) with reduced physical, sensory, or mental capabilities, or by individuals lacking sufficient experience or knowledge, unless they are supervised or instructed by a person responsible for their safety.
- If the power cord is damaged, discontinue use immediately and contact your SkinAct distributor, authorized service center, or a qualified service technician for repair or replacement.
- Before operating the chair, ensure that no objects or body parts are within the range of movement of any moving components.
- This equipment must be operated only by trained professionals.
- Do not service or perform maintenance on the equipment while it is in use by a patient.
- Do not use portable RF (radio frequency) transmitters or other electromagnetic transmitting devices near this equipment.
- Position the equipment so that the power cord and disconnect device remain easily accessible at all times.
- The third conductor in the power cord is the protective grounding conductor and must not be removed or modified.
- Maximum patient weight: 440 lb (200 kg). Exceeding this weight limit may result in equipment damage or personal injury.
- If the user or patient becomes aware of a serious incident involving this equipment, report it to the appropriate regulatory authority and to SkinAct as soon as possible.
- Dispose of the equipment and its accessories in accordance with applicable local regulations at the end of their service life.
- Do not modify this equipment. Unauthorized modifications may affect safety and performance.
- This equipment is intended for operation by licensed medical or aesthetic professionals. Patients should not operate the equipment themselves.
- The patient should remain aware of their surroundings during treatment and should always be supervised by the operator.
- This product is upholstered with artificial leather. If you experience an allergic reaction to the upholstery material, discontinue use immediately.
- During operation, the surface temperature of the hand controller, button panel, and metal frame may reach 106°F (41°C) but will not exceed 113°F (45°C). Patients should not remain in contact with these surfaces for extended periods (contact time should be less than 1 minute).
- For indoor use only.

Safety and Operating Instructions

To ensure safe and proper operation of this electric treatment table, please read the following instructions carefully and retain this manual for future reference.

- **Proper Setup and Grounding:**

This electric treatment table is equipped with a three-prong power cord and must be connected to a properly grounded three-prong outlet. Proper grounding helps reduce the risk of electric shock.

- **Avoid Water Exposure:**

Keep all plugs, controls, power cords, and connecting cables away from water and other liquids.

- **Inspect for Damage:**

Immediately discontinue use if the plug, controls, power cord, or connecting cables are damaged.

- **Usage Frequency:**

Do not operate the electric actuator continuously for more than 2 minutes. Allow it to rest for at least 18 minutes before operating it again.

- **Professional Repairs Only:**

Repairs and maintenance must be conducted exclusively by trained and qualified professionals.

Precautions

1. Health Restrictions:

- This product should not be used by individuals with osteoporosis or heart conditions.
- Pregnant individuals should avoid using this product.
- Individuals with injuries or skin conditions should not use this product.

2. Use by Minors and Individuals with Impaired Awareness:

- Children under 14 years of age and individuals with impaired consciousness must not use this product without supervision.

3. Inspection Before Use:

- Inspect the power cord and plug for damage before each use. If damage is found, discontinue use and have the damaged component replaced by a qualified technician or authorized service provider.

4. Voltage Compliance:

- Confirm that the product is connected to the correct voltage as specified. Incorrect voltage can result in product damage.

5. Electrical Safety:

- Do not handle the power plug with wet hands to prevent electric shock.
- Ensure the product is properly installed before powering it on or operating it.

6. Securing the Power Cord:

- Secure the power cord to prevent tripping, pulling, or damage.



7. Cleaning:

- Clean the product's surface with a soft cloth dampened with water and a neutral detergent.


8. Unplugging When Not in Use:

- Always unplug the product when it is not in use.


Dimensions & Angle Specifications

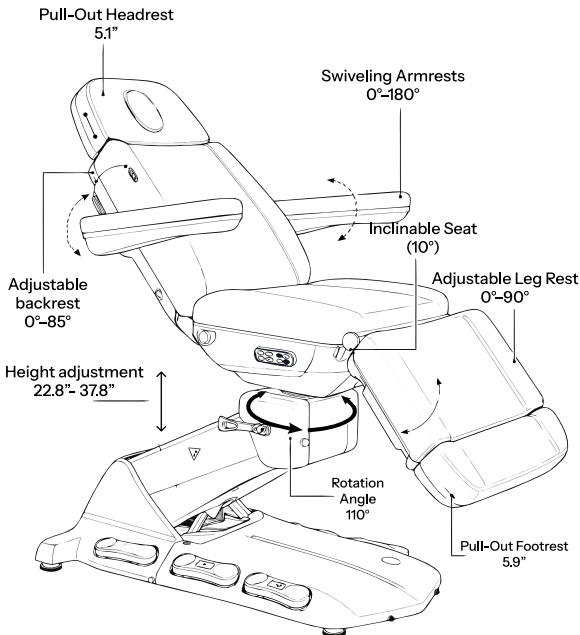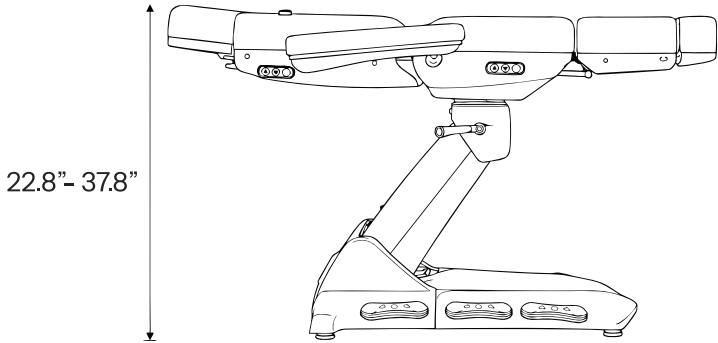


All measurements have a tolerance of 0.25 to 1.2 inches.

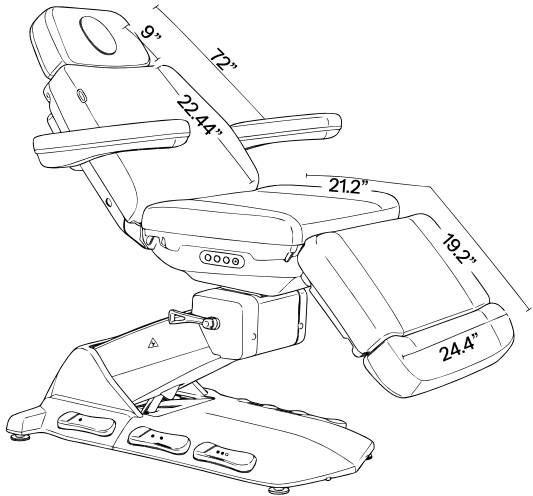

Built-In Unit Controls

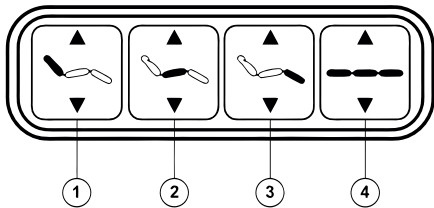


- 1. Backrest Up and Down
- 2. Seat Tilt Backward and Return to Horizontal
- 3. Leg Rest Up and Down
- 4. Table Height Up and Down

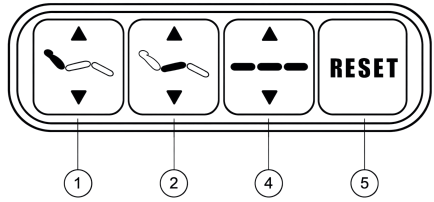

- 1. Backrest Up and Down
- 2. Seat Tilt Backward and Return to Horizontal
- 3. Table Height Up and Down
- 4. Chair Position Reset

Remote Control Instructions

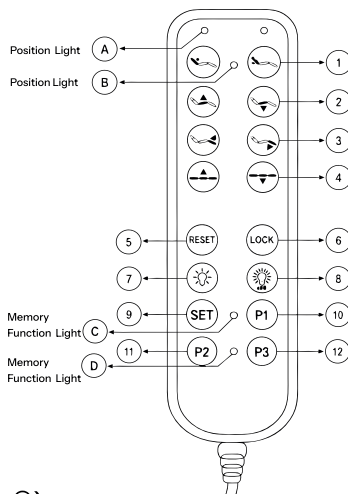

Backrest Adjustment (Button ①)

- Press the adjustment button to raise or lower the backrest.

Seat Tilt Adjustment (Button ②)

- Press the adjustment button to change the angle of the seat cushion.

Leg Rest Adjustment (Button ③)

- Press the adjustment button to raise or lower the leg rest.

Height Adjustment (Button ④)

- Press the adjustment button to raise or lower the entire chair.

Reset Function (Button ⑤)

- Press and hold the RESET button for 3 seconds to reset the chair.
- The backrest and seat will move to their maximum positions.
- The chair will lower to its minimum height.
- The leg rest will move to its minimum position.

Control Lock (Button ⑥)

- Press the LOCK button to lock the system and disable all controls.
- Press the LOCK button again to unlock the system and reactivate the controls.

Ambient Light Color (Button ⑦)

- Press the button to turn on the ambient light and cycle through the available colors.
- The light will turn off after the final color setting.

Ambient Light Brightness (Button ⑧)

- Press the button to adjust the brightness of the ambient light.


Memory Setup (Button ⑨)

- Adjust the chair to the desired position.
- Press and hold the SET button for 3 seconds to enter memory-recording mode.
- The Memory Function Indicator Light (C) will illuminate for 3 seconds.
- While the indicator light is on, press P1, P2, or P3 to save the current position.

Memory Recall (Buttons ⑩–⑫)

- P1 (Button ⑩): Press and hold for 3 seconds to move the chair to saved Position 1.
- P2 (Button ⑪): Press and hold for 3 seconds to move the chair to saved Position 2.
- P3 (Button ⑫): Press and hold for 3 seconds to move the chair to saved Position 3.
- Press any button while the chair is moving to stop the automatic movement.

Foot Pedal Controls

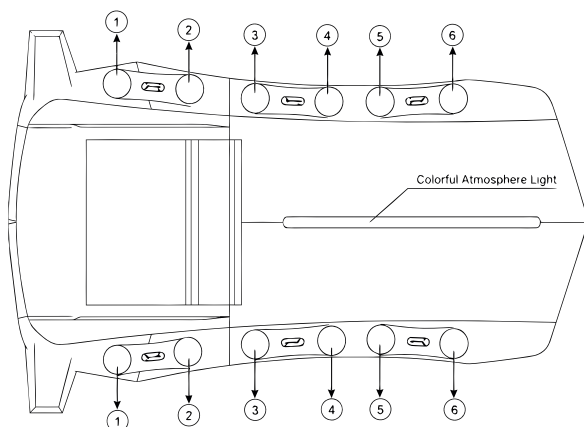

Backrest Adjustment

- Press Foot Control ① to raise the backrest.
- Press Foot Control ② to lower the backrest.


Chair Height Adjustment

- Press Foot Control ③ to raise the entire chair.
- Press Foot Control ④ to lower the entire chair.

Footrest Adjustment

- Press Foot Control ⑤ to raise the footrest.
- Press Foot Control ⑥ to lower the footrest.


Headrest Assembly

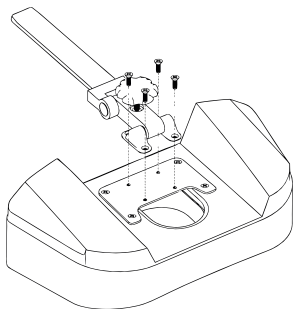

Align the headrest with the mounting holes and loosely insert all four screws as shown below. Once the headrest is properly aligned, tighten all screws securely. All required parts and hardware are included with the treatment table.

Power and Controller Connections

1. Align and securely connect the power cord and hand controller cord to their corresponding ports. Lock each connection as indicated by the arrows.
2. Turn on the power switch. The power indicator should remain steadily lit.
3. If the table does not respond, turn off and unplug the table before checking the fuse. If necessary, replace the fuse with one of the same type and rating.

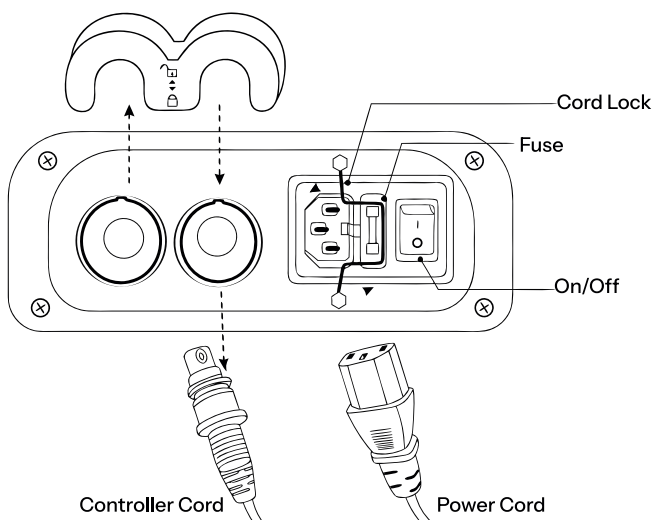

Maintenance

- When cleaning and sanitizing, do not use bleach, Barbicide, or other harsh chemicals. Using harsh chemicals can damage the upholstery.
- For best results, clean with **SkinAct All-In-One Upholstery & Disinfecting Multi-Purpose Cleaner** (SKU24261).
- Clean and sanitize the upholstery after each treatment.
- Every six months, inspect all visible parts of the unit. If there are any abnormal noises or vibrations during operation, call Spa & Equipment/SkinAct at 1-800-418-0830.

Troubleshooting

Motorized Sections Do Not Move

Solution: The selected section may have reached its maximum adjustment limit. Try pressing the opposite-direction key or a different function key.

Table Does Not Respond to the Hand Controller

Solution: Check that the controller is securely connected. If it remains unresponsive, replace the controller or contact SkinAct/Spa & Equipment.

Hand Controller Not Working

Solution: If all power and transformer connections are secure but the table still does not respond, the hand controller may be faulty. Discontinue use and contact SkinAct/Spa & Equipment for inspection or replacement.

Abnormal Noise or Shaking During Lifting or Lowering

Solution: Check for loose screws or hardware and tighten them securely. Apply a SkinAct-approved lubricant to the designated rotating or moving joints, if necessary. If the noise or shaking continues, stop using the table and contact SkinAct/Spa & Equipment.

Motor Performance Issue

Solution: If the motor operates irregularly, makes unusual noises, or fails to move smoothly, discontinue use and contact Spa & Equipment/SkinAct.

Specifications

Product Name:	Venice Electric Medical Spa Treatment Chair/Table (With Swivel)
Voltage:	110 V AC
Frequency:	50/60 Hz
Power Consumption:	300 VA (MAX)
Backrest Angle:	0° flat to 85° incline
Leg Rest Angle:	0° flat to 90° decline
Height Range:	22.8"–37.8"
Treatment Bed Length:	72.8"
Treatment Bed Width:	33"
Seat Tilt Angle:	10°
Rotation:	220° (110° to the left, 110° to the right)
Base Dimensions (L × W):	40" × 28"
Weight Capacity:	440 lb (200 kg)
Net Weight:	185 lb (84 kg)
Footrest Extension:	Up to 5.9"
Headrest Extension:	Up to 5.1"