


SKIN+ACT

# MicroLift Pro

## Microcurrent Machine

PRODUCT MANUAL

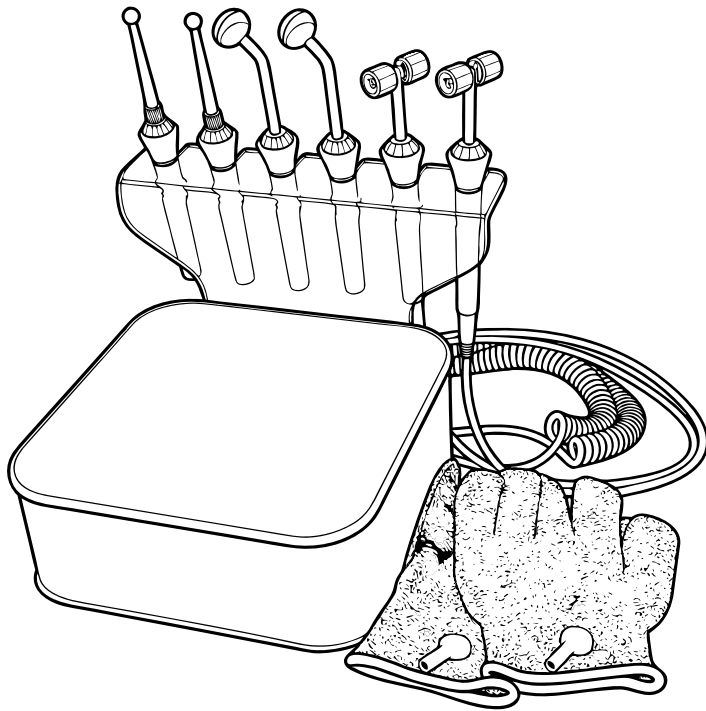

## NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchant ability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

## Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, ect..

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

---

## TABLE OF CONTENTS

Disclaimer ..... 4

Contraindications & Precautions ..... 5

Safety Instructions & Probe Care Instructions ..... 6

General Notes(Before/During/After Treatment) ..... 7

Included Items ..... 8

Device Components ..... 9

Device Components (Interface) ..... 10

Assembly Instructions & Instructions for Use ..... 11

Function Overview (Cleansing, Infusion, Pre-Lift) ..... 12

Function Overview(Lifting) & Operating Instructions ..... 13

Treatment Areas & Techniques ..... 14

Magic Gloves: Preparation ..... 16

Magic Gloves: How to Use ..... 17

Magic Gloves: Safety & Sensory Guide ..... 18

Specifications ..... 19

Warranty Information ..... 20

## DISCLAIMER

- This product has been designed to be used by esthetic professionals at spas and salons, it should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorization.
- We shall not be held liable for any material or personal damage due to the misuse of the product.
- Documents are made available without warranty of any kind. They may be updated, altered, expanded, or removed at any time. Skin Act does not warrant or guarantee any of the processes described herein or perform any independent analysis in connection with guidelines provided.
- Readers are expressly warned to consider and adopt all safety precautions indicated herein and to avoid all potential hazards. By following the instructions provided, the reader willingly assumes all risks in connection with such instructions.
- If the reader has questions about any of the guidelines provided by SkinAct we recommend they contact their nearest Esthetician Training Center to obtain more explicit instruction on Facial Machine Usage.

## Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH:

- Pacemakers, metallic implants, or other electronic medical devices.
- Heart conditions or a history of heart disease.
- Pregnancy.
- Inflamed, irritated, or damaged skin.
- Eczema, dermatitis, swelling, or sunburn.
- Epilepsy or other seizure disorders.
- Clients with reduced sensation should use under medical supervision.
- Malignant tumors.

## Precautions

- **Inspect the Device:** Check for any visible damage to the device or its power cable before and after each use.
- **Unplug When Not in Use:** Always disconnect the device from the power source before cleaning, storing, or performing maintenance.
- **Avoid Humidity:** Do not operate the device in excessively humid environments.
- **Position Cables Safely:** Ensure the power cable is positioned to prevent tripping hazards and is plugged into a grounded socket.
- **Keep Away from Children:** The device and its accessories should be stored securely out of reach.
- **Report Issues Promptly:** If you notice abnormal functioning, contact your distributor or an authorized technician immediately.
- **Remove Jewelry:** All jewelry and metal piercings must be removed prior to performing microcurrent treatments.

## Safety Instructions

- Plug the device into a properly grounded electrical outlet with the appropriate voltage.
- Do not plug the device into a extension cord.
- Do not use the device if it or its power cable is damaged.
- Never touch the device, power plug, or cables with wet hands.
- Avoid attempting to repair or modify the device. Repairs should only be performed by authorized technicians.
- Store the device in a cool, dry place away from direct sunlight and moisture.
- Avoid twisting or bending cables.
- Turn off the main power after each use, unplug device when not in use.

## Probe Care Instructions

- Do not leave the probe active without contact with the skin during treatment, as overheating may shorten its service life.
- Do not attempt to disassemble the stainless steel probe or handle, as this may damage the probe.
- Do not immerse the probe in water for cleaning, as this may cause damage.
- Clean probe with warm water and soap followed with a EPA disinfectant.
- Replace the probe if it is damaged or if the surface is scratched.

## General Notes

### Before Treatment

- Before starting any treatment please clean any residual traces of make up, oil, and soap from skin surface.

### During Treatment with Microcurrent

- Please keep in mind that in order for the microcurrent to be effective it has to be conductive. To increase conductivity we highly recommend you to use conductive adhesive gel.


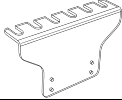
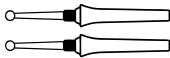
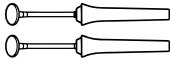
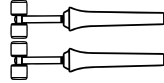
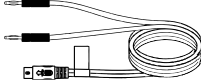


### After Treatment

- We recommend using a moisturizer to help skin absorb and retain moisture to improve elasticity and maintain a youthful experience.


**Note:** As with all electrical facial applications, the Microcurrent Operator should receive formal training on Microcurrent treatment on a client.

- Esthetic Microcurrent Technology uses electrical impulses to help increase ATP (adenosine triphosphate) production at the cellular level to enable tightening and toning of skin and muscles in the treatment area to provide a youthful and vigorous look.
- Micro amp stimulation works with the body's own electrical impulses to increase cellular reactions that slow down with age, due to injury, or due to stress. Microcurrent mimics the electrical impulses of the body to lead to repairing of distressed tissue.
- However the Skin Act Microcurrent is low amperage, there are still many safety guidelines that the operator must be aware of before performing this or any electrical treatment on clients.

## Included Items


Qty	Part Name	Image
1	Host Machine	
1	Accessory Rack	
2	Ball Probes	
2	Mushroom Probes	
2	Roller Probes	
1	Probe Connection Cable	
2	Magic Gloves	
1	Power Cable	

## Device Components

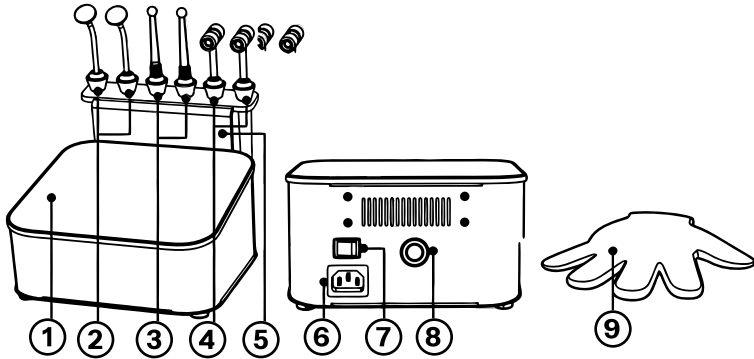


- 1. Control Panel
- 2. Mushroom Probe
- 3. Ball Probe
- 4. Roller Probe
- 5. Accessory Rack
- 6. Power Cord Outlet
- 7. Power Switch
- 8. Microcurrent Port
- 9. Magic Gloves

## Microcurrent Probe Introduction

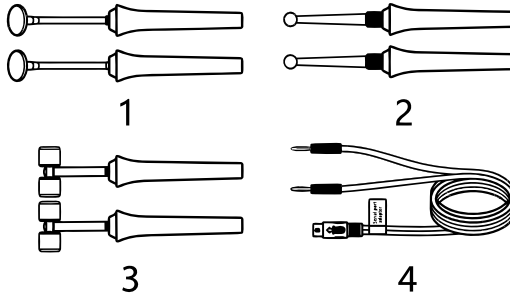


- 1. Mushroom Probe
- 2. Ball Probe
- 3. Roller Probe
- 4. Probe Connection Cable

## Device Components (Interface)

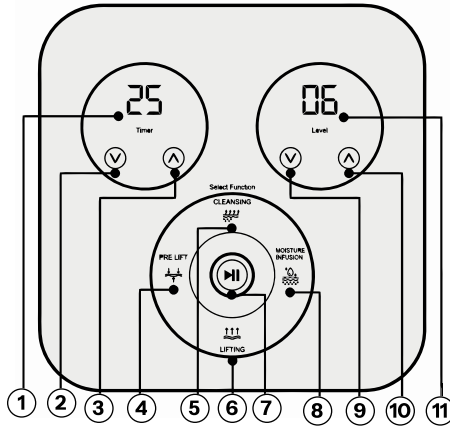

- |                   |                           |
|-------------------|---------------------------|
| 1. Time display   | 7. Start/Pause            |
| 2. Decrease Timer | 8. Moisture Infusion Mode |
| 3. Increase Timer | 9. Decrease Energy Level  |
| 4. Pre-lift Mode  | 10. Increase Energy Level |
| 5. Cleansing Mode | 11. Energy Level Display  |
| 6. Lifting Mode   |                           |

## Assembly Instructions

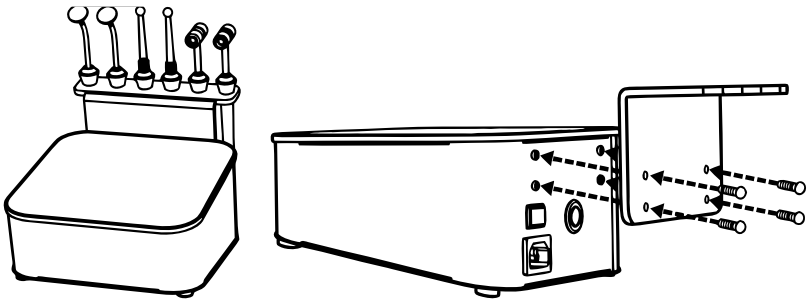

1. Secure the attachment rack to the back of the main unit using screws, following the direction of the arrows.
2. Place each attachment onto the rack.
3. Plug the handle connector into the corresponding ports on the back of the main unit. Preparation is now complete.
4. Connect the power cord to the main unit.

## Instructions for Use

1. Turn on the main power switch on the back of the device. The power light on the front will turn on, and the time display will show “15”, and the device will enter standby mode.
2. Use the touch-sensitive controls to select the desired function by tapping the corresponding icon.
3. Adjust the microcurrent energy level intensity between levels 1-9 using the corresponding controls.
4. Press the button to begin treatment.

## Function Overview

### Cleansing

- Uses a gentle electrical current to draw out impurities from the skin.
- Creates a mild electrical field to attract and remove dirt, oil, and debris from pores.
- Provides deep cleansing and prepares the skin for subsequent treatments.
- For optimal results, use with a desincrustation solution or a sodium bicarbonate-based product to enhance impurity removal.

**Client Feel:** Hydrated skin, Enhanced product absorption

### Moisture Infusion


- Uses a low-level current to increase skin permeability.
- Allows water based serums to penetrate more effectively.
- Enhances delivery of active ingredients.
- Improves hydration and promotes a plumper, more radiant complexion.

**Client Feel:** Hydrated skin, Enhanced product absorption

### Pre-Lift

- Applies a low-level electrical current to stimulate facial muscles.
- Helps awaken and prepare muscles for lifting treatments.
- Improves effectiveness of toning and contouring procedures.

**Client Feel:** Light muscle contractions, Mild tingling sensation

**Lifting** 

- Stimulates natural muscle contractions to tone and contour the skin.
- Helps relax facial muscles while lifting sagging areas.
- Defines facial contours and reduces the appearance of fine lines.
- Promotes firmer, smoother, and more youthful-looking skin.

**Client Feel:** Gentle massaging effect, Very mild current sensation

## Operating Instructions

- **Face Cleansing:** Thoroughly cleanse the face using a gentle cleanser, ensure all makeup, oil and impurities are removed.
- **Instrument Cleaning:** Wipe the probe with a cleaning swab or a soft cloth lightly dampened with water, ensure the probe is clean and hygienic before use.
- **Skin & Equipment Preparation:** Ensure the treatment is completely dry before starting.
- **Application Medium:** Use a specialized conductive gel to enhance conductivity and ensure smooth operation this helps reduce skin traction and provides lubrication and hydration. Do not use tap water, essential oils, lotions, or oil-based products, as they may reduce effectiveness or damage the device.
- **Cleansing/Hydration Mode:** Use in combination with toner, serum or moisturizing gel. This helps maintain skin hydration and improve absorption or cleaning results.

## Treatment Areas & Techniques

### Facial Sections:

- Divide the face into the following areas: forehead, eye area, cheeks, nose wings, nasolabial folds and jawline.
- Each area should be 3 to 7 passes per area.

### Core Technique Principles:

- **Slow Movement:** Move the probe slowly and at a steady pace; avoid rapid or abrupt movements.
- **Full Contact:** Keep the probe in constant contact with the skin; do not lift or suspend during treatment.
- **Lifting Direction:** Keep the probe in constant contact with the skin; do not lift or suspend during treatment.
- **Pinch Technique:** Using two probes to pinch muscle tissue to enhance lifting results. Use this technique for cheeks, jawline, and orbital (eyebrow) area.

## Area-Specific Techniques

*Follow the movement directions shown in Figure 1 (Page )*

### Forehead:

- Glide from the center of the eyebrows upward towards the hairline.

### Eye Area (Orbital):

- Start at the inner corner of the eye
- Move along the orbital bone outward and upward toward the temples.
- **Avoid direct contact with eyeball.**
- Use light pressure; optional pinch technique on eyebrow area

**Nasolabial Folds:**


- Lift diagonally upward from the sides of the nostrils to below cheekbones.

**Jawline:**

- Start at the center of the chin
- Follow the jawline upward toward the back of the ear
- Use pinch technique for improved definition

**Neck:**

- Move in upward strokes from the base of the neck toward the jawline
- Use low to moderate intensity only
- Apply gentle pressure due to sensitivity of the area

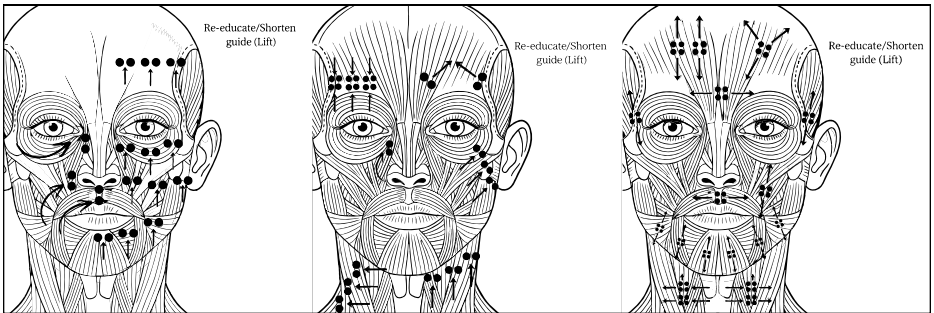

**IMPORTANT: Avoid using the device over the Thyroid area, as stimulation in this region may pose serious health risks.**

## Magic Gloves

The Magic Gloves provides microcurrent stimulation across a wide surface area, allowing even distribution of energy across the hands, wrist and forearms. The gloves provide full-contact coverage making them ideal for circulation enhancement, muscle relaxation, and overall skin conditioning.

### Preparation Before Use

#### Hands & Arms:

- Clean hands, wrists, and forearms with a mild cleanser.
- Ensure no cream, oil, or residue remains.
- Skin must be completely dry before use.

#### Inside the Gloves:

- Wipe the internal electrode contact points with a soft, damp cloth.
- Use water or disinfectant alcohol as needed to maintain hygiene.
- Wear a thin pair of vinyl or nitrile gloves underneath to protect from the current.

#### Skin Preparation:

- Ensure both hands and the inner glove surfaces are completely dry.

#### Instrument Preparation:

- Connect the gloves to the main unit using the cable.
- Turn on the device.
- Select desired mode and begin at the lowest intensity level.

### **Wearing the Gloves Properly:**

- Wear the gloves as you would standard gloves
- Ensure a snug fit across fingers, hands, and wrists
- Avoid looseness to maintain consistent electrode contact

## **How To Use**

### **Core Technique Principles:**

- **Direction:** Always work upward, from fingertips toward the arm (centripetal movement)
- **Moisten for Conductivity:** Lightly dampen the Magic Gloves with distilled water or a water-based conductive spray before putting them on.
- **Gel Application:** Alternatively, apply a thin, even layer of conductive gel to the client's skin before performing the glove massage. Using the gloves dry will increase electrical resistance, leading to client discomfort.
- **Gradual Intensity:** Start at a low level and increase gradually for comfort.

### **Treatment Techniques:**

#### **Finger Area:**

- Begin at the fingertips
- Use a gentle pushing motion toward the palm

#### **Back of Hand & Palm:**

- Start at the base of the fingers
- Lift toward the wrist
- Use circular motions to enhance massage effect

#### **Wrist & Forearm:**

- Begin at the wrist and lift upward toward the elbow
- Repeat multiple times to support circulation and reduce swelling

**Static Stimulation:**

- Hold the gloves in place on targeted areas (e.g., wrinkles or sore spots)
- Allow microcurrent to work without movement for a short duration

**Treatment Protocol:**

- 5-20 minutes per session, 2-3 times per week to avoid overuse.

**Aftercare:**

- Wipe the interior of gloves with a soft cloth after each use.
- Allow to air dry in a well-ventilated area.
- Do not immerse gloves in water.

**Post Treatment Care:**

- Apply hand care cream or serum after use.
- Skin Absorption is enhanced after treatment.

## **Usability & Safety**

**Normal Sensation:**

- A mild tingling, slight numbness, or gentle muscle stimulation may be felt
- Warmth and light vibration during use are normal

**Abnormal Sensation:**

- If sharp stinging, burning, or discomfort occurs, stop use immediately.
- This may indicate excessive intensity or improper contact (e.g., dry or uneven skin surface)

## Specifications

<b>Product Name:</b>	MicroLift Pro Microcurrent Machine
<b>Voltage:</b>	AC100-120V/220-240V
<b>Frequency:</b>	50/60Hz
<b>Power Consumption:</b>	3W
<b>Dimensions(L*W*H):</b>	8.7 x 7.5 x 4.5in
<b>Net Weight:</b>	4.4 lbs

## Warranty Information

The manufacturer (the “warrantor”) warrants to the original purchaser that this equipment will be free from defects under normal and proper operation for a period of one (1) year from the date of purchase. This warranty covers all parts and labor for that one (1) year period. This warranty shall not apply to any equipment that has been subjected to abuse, misuse and alteration of any type or to any defect or damage caused by repair, replacement, substitution, or use with parts other than parts approved by the manufacturer. The manufacturer shall not be liable for the loss of use of this equipment, loss of time or business, inconvenience or any other indirect, consequential, special or incidental damages due to defects covered by this warranty or any implied warranty. To implement this warranty, contact the dealer from whom you purchased the equipment. If necessary, you will be asked to return all, or the appropriate part, of the equipment. If warranted, the repairs will be performed at no charge and the equipment returned to you. Unit owner must prepay all shipping costs when returning items for warranty service. This limited warranty is the only written or express warranty given by the manufacturer. This warranty gives you specific legal rights but you may also have other rights, which may vary from state to state or province to province.

### SKIN+ACT

Phone: .....1-800-418-0830

Email: .....info@skinact.com

Website: .....skinact.com