


SKIN+ACT

OxyFusion Microdermabrasion plus Oxygen Sprayer

PRODUCT MANUAL

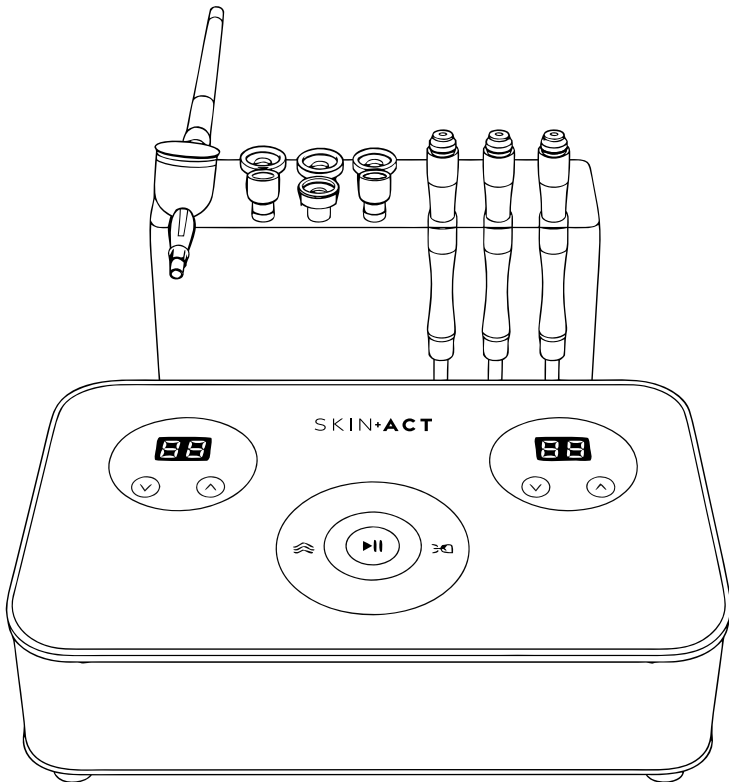

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

TABLE OF CONTENTS

Disclaimer & Contraindications 4

Precautions 5

Safety Instructions & Warning 6

Included Items 7

Device Components 8

Device Components — Interface 9

Microdermabrasion Diamond Tip Guide 10

Assembly Instructions 11

Instructions for Use — Vacuum & Spray Gun Functions 12

Instructions for Use — Microdermabrasion Function 13

Oxygen Sprayer Cleaning & Maintenance 14

Microdermabrasion Tip Cleaning & Maintenance 15

Specifications 16

Warranty Information 17

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH THE FOLLOWING:

- Pacemakers, metallic implants, or other electronic medical devices.
- Heart conditions or a history of heart disease.
- Pregnancy.
- Inflamed, irritated, or damaged skin.
- Severe acne rosacea or vascular conditions.
- Epilepsy or other seizure disorders.
- Clients undergoing medical treatments or with underlying health conditions must obtain prior clearance from a physician.

Precautions

- **Inspect the Device:** Check for any visible damage to the device or its power cable before and after each use.
- **Unplug When Not in Use:** Always disconnect the device from the power source before cleaning, storing, or performing maintenance.
- **Avoid Moisture and Outdoor Use:** Do not operate the device outdoors or in damp, humid, or wet environments.
- **Position Cables Safely:** Ensure the power cable is positioned to prevent tripping hazards and is plugged into a grounded socket.
- **Unplug After Daily Use:** Disconnect the equipment from the power source at the end of each workday.
- **Keep Away from Children:** The device and its accessories should be stored securely out of reach.
- **Report Issues Promptly:** If you notice abnormal functioning, contact your distributor or an authorized technician immediately.
- **Control the Water Flow:** Keep the water pressure moderate and adjust the flow control lever to maintain a steady, drip-free stream.
- **Clean in the Correct Direction:** When performing focused rinsing, work from lower areas toward higher areas for effective cleaning.
- **Connect Air Hoses Correctly:** Match each hose label to its designated outlet to prevent incorrect installation or equipment malfunction.
- **Pause Before Replacing the Microdermabrasion Tip:** Stop the main unit's output before replacing the microdermabrasion tip.
- **Protect the Spray Hose:** Do not bend the hose or block the air nozzle while the spray function is operating, as this may damage the main unit.
- **Repair Cracked Hose Ends:** Trim away the damaged section with scissors and securely reconnect the hose to the appropriate connector.





Safety Instructions

- Keep the device and its accessories out of reach of children. Plug the device only into a properly installed and grounded outlet. Make sure the local power supply voltage matches the device's technical specifications.
- Keep the power outlet easily accessible so the device can be unplugged quickly in an emergency. Always unplug the device by holding the plug, not by pulling the power cord.
- Do not tamper with electrical components or open or repair the device yourself. Repairs should only be performed by the distributor or a qualified technician.
- Do not operate the device if the unit or power cord shows visible damage. Never touch the device, plug, or power cord with wet hands.


Warning

- When using the device, monitor the hose connecting the attachment to the main unit and avoid pulling it excessively. Do not allow water or corrosive liquids to splash onto the main unit.
- Connect each hose only to its designated outlet. Never connect the spray hose to the suction outlet, as this may draw liquid into the machine and damage the main unit.
- Do not use attachments with sharp edges, rough surfaces, or visible damage. Replace damaged attachments through your dealer or an authorized service center.
- If the device operates abnormally, shut it down immediately and unplug the power cord. Contact an authorized dealer or qualified technician for inspection and repair. Do not open or repair the device yourself.
- A damaged power cord must be replaced with an approved replacement cord or component supplied by the manufacturer or an authorized service center.
- Do not use or store the device in bathrooms, damp areas, or near flammable or explosive materials.

Included Items

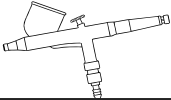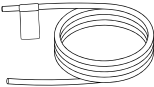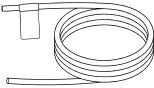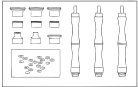
Qty	Part Name	Image
1	Host Machine	
1	Oxygen Injection Handle	
1	Spray Hose Tube	
1	Suction Hose Tube	
1	Diamond Probes & Handles	

Device Components

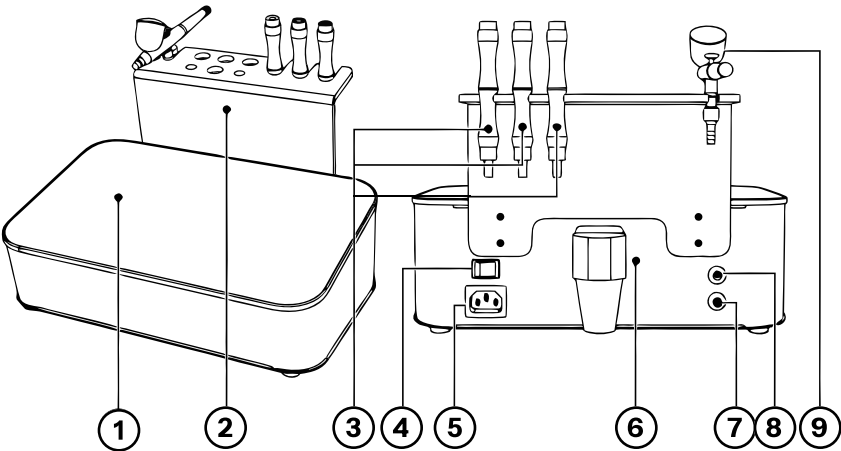


- | | |
|------------------------------|--------------------------------------|
| 1. Control Panel | 6. Filter Port |
| 2. Attachment Rack | 7. Microdermabrasion Function Output |
| 3. Microdermabrasion Handles | 8. Oxygen Output |
| 4. Main power switch | 9. Oxygen Gun |
| 5. Power Outlet | |

Oxygen Spray Gun

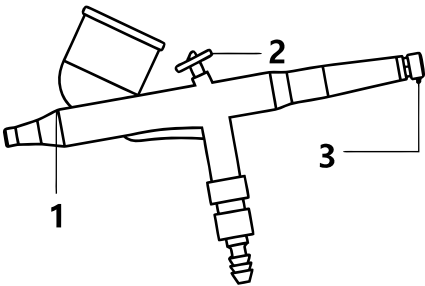

- | |
|----------------------|
| 1. Reservoir |
| 2. Oxygen Button |
| 3. Flow Control Knob |

Device Components (Interface)

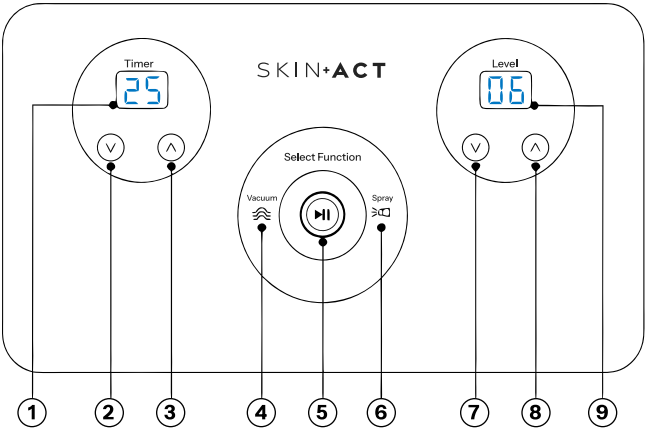

- | | |
|---------------------------------|--------------------|
| 1. Time display | 6. Oxygen Gun Key |
| 2. Decrease Timer | 7. Decrease Energy |
| 3. Increase Timer | 8. Increase Energy |
| 4. Microdermabrasion Vacuum Key | 9. Energy Display |
| 5. Start/Pause | |

Microdermabrasion Diamond Tip Guide




D-75: Provides deep, intensive exfoliation for thick or rough skin. Ideal for body areas such as the elbows, knees, and other areas with heavy buildup or uneven texture. Avoid use on delicate or sensitive areas.

D-100: Provides a more focused exfoliation and is ideal for treating scars, fine lines, minor texture irregularities, and small areas of uneven skin. Best used with controlled, gentle passes to avoid over-exfoliation.


D-150: Provides light to moderate exfoliation. Ideal for general facial treatments, sensitive areas, and clients who are new to microdermabrasion. This tip can be used to help improve dullness, mild congestion, and uneven skin texture.

D-200: For versatile use on the face, nose, and body. Helps treat blackheads, oil buildup, congestion, and uneven skin texture.


Assembly Instructions

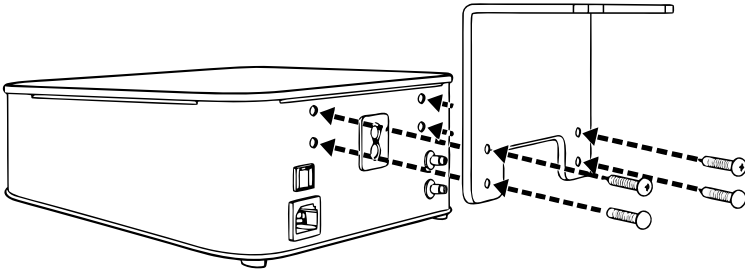


1. Secure the attachment rack to the back of the main unit using the provided screws, following the direction of the arrows.
2. Loosen the air pump mounting screws on the bottom of the unit before operation to prevent abnormal noise during use. Connect the flexible air hose labeled “Spray Tube” to the spray gun, then connect the other labeled end to the spray output port on the main unit, as shown in the illustration.

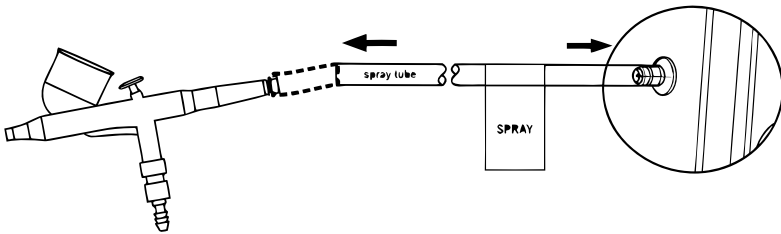

3. Insert the end of the flexible hose labeled “Vacuum Tube” into the vacuum handle. Connect the labeled end of the hose to the corresponding vacuum output port on the main unit.

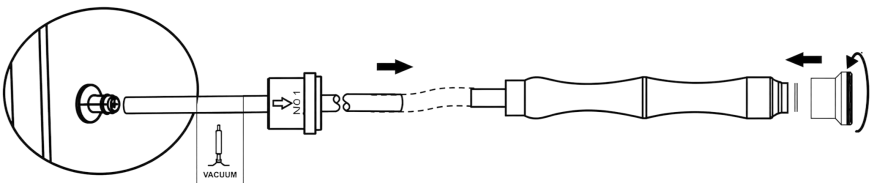

4. Place the attachment securely on the attachment rack, as shown in the diagram.

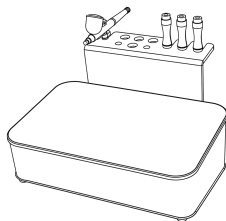

Instructions for Use

This device uses touch-sensitive controls. Tap the appropriate function button to select the desired treatment mode.

Vacuum Function:

While the device is in standby mode, select the Vacuum function. Set the treatment time to 15 minutes and the energy intensity to Level 1–5. Adjust the settings as needed before beginning treatment.

Spray Gun Function:

1. While the device is in standby mode, select the Spray function.
2. Fill the spray gun reservoir with an Oxygen Activator, such as the SkinAct Collagen, SkinAct Hyaluronic Acid, SkinAct Vitamin C Brightening, SkinAct Lavender, or SkinAct Detox formula. Select the product according to the client's skin type and treatment goals. Do not use thick, oily, corrosive, or unapproved solutions in the spray gun.
3. Set the treatment time between 1 and 30 minutes using the time adjustment controls. The energy intensity cannot be adjusted for this function.
4. Press the Start/Pause button to begin operation. Adjust the spray output using the control knob located behind the spray gun handle.
5. Ask the client to close their eyes and mouth before treatment. Begin spraying at the forehead and work downward, keeping the spray gun at a comfortable distance from the face. Use extra caution around the eyes, nostrils, and mouth.
6. After treatment, pause the device and gently blot away excess product with cotton pads or soft tissue. Empty and clean the spray gun reservoir and hose according to the device's cleaning instructions.

Instructions for Use

Microdermabrasion Function:

1. While the device is in standby mode, select the Vacuum function.
2. Set the treatment time between 1 and 30 minutes using the time adjustment controls.
3. Adjust the suction intensity to the desired level. The suction level can be set from Level 1 to Level 9. Begin with a low setting, then gradually increase the suction according to the client's skin type, treatment area, and comfort level.
4. Press the Start/Pause button to begin operation.
5. Hold the microdermabrasion handle firmly, similar to holding a pen. Place the selected diamond tip flat against the skin to create a secure seal and allow proper suction.
6. Gently glide the handle across the skin using smooth, controlled strokes. Keep the handle moving continuously and avoid holding the tip in one area for an extended period.
7. Select the appropriate diamond tip for the treatment area and the client's skin condition. Use lower suction and finer tips on sensitive areas, and avoid treating irritated, inflamed, broken, or compromised skin.
8. After treatment, press the Start/Pause button to stop the device. Clean and disinfect the handle and reusable attachments according to the cleaning instructions.

Oxygen Sprayer Cleaning & Maintenance

After each use, turn off and unplug the device before cleaning.

1. Empty any remaining treatment solution from the spray gun reservoir. Do not leave product inside the reservoir or hose between treatments, as residue may dry and clog the spray nozzle.
2. Fill the reservoir with a small amount of distilled water. Operate the spray function for several seconds to flush the reservoir, internal tubing, and nozzle. Continue until the spray runs clear.
3. Wipe the inside of the reservoir with a clean, lint-free cloth or cotton swab.
4. Wipe the spray gun, handle, hose exterior, and main unit with a soft cloth lightly dampened with an appropriate disinfectant. Do not spray liquid directly onto the main unit or allow moisture to enter electrical openings.
5. Wipe the exterior of the spray gun with a soft cloth lightly dampened with an appropriate disinfectant. Do not soak or immerse the entire spray gun.
6. Inspect the nozzle regularly for buildup or reduced spray output. If necessary, turn off the device and gently clear the nozzle using the manufacturer-provided cleaning needle. Never enlarge or force the nozzle opening.
7. Check the air hose and connections for cracks, leaks, looseness, or residue. Replace damaged components before operating the device.
8. Allow all parts to dry completely before reassembling and storing the spray gun. Store it empty in a clean, dry area.
9. Use only thin, water-based solutions suitable for spray gun application. Do not use oils, thick serums, abrasive products, or corrosive liquids, as they may clog or damage the spray gun.

Microdermabrasion Tip Cleaning & Maintenance

After each use, turn off and unplug the device before cleaning.

1. Remove the diamond tip from the handpiece and separate any removable filters, gaskets, or collection components.
2. Wash the tip with warm water and a mild, non-abrasive detergent to remove skin debris and product residue. Use a soft brush to clean around the diamond surface and inside the opening.
3. Rinse the tip thoroughly with clean water, then disinfect it using a manufacturer-approved disinfectant suitable for reusable metal instruments. Follow the disinfectant's required dilution and contact time.
4. Do not use abrasive pads, metal brushes, bleach, acetone, or harsh chemicals, as these may damage the diamond surface or metal finish.
5. Inspect the tip opening for buildup or blockage. Clear debris with a soft cleaning brush or approved tool. Do not use sharp objects that could scratch or enlarge the opening.
6. Allow the tip to air-dry completely on a clean, lint-free surface before reattaching it to the handpiece or placing it in storage.
7. Check each tip regularly for cracks, rough edges, corrosion, loose diamond coating, or other damage. Discontinue use and replace any damaged or excessively worn tip.
8. Replace disposable filters after each client or according to the manufacturer's instructions. Never reuse a contaminated filter.
9. Store clean, dry tips in a covered container or designated holder to protect them from dust and cross-contamination.

Specifications

Product Name:	OxyFusion Microdermabrasion plus Oxygen Sprayer
Voltage:	AC 100-240 V
Frequency:	50/60 Hz
Power Consumption:	22 W
Dimensions(L*W*H):	14.5 x 14 x 6.4”
Net Weight:	6.5 lbs

Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor's discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

SKIN+ACT

Phone:1-800-418-0830
Email:info@skinact.com
Website:www.skinact.com