

SKIN+ACT

Madison II

Medical Spa Electric Treatment Table

PRODUCT MANUAL

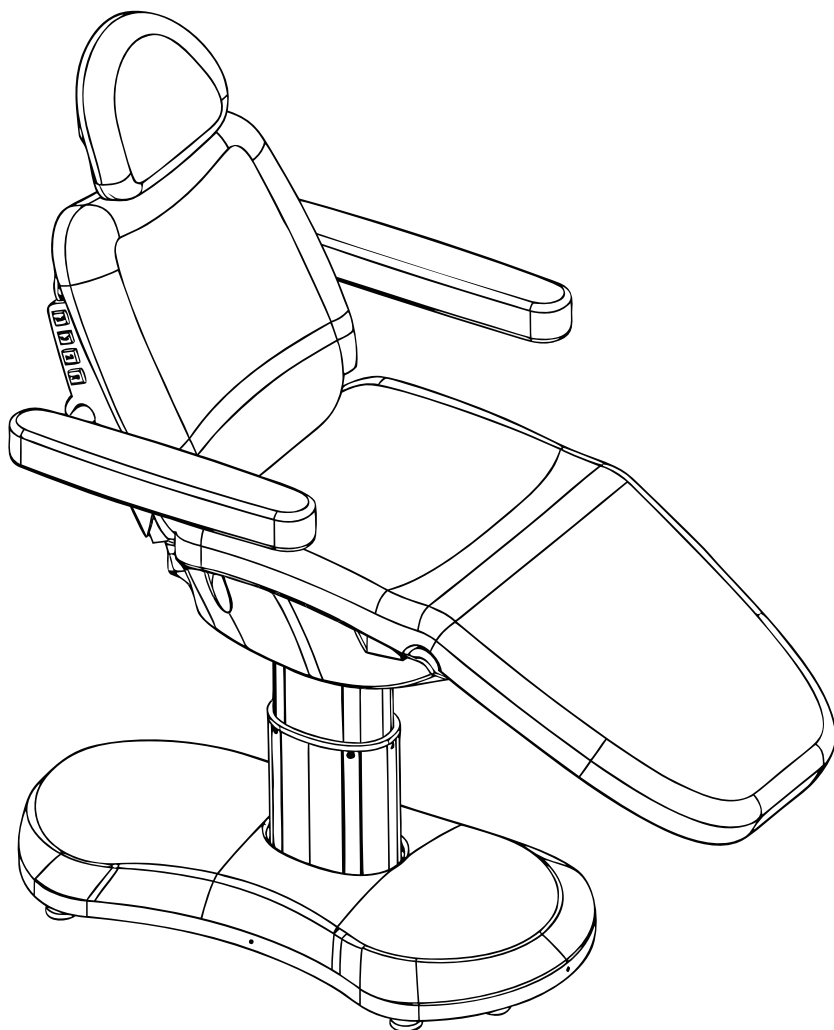


TABLE OF CONTENTS

Notice & Disclaimer 3

Stability and Rotation Safety 4

Safety and Operating Instructions 5

Precautions 5

Dimensions 6

Angle Specifications 7

Installation 8

Remote Control Instructions 9

Built-In Unit Controls 10

Lift Adjustment & Backrest Adjustment 11

Seat Cushion Adjustment & Footrest Adjustment 12

Pillow Adjustment & Maintenance 13

Troubleshooting 14

Technical Specifications 15

Warranty Information 16

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make any express or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

DISCLAIMER

This product is intended for professional use by trained spa, salon, esthetic, wellness, or other qualified professionals. It must be used only for its intended purpose and in accordance with all instructions, warnings, contraindications, maintenance requirements, and safety guidelines provided in this manual.

The information in this manual is for general product operation and safety reference only. It does not replace professional training, licensing requirements, medical advice, service instructions, applicable laws, or workplace safety rules. Users are responsible for proper training, safe operation, inspection, client screening when applicable, sanitation, maintenance, and compliance with all applicable regulations.

SkinAct/Spa and Equipment does not provide medical advice and does not diagnose, treat, cure, or prevent any disease, medical condition, injury, or skin disorder. Any treatment descriptions, suggested uses, or expected results are for general cosmetic and professional use reference only, and results may vary.

Before each use, inspect the product, power cord, accessories, attachments, moving parts, and related components, as applicable, for damage, looseness, contamination, instability, or abnormal operation. Do not use the product if it appears damaged, defective, improperly assembled, unsafe, or if any component is missing or malfunctioning.

The user assumes full responsibility for proper assembly, installation, operation, client selection when applicable, treatment technique when applicable, cleaning, maintenance, and storage. SkinAct/Spa and Equipment shall not be responsible for injury, property damage, treatment complications, equipment damage, loss of business, or other damages resulting from misuse, improper use, lack of training, failure to follow instructions, unauthorized repairs or modifications, improper maintenance, or use outside the intended purpose.

Do not modify, disassemble, repair, or alter this product unless authorized in writing by SkinAct/Spa and Equipment or performed by an approved service provider. Unauthorized modification, misuse, improper maintenance, or failure to follow this manual may void the warranty.

Product designs, specifications, components, accessories, images, and instructions may vary from the actual product and may be changed or updated without prior notice.

This product contains electric adjustment components and moving parts. Improper use may cause injury, pinching, equipment damage, or unsafe operation. Users are responsible for operating the table safely and following all instructions, warnings, and maintenance requirements in this manual. The user is solely responsible for confirming that use of this device is permitted under applicable state/local laws, scope-of-practice rules, licensing requirements, and workplace safety standards. This device is not intended for self-use, home use, or use by untrained individuals.

Stability and Rotation Safety

When operating a hydraulic rotating chair or single-post electric treatment table, the operator is responsible for maintaining stability and ensuring client safety. Always follow the instructions provided for the specific model.

To ensure stability

- 1. Place the chair or table on a level, dry, and non-slip surface. Adjust the leveling feet, if equipped, so the base remains stable and does not rock.
- 2. Inspect the base, controls, adjustment mechanisms, and fasteners before each use. Do not operate the unit if any component is loose, damaged, or functioning improperly.
- 3. Keep the client's weight centered over the base. Position the chair or tabletop in line with the base whenever possible, particularly during treatment.
- 4. Do not allow anyone to sit, stand, lean, or place excessive weight on the headrest, backrest, leg rest, foot section, or armrests.
- 5. Whenever possible, rotate or reposition the unit before the client enters it. Do not rotate or adjust the unit while the client is entering or exiting.
- 6. If rotation is necessary while occupied, place the client in a stable upright position, keep the client centered, support the chair or table securely, and rotate it slowly.
- 7. Keep hands, feet, clothing, cords, and other objects away from moving parts and potential pinch points during rotation or adjustment.
- 8. Ensure all rotation locks and adjustable sections are securely engaged before beginning treatment.
- 9. Do not exceed the stated weight capacity or apply sudden, concentrated, or excessive downward pressure. These products are not intended for massage or procedures that require substantial downward force.
- 10. Do not leave a client unattended while the unit is elevated, reclined, rotating, or being adjusted.
- 11. Stop using the unit immediately if it becomes unstable, shifts unexpectedly, makes unusual sounds, or fails to hold its selected position.



Warning: Do Not Sit or Apply Excess Force



Do not sit, stand, jump, or place excessive weight on the backrest or footrest. Doing so may cause the product to tip, damage the backrest/footrest joints, or result in injury.

30%

Back Cushion

55%

Seat Cushion

15%

Leg Cushion



Safety and Operating Instructions

To ensure safe and proper operation of this electric treatment chair/table, please read the following instructions carefully and retain this manual for future reference.

- **Proper Setup and Grounding:**
- This electric treatment table requires a three-wire power cable and must be connected to a grounded three-prong power outlet. This ensures proper grounding, prevents accidental electric shock, and supports normal operation.
- **Avoid Water Exposure:**
- Keep all plugs, controls, and connecting cables away from water and other liquids during use.
- **Inspect for Damage:**
- Immediately discontinue use of the table if any plugs, buttons, or connecting cables are damaged.
- **Usage Frequency:**
- The electric push rod of the treatment table should only be operated for up to 6 minutes per hour.
- **Professional Repairs Only:**
- Repairs and maintenance must be conducted exclusively by trained and qualified professionals.

Precautions

1. Health Restrictions:

- Pregnant individuals should avoid using this product.
- Clients with injuries or skin conditions that could be aggravated by positioning or contact with the chair should not use the product without appropriate medical clearance.

2. Use by Minors and Individuals with Impaired Awareness:

- Children under 14 years of age and individuals with impaired consciousness must not use this product without supervision.

3. Inspection Before Use:

- Inspect the power cord and plug for damage before each use. If damage is found, they must be replaced by a qualified operator or technician.

4. Voltage Compliance:

- Confirm that the product is connected to the correct voltage as specified. Incorrect voltage can result in product damage.

5. Electrical Safety:

- Do not handle the power plug with wet hands to prevent electric shock.
- Ensure the product is fully assembled and properly installed before connecting it to power.

6. Cord Safety:

- Secure the power cord to prevent tripping, pulling, or damage.



7. Cleaning:

- Clean the product's surface with a soft towel dampened with neutral detergent.


8. Unplugging When Not in Use:

- Always unplug the product when it is not in use.

Dimensions

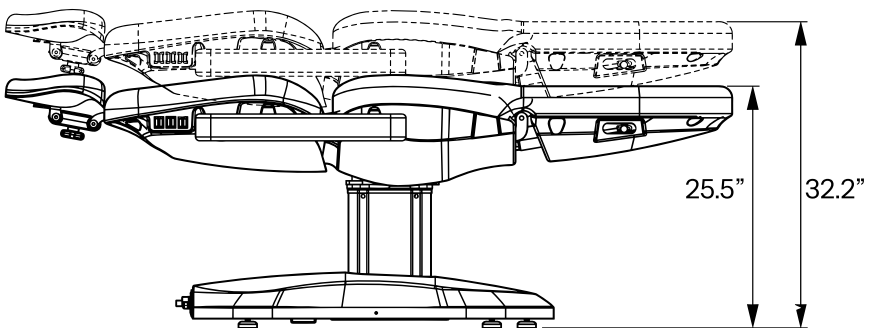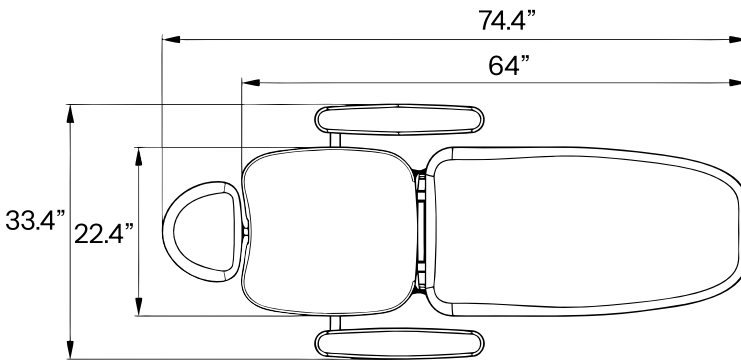

Angle Specifications

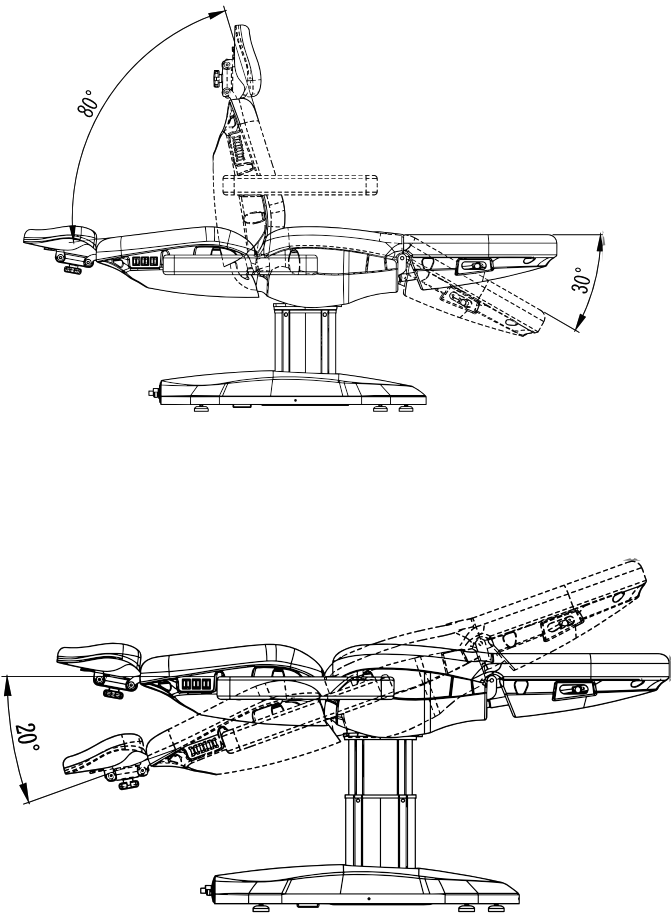



Installation

1. Install the headrest.


Attach the headrest to the unit. Loosen the headrest adjustment knob, set the headrest to the desired height, then tighten the knob to secure it in place. See Figure ①.

2. Install the armrests.

Insert the left and right armrest connecting rods into the corresponding mounting slots on the unit. Rotate each armrest counterclockwise 90° until it is securely locked in place. See Figure ②.

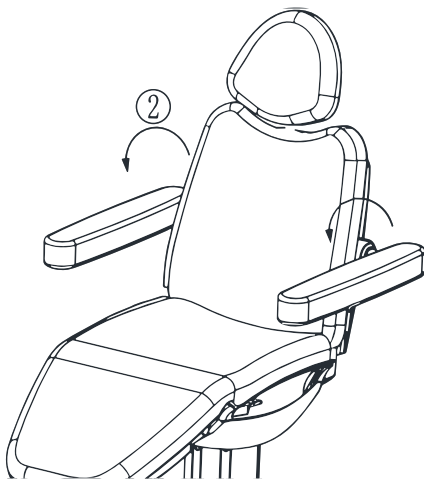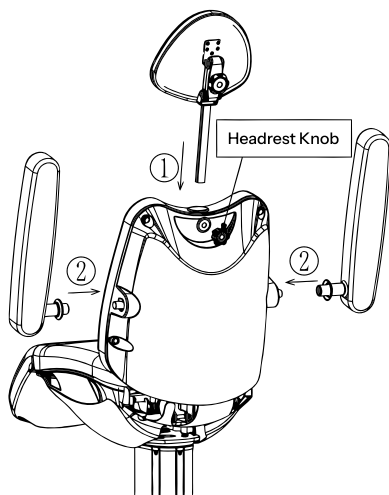

Remote Control Instructions

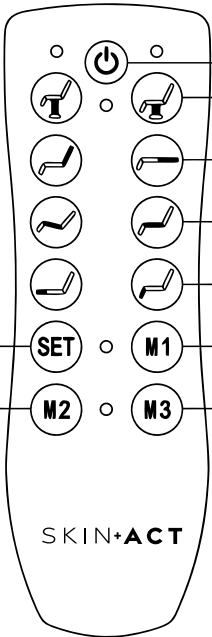

The diagram shows a vertical remote control with the following buttons and callouts:

- 1** Power / Reset Button: A power button icon at the top.
- 2** Unit Height Adjustment: Two buttons with up and down arrows for the unit height.
- 3** Backrest Adjustment: Two buttons with up and down arrows for the backrest angle.
- 4** Seat Cushion Adjustment: Two buttons with up and down arrows for the seat cushion angle.
- 5** Leg Rest Adjustment: Two buttons with up and down arrows for the leg rest angle.
- 6** SET: A button with the text "SET" for saving memory positions.
- 7** M1: A button with the text "M1" for saving the current position.
- 8** M2: A button with the text "M2" for saving the current position.
- 9** M3: A button with the text "M3" for saving the current position.

SKIN+ACT

① Power / Reset Button
Press the Power/Reset Button to activate the hand controller. Press it again to return the unit to its original position. During the reset process, press the button again to stop movement.

② Unit Height Adjustment
Press the up or down button to raise or lower the entire unit.

③ Backrest Adjustment
Press the up or down button to adjust the angle of the backrest.

④ Seat Cushion Adjustment
Press the up or down button to adjust the angle of the seat cushion.


⑤ Leg Rest Adjustment
Press the up or down button to raise or lower the leg rest.

⑥–⑨ Automatic Mode Buttons
Press any automatic mode button to move the unit to a preset height and angle position.

Setting Automatic Mode Positions

Adjust the unit height, backrest, seat cushion, and leg rest to the desired position. Press and hold the SET button until the blue indicator light turns on to activate the memory function. Then press M1 to save the current position to the M1 button. Follow the same steps to save positions for M2 and M3.

Built-In Unit Controls

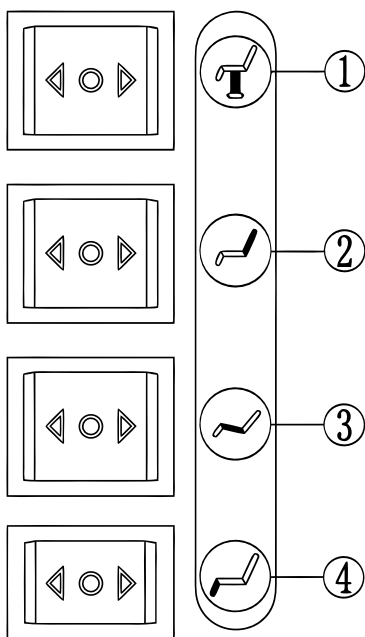

① Unit Height Adjustment

Press and hold the up button to raise the entire unit. Press and hold the down button to lower the entire unit.

② Backrest Adjustment

Press and hold the up button to increase the backrest incline angle. Press and hold the down button to decrease the backrest incline angle.

③ Seat Cushion Adjustment


Press and hold the up button to increase the seat cushion incline angle. Press and hold the down button to decrease the seat cushion incline angle.

④ Leg Rest Adjustment

Press and hold the up button to raise the leg rest. Press and hold the down button to lower the leg rest.


Lift Adjustment

1. Turn on the power switch before adjusting the unit height.
2. Press and hold the Raise button on the hand remote or built-in controls to lift the unit. 
3. Press and hold the Lower button on the hand remote or built-in controls to lower the unit. 

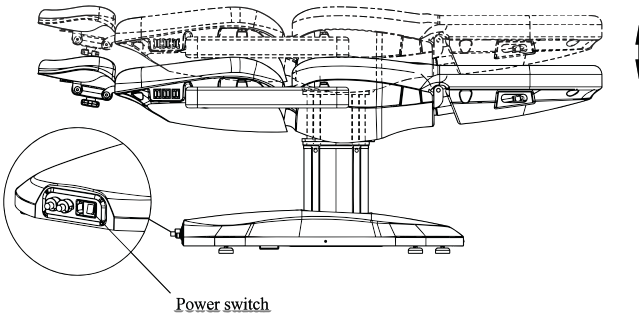


Backrest Adjustment

1. Press and hold the Backrest Up button on the hand remote or built-in controls to raise the backrest. 
2. Press and hold the Backrest Down button on the hand remote or built-in controls to lower the backrest. 

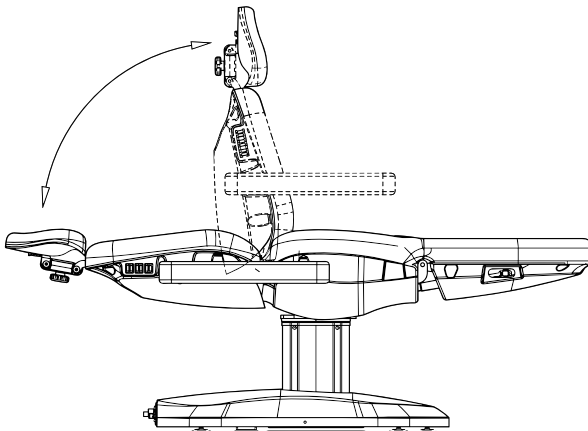


Seat Cushion Adjustment

1. Press and hold the Seat Cushion Up button on the hand remote or built-in controls to increase the seat cushion incline angle. 
2. Press and hold the Seat Cushion Down button on the hand remote or built-in unit switch to decrease the seat cushion incline angle. 

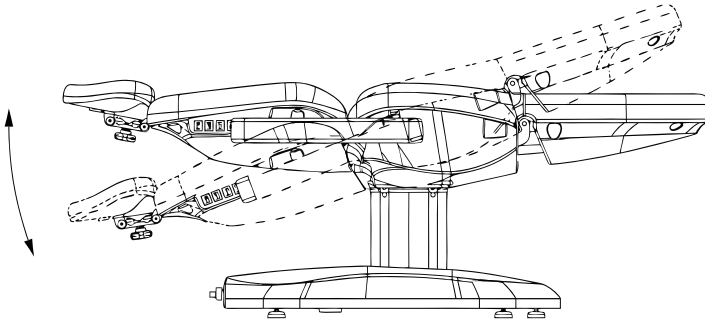

Footrest Adjustment

1. Pull back and hold the footrest adjustment handle to adjust the angle of the leg pad.
2. Release the handle once the leg pad is in the desired position to lock it in place.

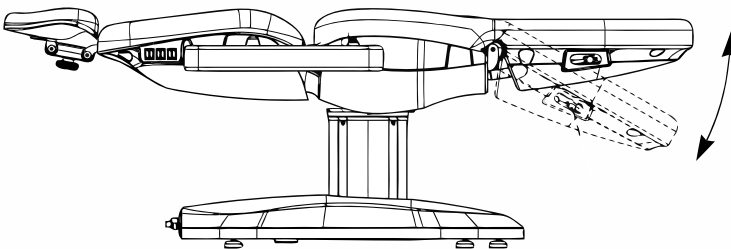


Pillow Adjustment

1. Adjust the pillow height.

Loosen Knob A to raise or lower the pillow to the desired height. Tighten the knob to secure it in place.

2. Adjust the pillow angle.

Loosen Knob B to rotate or tilt the pillow to the desired angle. Tighten the knob to lock the pillow in position.

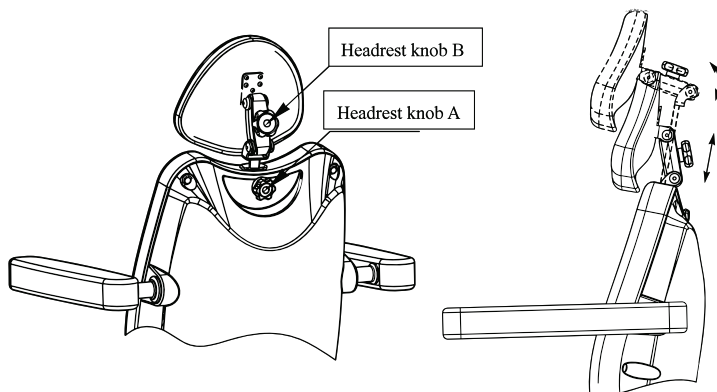

Maintenance

- Do not use bleach or other harsh chemicals when cleaning or sanitizing the product.
- For best results, we recommend cleaning with SkinAct All-In-One Upholstery & Disinfecting Multi-Purpose Cleaner (SKU24261).
- Clean and sanitize the upholstery after each treatment.
- Every six months, visually inspect all visible parts of the unit. If there are any abnormal noises or vibrations during operation, call Spa & Equipment/SkinAct at 1-800-418-0830.

Troubleshooting

Unit Does Not Respond When Pressing Hand Controller Keys

Solution: Check that the controller power switch is turned on and that the hand controller is securely connected. Make sure the power plug is inserted firmly and all transformer connectors are properly connected. If one function does not respond, the unit may have reached its adjustment limit; try pressing a different control key or the opposite direction key.

Abnormal Noise or Shaking During Lifting/Lowering

Solution: Check for loose screws or hardware around the lifting mechanism, base, and moving parts. Tighten all screws and hardware securely. If the noise or shaking continues, stop using the product and contact SkinAct/Spa & Equipment for assistance.

Power Indicator Light Does Not Turn On

Solution: Check that the power supply is properly connected. If the power indicator still does not light up, the fuse may need to be replaced. Replace the fuse with one of the same type and rating, or contact SkinAct/Spa & Equipment for further assistance.

Technical Specifications

Product Name:	Madison II Medical Spa Electric Treatment Table/Procedure Chair
Voltage:	100–130 V AC or 220–240 V AC
Frequency:	50/60 Hz
Power Consumption:	200 W, 2 A
Seat Tilt Angle:	20°
Length with Headrest:	74.4”
Width with Armrests:	33.4”
Width without Armrests:	22.4”
Adjustable Height Range:	25.5”- 32.2”
Weight Capacity:	440 lb
Net Weight:	158 lb

Warranty Information

The manufacturer/warrantor warrants to the original purchaser that this product will be free from defects in materials and workmanship under normal and proper use for a period of one (1) year from the original date of purchase, unless otherwise stated in writing for a specific product.

This limited warranty applies only to the original purchaser and is not transferable. Proof of purchase may be required to obtain warranty service.

This warranty covers parts and labor required to correct defects in materials or workmanship during the warranty period. At the warrantor’s discretion, the product may be repaired, replaced, or substituted with a comparable item or component.

This warranty does not cover damage or defects caused by misuse, abuse, improper assembly, improper installation, unauthorized modification, unauthorized repair, neglect, accident, improper cleaning, use of harsh chemicals, power surges, water or liquid damage, use outside the intended purpose, failure to follow the product manual, or use with parts/accessories not approved by SkinAct.

This warranty does not cover normal wear and tear, cosmetic damage, upholstery wear, stains, discoloration, scratches, dents, wheels/casters, bulbs, glass parts, filters, tips, probes, disposable parts, consumables, or accessories subject to regular replacement unless the defect is determined to be due to a manufacturing issue.

The warrantor shall not be liable for loss of use, loss of time, loss of business, lost revenue, inconvenience, or any indirect, incidental, special, or consequential damages arising from the use of this product or from any defect covered by this warranty.

To request warranty service, contact the dealer or SkinAct customer support. If return service is required, the purchaser may be asked to return the product or the affected component for inspection. The purchaser is responsible for properly packaging the item and prepaying all shipping costs to send the item in for warranty service. If the issue is covered under warranty, repair or replacement will be performed at no charge, subject to the terms of this limited warranty.

Warranty does not cover damage, contamination, upholstery damage, sanitation-related issues, injury claims, or equipment issues resulting from improper cleaning, improper maintenance, misuse, use by untrained users, or use outside applicable safety guidelines. This limited warranty is the only written or express warranty provided by the warrantor. No employee, representative, dealer, or third party is authorized to modify or extend this warranty unless confirmed in writing by SkinAct. This warranty gives you specific legal rights, and you may also have other rights, which vary by state or province.

SKIN+ACT

Phone:1-800-418-0830
Email:info@skinact.com
Website:www.skinact.com