


SKIN+ACT

# 2 in 1 Ultrasonic & Skin Scrubber Unite Elite Edition

PRODUCT MANUAL

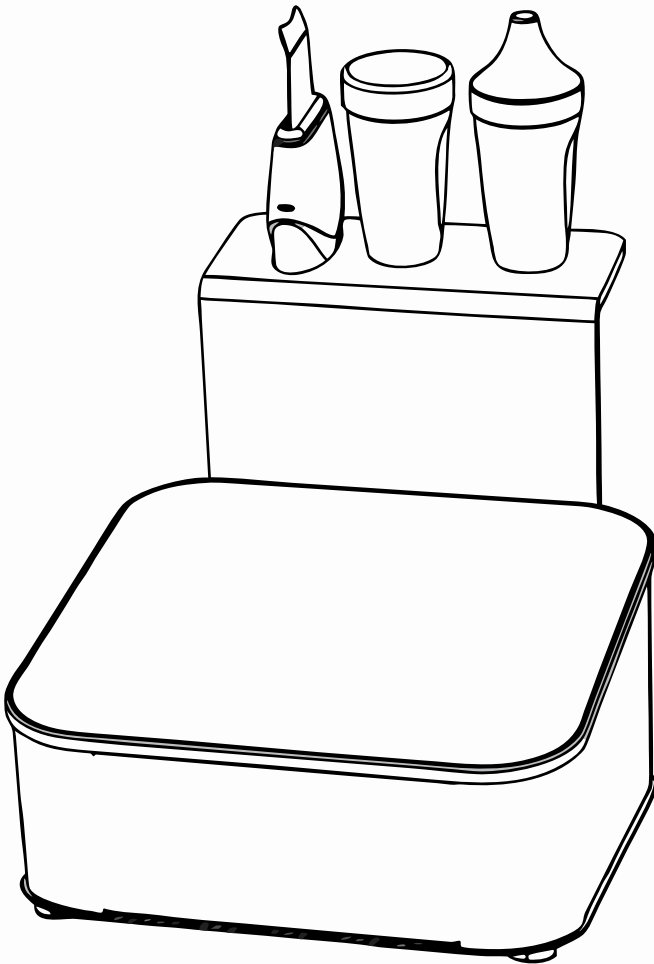

## NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

## Thank You For Purchasing This Product!

Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.

Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.

Some components of the product may differ from the image that appears in catalogues, websites, brochures, ect..

In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

## TABLE OF CONTENTS

Disclaimer .....	4
Contraindications & Precautions .....	5
Safety Instructions .....	6
Included Items .....	7
Device Components .....	8
Device Components .....	9
<b><u>Ultrasonic Function - Operating Instructions</u></b> .....	11
Instructions For Use .....	12
Daily Maintenance .....	13
<b><u>Skin Scrubber Function - Operating Instructions</u></b> .....	14
Instructions For Use .....	14
Post-Treatment Care .....	16
Important Notes .....	16
Specifications .....	17
Warranty Information .....	18

## DISCLAIMER

- This product has been designed to be used by esthetic professionals at spas and salons, it should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorization.
- We shall not be held liable for any material or personal damage due to the misuse of the product.
- Documents are made available without warranty of any kind. They may be updated, altered, expanded, or removed at any time. Skin Act does not warrant or guarantee any of the processes described herein or perform any independent analysis in connection with guidelines provided.
- Readers are expressly warned to consider and adopt all safety precautions indicated herein and to avoid all potential hazards. By following the instructions provided, the reader willingly assumes all risks in connection with such instructions.
- If the reader has questions about any of the guidelines provided by SkinAct we recommend they contact their nearest Esthetician Training Center to obtain more explicit instruction on Facial Machine Usage.

## Contraindications

THIS DEVICE SHOULD NOT BE USED ON CLIENTS WITH:

- Pacemakers, metallic implants, or other electronic medical devices.
- Heart conditions or a history of heart disease.
- Pregnancy.
- Inflamed, irritated, or damaged skin.
- Severe acne rosacea or vascular conditions.
- Epilepsy or other seizure disorders.
- Clients undergoing medical treatments or with underlying health conditions must obtain prior clearance from a physician.

## Precautions

- **Inspect the Device:** Check for any visible damage to the device or its power cable before and after each use.
- **Unplug When Not in Use:** Always disconnect the device from the power source before cleaning, storing, or performing maintenance.
- **Avoid Humidity:** Do not operate the device in bathrooms or other excessively humid environments.
- **Position Cables Safely:** Ensure the power cable is positioned to prevent tripping hazards and is plugged into a grounded socket.
- **Keep Away from Children:** The device and its accessories should be stored securely out of reach.
- **Report Issues Promptly:** If you notice abnormal functioning, contact your distributor or an authorized technician immediately.




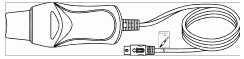
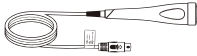
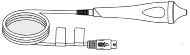
- **Use Authorized Replacement Parts:** If the power cord is damaged, it must be replaced with an original cord or an approved replacement provided by the manufacturer.
- **Maintain Probe Hygiene:** Keep the probe clean. After each use, clean and disinfect the probe with alcohol or another disinfectant. When using alcohol, wipe only the metal part of the probe to avoid damaging other components.
- **Avoid Sensitive Areas:** When using the ultrasound probe, never direct the probe toward the eyes.
- **Follow Recommended Treatment Time:** Treatment sessions should generally not exceed 15 minutes. For smaller treatment areas, reduce the treatment time accordingly.
- **Keep the Probe Moving:** Do not keep the probe in the same position on the skin for extended periods. Prolonged exposure to high energy may cause skin irritation or damage.
- **Check Attachment Connections:** Ensure attachments and probes are securely connected to the main unit before use. Avoid pulling excessively on the attachments or cables.

## Safety Instructions


- Plug the device into a properly grounded electrical outlet with the appropriate voltage.
- Do not use the device if it or its power cable is damaged.
- Never touch the device, power plug, or cables with wet hands.

- Always unplug the device before cleaning or maintenance.
- Keep the probe clean and disinfect it after each use.
- Do not keep the probe in the same position for long periods


### Included Items

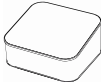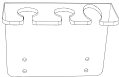Qty	Part Name	Image
1	Ultrasonic + Skin Scrubber Unit	
1	Attachment Rack	
1	Power Cable	
1	Skin Scraper Knife	
1	Face (medium) Probe	
1	Eyes (pointed) probe	


## Device Components

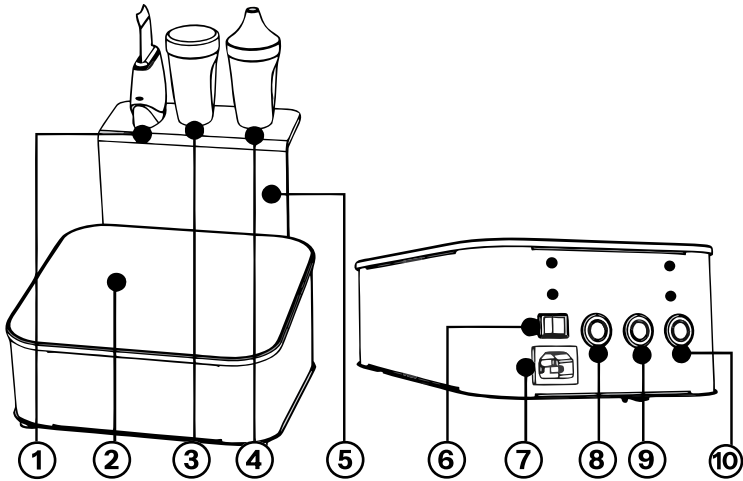


- |                      |                           |
|----------------------|---------------------------|
| 1. Skin Scrubber     | 6. Main Power Switch      |
| 2. Control Panel     | 7. Power Outlet           |
| 3. Facial Ultrasound | 8. Eye Ultrasound Port    |
| 4. Eye Ultrasound    | 9. Facial Ultrasound Port |
| 5. Attachment Rack   | 10. Skin Scrubber Port    |

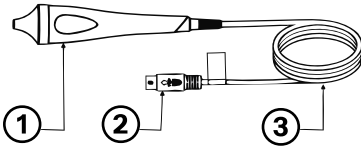


1. Eye Ultrasonic Handle
2. Eye Ultrasonic 5 Pin Plug
3. Eye Ultrasonic Cable

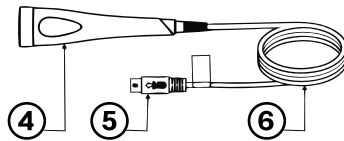

4. Face Ultrasonic Handle
5. Face Ultrasonic 5 Pin Plug
6. Face Ultrasonic Cable

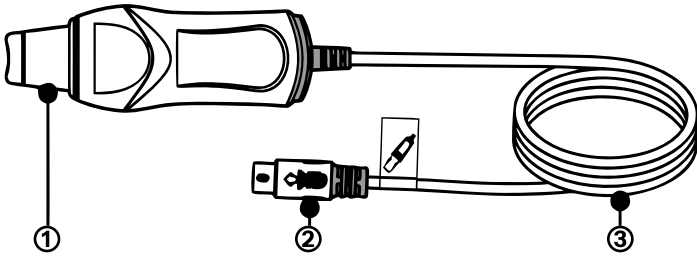


1. Skin Scrubber Blade
2. Skin Scrubber 5 Pin Plug
3. Skin Scrubber Cable

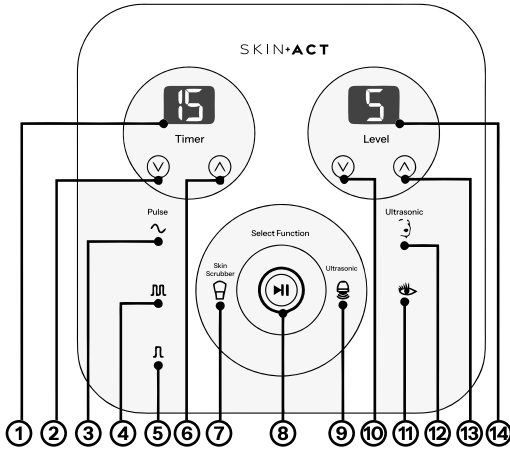


- |                    |                           |
|--------------------|---------------------------|
| 1. Timer Display   | 8. Start/Pause            |
| 2. Decrease Timer  | 9. Ultrasonic             |
| 3. Continuous Wave | 10. Decrease Energy Level |
| 4. Mixed Wave      | 11. Eye Selection         |
| 5. Wide Wave       | 12. Face Selection        |
| 6. Increase Timer  | 13. Increase Energy Level |
| 7. Skin Scrubber   | 14. Energy Level Display  |

## Assembly Instructions

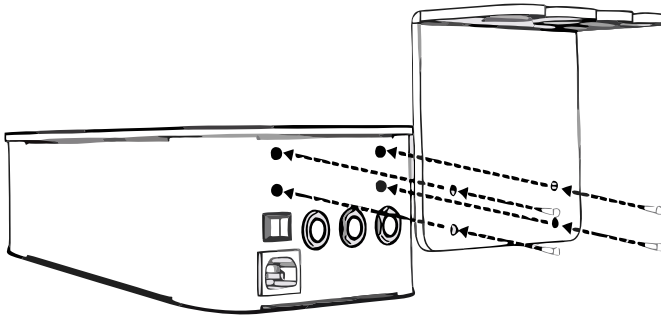

- 1. Attach the Rack:** Secure the attachment rack to the back of the main unit using the provided screws, following the direction of the arrows.
- 2. Place Attachments:** Place each attachment onto the rack.
- 3. Connect Attachments:** Plug all handles into corresponding output ports on the back of the main unit. Handles will not work unless attached to the correct interface.

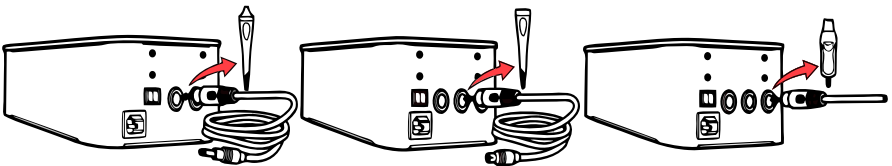

- 5. Power On The Device:** Turn on the main power switch located on the back of the device.
- 6. Standby Mode:** When the device is powered on, it enters standby mode. The time display will show “15,” and the energy level will display “1.” The default function indicator will be illuminated.

7. **Start or Pause Operation:** Press the start/pause button to begin or stop the device output while in standby or adjustment mode.
8. **Adjust Treatment Time:** Press the “Timer” button, then use the increase and decrease buttons to set the desired treatment time. The adjustable range is 1 to 30 minutes.
9. **Adjust Energy Level:** Use the Level Increase/Decrease buttons to adjust the energy output. The adjustable range is Level 1 to Level 9 to adjust the energy output.
10. **Select Function:** Functions can be adjusted as needed while in standby mode before starting the treatment.

## **Ultrasonic Function - Operating Instructions**

1. **Select Attachment:** Choose the appropriate ultrasonic handle for the treatment area (face or eye).
2. **Select Ultrasonic Function:** Press the ultrasonic icon on the control panel. The indicator light will turn on to confirm selection
3. **Choose Treatment Mode:** Select either facial or eye treatment mode on the control panel.
4. **Adjust Treatment Time:** Use the increase and decrease buttons to set the desired treatment time. The adjustable range is 1 to 30 minutes.
5. **Adjust Energy Level:** Use the Energy Increase/Decrease buttons to adjust the energy output. The adjustable range is Level 1 to Level 9. The energy level ranges from 1 to 9. Always begin at the lowest setting and increase as you go.

- 6. Start or Pause Operation:** Press the start/pause button to begin or stop the treatment at any time
- 7. Select Output Mode:** Choose the desired waveform based on treatment needs:

<p><b>Continuous Wave = Deep Push</b></p> <ul style="list-style-type: none"><li>▪ Deep Depth</li><li>▪ Thick/Dry/ Resilient Skin</li><li>▪ Strong use, keep moving, short time</li></ul>
<p><b>Mixed Wave = Balanced Push</b></p> <ul style="list-style-type: none"><li>▪ Medium Depth</li><li>▪ Normal/Combo/Oily Skin</li><li>▪ General full-face safe + absorption</li></ul>
<p><b>Wide Pulse Wave = Gentle Push</b></p> <ul style="list-style-type: none"><li>▪ Shallow Depth</li><li>▪ Sensitive/Thin Skin</li><li>▪ Light, delicate areas</li></ul>

Figure 1

## Instructions For Use

- 1. Prepare the Skin:** Cleanse the skin using an appropriate facial cleanser. Remove any remaining product and ensure the skin is clean and dry.
- 2. Apply Astringent (Optional):** Apply a light astringent if needed, then gently remove any excess cleaner.
- 3. Apply Treatment Medium:** Based on the skin type, apply a suitable gel or serum evenly to the treatment area to allow smooth probe movement.
- 4. Select Attachment:** Choose the appropriate wand (eye or face) for the treatment area.
- 5. Power On the Device:** Press the power button to turn on the device and enter standby mode.

6. **Adjust Energy Level:** Set the appropriate energy level based on the treatment area. For facial treatments, higher levels may be used as tolerated; for sensitive areas such as around the eyes, use a lower setting.
7. **Select Output Mode:** Choose the desired waveform based on treatment needs (Refer to Figure 1/Page 12 for detailed waveform descriptions):
  - **Continuous Wave** = Deep Depth
  - **Mixed Wave** = Medium Depth
  - **Wide Pulse** = Gentle
8. **Perform Treatment:** Move the probe evenly across the treatment area using slow, continuous motions. Use circular or back-and-forth movements as needed. Do not hold the probe in one place.
9. **Treatment Duration:** Recommended treatment time is approximately 3 - 5 minutes.

## **Daily Maintenance**

1. **Handle Cables Carefully:** Ensure all cables remain untangled and avoid excessive bending or twisting.
2. **Clean the Probe:** After each use, clean and disinfect the probe, then dry it thoroughly before storage.
3. **Clean the Device:** Wipe the main unit with a dry cloth. Do not use water or immerse the device.
4. **Power Off After Use:** Turn off the device and unplug it after each use or at the end of the workday.
5. **Store Properly:** Keep the device in a clean, dry environment away from sunlight, moisture, dust, and smoke.

## Skin Scrubber Function- Operating Instructions


- 1. Select Skin Scrubber Function:** Press the skin scrubber icon on the control panel. The indicator light will turn on to confirm selection.
- 2. Adjust Treatment Time:** Use the increase and decrease buttons to set the desired treatment time. The adjustable range is 1 to 30 minutes.
- 3. Adjust Energy Level:** Use the increase and decrease buttons to adjust the energy output. The energy level ranges from 1 to 9.
- 4. Start or Pause Operation:** Press the start/pause button to begin or stop the treatment at any time.
- 5. Select Output Mode:** Choose the desired skin scrubber waveform based on treatment needs:

<b>Continuous Pulse = Strong Clean</b> <ul style="list-style-type: none"><li>▪ Constant Vibration</li><li>▪ Deeper Pores</li><li>▪ Oily/congested Skin</li><li>▪ Strong debris &amp; sebum removal</li></ul>
<b>Mixed Pulse = Balanced Clean</b> <ul style="list-style-type: none"><li>▪ Alternating Strong + Soft Vibration</li><li>▪ Mid-Level depth pores</li><li>▪ Combo/general Use</li><li>▪ Even exfoliation, less irritation</li></ul>
<b>Wide Pulse = Gentle Clean</b> <ul style="list-style-type: none"><li>▪ Spaced pulse vibrations</li><li>▪ Surface level depth</li><li>▪ Sensitive/thin Skin</li><li>▪ Light exfoliation</li></ul>

Figure 2

## Instructions For Use

- 1. Prepare the Skin:** Soften the skin using warm water or a hot steam function before treatment.
- 2. Set Energy Level:** Adjust the energy output to a suitable level based on skin sensitivity and treatment area
- 3. Apply Cleansing Product:** Apply a cleansing product (such as cleansing milk) evenly to the skin and keep the skin moist throughout the treatment
- 4. Perform Treatment:** Follow the recommended treatment techniques as shown in Charts A, B, C, and D.

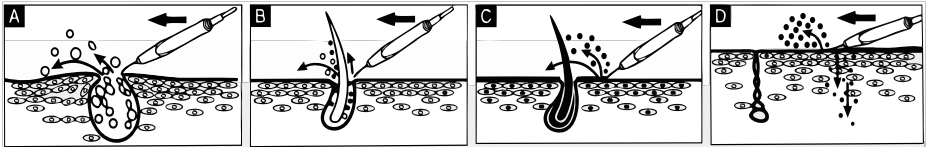

**Acne Removal (Figure A):** Ultrasonic vibrations help break down impurities and debris within the pores into smaller particles, allowing them to be easily removed. The process is gentle and non-invasive.

**Deep Pore Cleansing (Figure B):** High-frequency vibrations help remove dirt, oil, and cosmetic residue from deep within the pores, improving skin clarity. Corners of skin scrubber may be used for precision. Use gently and communicate with client to ensure their comfortability. Not to be used on severely inflamed skin.

**Surface Cleansing (Figure C):** The scraper head effectively removes dirt, oil, and dead skin cells from the surface of the skin.

**Product Infusion & Massage (Figure D):** Glide the spatula along the skin using upward motions to help enhance absorption of serums and skincare products.

## Post-Treatment Care

- 1. Cleanse the Skin:** Rinse the treated area with clean water or use a mild facial cleanser to remove any remaining residue.
- 2. Moisturize:** Apply a gentle moisturizer or repair serum to hydrate and soothe the skin. Avoid using harsh or irritating products immediately after treatment.
- 3. Clean the Spatula:** Rinse the scraper head with clean water and dry thoroughly with a soft cloth after each use.

## Important Notes

- **Keep the Device Moving:** Do not hold the scraper in one position on the skin to avoid irritation or damage.
- **Use Proper Angle (Cleansing):** When cleansing, hold the spatula at approximately a 45° angle (tip pointing down) to the skin and apply gentle pressure for best results.
- **Use Proper Angle (Infusion):** For product infusion or massage, keep the spatula head flat against the skin with the handle at approximately a 20° (flat side of blade) angle.
- **Maintain Skin Moisture:** Ensure the skin remains moist during treatment. Reapply water or skincare products as needed to prevent dryness.
- **Handle with Care:** Use the scraper gently and avoid excessive pressure during treatment.
- **Clean After Use:** After each use, wipe the scraper head clean, then disinfect and dry it thoroughly. Do not immerse the device in water.

## Specifications

<b>Product Name:</b>	<b>2 In 1 Ultrasonic &amp; Skin Scrubber Unit Elite Edition</b>
<b>Voltage:</b>	<b>AC100-120V/220-240V</b>
<b>Frequency:</b>	<b>50/60Hz</b>
<b>Power Consumption:</b>	<b>18W</b>
<b>Dimensions(L*W*H):</b>	<b>12x10x9 in</b>
<b>Net Weight:</b>	<b>4.6 lbs</b>

## Warranty Information

The manufacturer (the “warrantor”) warrants to the original purchaser that this equipment will be free from defects under normal and proper operation for a period of one (1) year from the date of purchase. This warranty covers all parts and labor for that one (1) year period. This warranty shall not apply to any equipment that has been subjected to abuse, misuse and alteration of any type or to any defect or damage caused by repair, replacement, substitution, or use with parts other than parts approved by the manufacturer. The manufacturer shall not be liable for the loss of use of this equipment, loss of time or business, inconvenience or any other indirect, consequential, special or incidental damages due to defects covered by this warranty or any implied warranty. To implement this warranty, contact the dealer from whom you purchased the equipment. If necessary, you will be asked to return all, or the appropriate part, of the equipment. If warranted, the repairs will be performed at no charge and the equipment returned to you. Unit owner must prepay all shipping costs when returning items for warranty service. This limited warranty is the only written or express warranty given by the manufacturer. This warranty gives you specific legal rights but you may also have other rights, which may vary from state to state or province to province.

SKIN+ACT

Phone: .....1-800-418-0830

Email: .....info@skinact.com

Website: .....skinact.com