

**SKINACT SUPRA 5 IN 1 MICRODERMABRASION  
+ ULTRASONIC + LED LIGHT THERAPY +  
MICROCURRENT + COLD&HOT THERAPY  
(SKU17813)**



**SKIN+ACT**

TOLL FREE: 1.800.418.0830

WWW.SKINACT.COM  
WWW.SPAANDEQUIPMENT.COM

INTERNATIONAL: 818.834.6640

# TABLE OF CONTENTS

|  |    |
|--|----|
| DISCLAIMER.....                                    | 03 |
| PANEL CONTROL .....                                | 04 |
| SPECIFICATIONS.....                                | 05 |
| TOOLS.....   | 06 |
| BEFORE YOU BEGIN.....                              | 06 |
| TREATMENT FUNCTIONS.....                           | 07 |
| GETTING READY FOR YOUR TREATMENT.....              | 09 |
| WARRANTY AND PRODUCT SPECIFICATION DATA SHEET..... | 11 |

## **THANK YOU FOR CHOOSING OUR PRODUCT!**

Before using the device, we kindly ask you to carefully read the entire instruction manual provided. Please pay close attention to any warnings outlined in this manual. It's important to keep this instruction manual easily accessible at all times. If you pass the device to another individual, please ensure to provide them with the manual, as it is an integral part of the product. Please note that certain components of the product may differ from the images presented in catalogues, websites, brochures, etc. As part of our commitment to continually improve our products, the manufacturer reserves the right to modify features and/or components without prior notice.

**THANK YOU AGAIN FOR YOUR PURCHASE. WE HOPE YOU ENJOY  
USING OUR PRODUCT!**

# DISCLAIMER

## CORRECT USE OF THE DEVICE

This machine is intended for general wellness purposes only. It is not a medical device and is not intended to diagnose, treat, cure, or prevent any disease. The information provided in this user manual is not a substitute for professional medical advice. Always consult with a physician before using this device, especially if you have any pre-existing medical conditions, are pregnant, or nursing. Discontinue use immediately if you experience any discomfort or adverse effects. The manufacturer and distributor disclaim any liability for damages or injuries resulting from the improper use of this device.

## WHO SHOULD NOT USE THIS DEVICE

- Individuals with heart conditions, pacemakers, or irregular heartbeats.
- Pregnant or breastfeeding women.
- People with epilepsy, alcoholism, ulcers, or digestive system disorders.
- Those diagnosed with cancer or undergoing cancer treatment.
- Individuals who have had major surgery in the past year.
- People with asthma, cold, flu, or upset stomach.
- Anyone with a serious illness or disease not listed above.

## AREAS TO AVOID TREATMENT

- Sensitive skin.
- Skin rashes, damaged, inflamed, or infected areas.
- Surgical scars less than 12 months old.
- Swollen skin.
- Eyeballs (probes should never point directly at them).
- Genitals.

## TREATMENT CAUTIONS

- Always start with the lowest settings and gradually increase based on your skin's reaction.
- Mild swelling and redness may occur after treatment but should disappear within a few hours.
- Overly aggressive techniques can cause temporary flaking or bleeding.
- Avoid using the microdermabrasion function on eyelids (keep them closed and covered during treatment).
- **Certain areas are not suitable for treatment: skin tumors, moles, birthmarks, lips, eyes, herpes, skin cancer, and liver spots.**

## DEVICE MAINTENANCE

- Do not modify or attempt to repair the machine yourself.
- Contact a qualified technician for maintenance needs.

# CONTROL PANEL

## MAIN UNIT


- Display Screen
- Touchscreen Interface

## POWER

- 110Volt
- 2 AMP
- 70 Watt



INTENSITY CONTROL KNOB

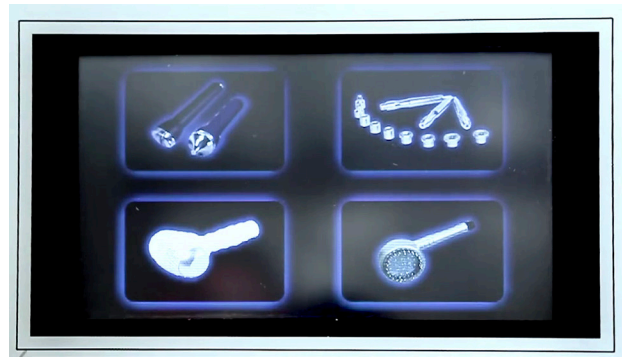

TOUCHSCREEN INTERFACE




DIAMOND MICRODERMABRASION



COLD AND HOT HAMMER



MICROCURRENT WITH LED LIGHT THERAPY

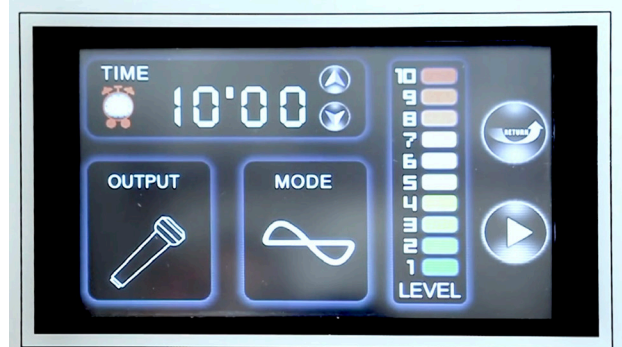

ULTRASONIC

## FEATURES:

- **Diamond Microdermabrasion:** Steam intensity, timer and ozone option.
- **Ultrasonic:** 5 professional glass electrodes, high frequency handle, intensity level adjustment.
- **LED Light Therapy:** 3 LED light treatments (red, green, and blue).
- **Microcurrent:** Microcurrent treatment wand.
- **Cold/Hot Hammer Treatment:** Adjustable arm, bright cool-tone LED, and magnifying lens.

## SPECIFICATIONS

### DIAMOND MICRODERMABRASION

- Diamond Microdermabrasion has 3 tip grades
- 3 Large: Fine, Medium, Coarse
- 3 Medium: Fine, Medium, Coarse
- 3 Small: Fine, Medium, Coarse
- \*\*\*The optional Diamond Microdermabrasion comes with 9 Tips and 3 Handles with filters\*\*\*
- \*\*\*The Diamond Microdermabrasion on this unit is class 1 grade. It is only for the surface of the skin\*\*\*

### ULTRASONIC

This kit comes with two different ultrasonic handles. The bigger flat handle is for the larger areas like the back or stomach. The smaller probe is for around the eyes

### LED LIGHT THERAPY

- **Red:** The red light therapy stimulates collagen production that is responsible for the elasticity and firmness of your skin. Collagen is essential to reduce fine lines and wrinkles.
- **Blue:** The blue light therapy kills the Propionibacterium acnes bacteria. This bacteria is one of the causes of acne. The bacteria absorbs the light and the light kills the bacteria.
- **Green:** The green LED treatment will lighten hyper pigmentation spots on the face and neck area.

### MICROCURRENT

- Tones and condition facial muscles, while tightening skin and improving elasticity.
- Smooth lines, wrinkles, fine lines and improve skin complexion.
- Boost circulation and increase supplies of nutrients to skin.
- Body sculpting and toning.
- Lymphatic drainage.
- Minimize eye bags and dark circles.

### COLD/HOT HAMMER

- Cold Hammer Minimum Temperature: ~52° F / ~11° C
- Hot Hammer Minimum Temperature: ~120° F / ~49° C

## Tools



**Diamond Microdermabrasion:** Diamond microdermabrasion resurfaces the skin non-surgically by using sterile diamond heads to peel and rub off the dead cells of the top skin layer. The microdermabrasion handles are attached to a vacuum. So, the dead skin cells are sucked away.



**Ultrasonic:** Stimulates the skin tissue with acoustic vibrations. Improves cell membrane and regenerates collagen. Reduces wrinkles and fine lines for younger looking and feeling skin. Ultrasonic also generates frictional heat which accelerates blood circulation and helps muscles to relax. Decomposes cellulite and allows deeper product penetration into the skin.



**LED Light Therapy:** The Red, Green, and Blue LED light therapy provides several benefits such as: the reduction of fine lines/wrinkles, the reduction of acne, as well as lightening of hyper pigmentation spots on the face and neck area. Each LED light color has it's own treatment benefits.



**Microcurrent:** The Microcurrent stimulates the muscle groups that need help lifting and firming. The electrical current increases the ATP and collagen production in the skin for younger, beautiful results without any down time. Microcurrent is a safe and painless facial which helps erase fine lines and wrinkles.



**Cold/Hot Hammer:** The Cold/Hot Hammer cools and soothes the skin with the cold side and warms the skin on the other side. Cooling the skin is important after a microdermabrasion treatment. Warming the skin opens pores and helps products to penetrate more deeply.

## BEFORE YOU BEGIN

- **CONSULTATION.** It is highly recommended to consult with a dermatologist before using this device, especially if you have any pre-existing skin conditions, allergies, or concerns.
- **SKIN PREPARATION.** Always cleanse and thoroughly dry your face before using any of the system's functions. Remove any makeup or skincare products.
- **REVIEW INSTRUCTIONS.** Carefully review the separate attachment guide for detailed instructions on proper usage, recommended settings, and cleaning procedures for each applicator.

# TREATMENT FUNCTIONS

## 1. MICRODERMABRASION

Microdermabrasion gently exfoliates the top layer of dead skin cells, revealing a smoother, brighter, and healthier-looking surface.

### THIS TREATMENT EFFECTIVELY ADDRESSES:

- Fine lines and wrinkles
- Crow's feet
- Age spots
- Acne scars

### BENEFITS:

- Provides a fresh, healthy glow
- Non-surgical, safe, and quick "lunchtime" treatment
- Suitable for most skin types with proper settings (consult attachment guide)
- No anesthesia required (ideal for those with sensitive skin)
- Allows immediate resumption of normal activities

## 2. HOT & COLD THERAPY

This dual-function therapy provides targeted benefits for your skin:

- **Cold Therapy:** Soothes and calms the skin after microdermabrasion or other exfoliating treatments, minimizing redness and irritation.
- **Hot Therapy:** Opens pores to enhance the absorption of skincare products, maximizing their effectiveness.

## 3. ULTRASONIC FUNCTION

Ultrasonic waves gently stimulate the skin, promoting collagen and elastin production, the building blocks of youthful skin.

### THIS RESULTS IN:

- Reduced fine lines and wrinkles
- Improved skin tone and texture
- Enhanced blood circulation and lymphatic drainage
- Increased hydration and plumping of the skin
- Deeper penetration of skincare products

## 4. BIO MICROCURRENT FUNCTION

BIO microcurrent technology uses low-level electrical currents to mimic the body's natural electrical signals.

### THIS GENTLE STIMULATION OFFERS A RANGE OF BENEFITS:

- Tones and conditions facial muscles, leading to a more sculpted appearance.
- Smooths wrinkles and fine lines by stimulating collagen production.
- Tightens skin and improves elasticity for a youthful look.
- Boosts circulation and nutrient delivery to the skin for a healthy glow.
- Reduces puffiness and dark circles around the eyes.
- Improves overall skin complexion.

## 5. LED LIGHT THERAPY

LED light therapy utilizes different wavelengths of light to address specific skin concerns:

- **Red Light:** Stimulates collagen production, reducing fine lines and wrinkles while promoting skin repair.
- **Blue Light:** Eliminates acne-causing bacteria, reducing breakouts and inflammation.
- **Green Light:** Lightens hyperpigmentation and dark spots, promoting a brighter complexion.

## TREATMENT GUIDELINES

Start with lower settings and gradually increase intensity as your skin tolerates. Discontinue use if you experience any discomfort or irritation.

## SAFETY INFORMATION

**Contraindications:** Do not use this device if you have any of the following:

- Active skin conditions (e.g., rosacea, eczema)
- Open wounds or infections
- Pacemaker or other implanted electronic devices
- Pregnancy or breastfeeding
- **Sunburn:** Do not use this device on sunburned skin.
- **Skin Sensitivity:** Individuals with very sensitive skin should consult a dermatologist before use and perform a patch test on a small area of the face to assess tolerance.
- **Post-Treatment Care:** Always follow recommended post-treatment care instructions, such as applying sunscreen with SPF 30 or higher after using the LED Light Therapy function.

**MAINTENANCE:** Regularly clean and disinfect the applicators according to the instructions in the attachment guide.

# SUPRA 5 IN 1: GETTING READY FOR YOUR TREATMENT

Here's a simplified guide to prepare for your Supra 5 in 1 treatment and how to use each function:

## BEFORE WE BEGIN:

1. **Skin Check:** A trained professional will examine your skin and create a personalized treatment plan.
2. **Protection:** The operator will wear gloves and a mask for hygiene.
3. **Clean Up:** For oily skin, a steamer or degreasing lotion may be used before cleansing. All skin types will be cleansed with a wipe and dried.

## OPERATING STEPS:

### 1. MICRODERMABRASION:

1. **Attach the Handpiece:** Connect the handpiece and tube to the machine.
2. **Diamond Tip Prep:** Place a cotton filter in the diamond tip, screw on the headpiece, and remove the filter after treatment.
3. **Power Up:** Select the desired power level (low, medium, high), treatment time, and start the machine.
4. **Treatment Technique:** Glide the handpiece in slow, even motions across the treatment area, avoiding staying in one spot too long. Stretch the skin gently for better coverage.
5. **Moisturize:** After treatment, dry your skin and apply a soothing gel.
6. **Clean Up:** Use a toothbrush or cotton swab with alcohol to clean the diamond tip. Disinfect it with UV light or high heat.

### 2. HOT & COLD THERAPY:

**Soothe & Enhance:** This dual function offers targeted benefits.

- **Cold Treatment:** Use this function to calm and reduce redness after microdermabrasion or other exfoliating treatments. It's especially helpful as your skin is more sensitive after peeling.
- **Hot Treatment:** Use this function to open pores for better absorption of skincare products, maximizing their effectiveness.
- **Temperature Selection:** The professional will help you choose the appropriate setting (hot or cold) based on your treatment needs.

**Targeted Application:** The professional will gently apply the attachment to the desired area for the recommended treatment duration.

### 3. ULTRASONIC:

1. **Function Selection:** Choose "Eye" or "Face" mode, set power level and treatment time, then start the machine.
2. **Power Adjustments:** You can adjust power during treatment. To switch modes, pause, select the new mode, and resume.

#### A. FACE TREATMENT:

1. Use the larger probe.
2. Choose your desired intensity level.
3. Select pulse or continuous wave mode.
4. Gently move the probe around your face, avoiding the eye area.

#### B. EYE TREATMENT:

1. Use the smaller probe.
2. Choose your desired intensity level.
3. Select pulse or continuous wave mode.
4. Gently move the probe around the eye area.

#### **4. BIOLIFT & LED LIGHT THERAPY:**

- 1. Function Selection:** Choose "Biolift" or "Biolift & LED" mode, set power level and treatment time, then start the machine.
- 2. Power Adjustments:** You can adjust power gradually throughout treatment. To switch modes, pause, select the new mode, and resume.
- 3. Gentle Power:** Increase power slowly based on your comfort level.
- 4. Treatment Time:** A typical treatment lasts 5-10 minutes. Avoid staying in one area for too long.
- 5. Clean Up:** After treatment, turn off the machine, unplug it, and clean the probe.

#### **5. LED LIGHT THERAPY:**

- **Lighten Up:** LED light therapy uses different colored lights to address specific concerns. Red light targets wrinkles, blue light fights acne, and green light lightens dark spots.
- **Choose Your Color:** The professional will advise on the appropriate light color based on your skin concern.
- **Targeted Treatment:** The professional will position the LED attachment for targeted treatment and follow the recommended treatment duration for each light color.

SpaandEquipment offers a 1-year limited warranty on most equipment, covering malfunctions and manufacturer defects only (parts only, shipping not included). Warranty is void for improper use, wear & tear, alterations, or lack of professional installation for plumbing & electrical equipment (proof required). Customers are responsible for return shipping costs and damage during return shipment voids warranty (ship with insurance).

**For details or questions, contact [SpaandEquipment.com](http://SpaandEquipment.com) at 1.800.418.0830.**

For further details regarding warranty coverage, please refer to our Warranty Information Policy on our website.

**You can find it under the “Return/Shipping/Warranty” section.**

#### **SPECIFICATIONS AND TECHNICAL DATA**

##### **PRODUCT NAME:**

SKINACT Supra 5 in 1 Microdermabrasion + Ultrasonic + LED Light Therapy + Microcurrent + Cold&Hot Therapy (SKU17813)

##### **FUNCTION:**

This revolutionary equipment features a Diamond Microdermabrasion, Ultrasonic, LED Light Therapy, Microcurrent, and Cold/Hot Hammer treatment capabilities all in one convenient device.

##### **SPECIFICATIONS:**

**17 “ W x 22 “ L x 9 “ H**