


SKIN+ACT

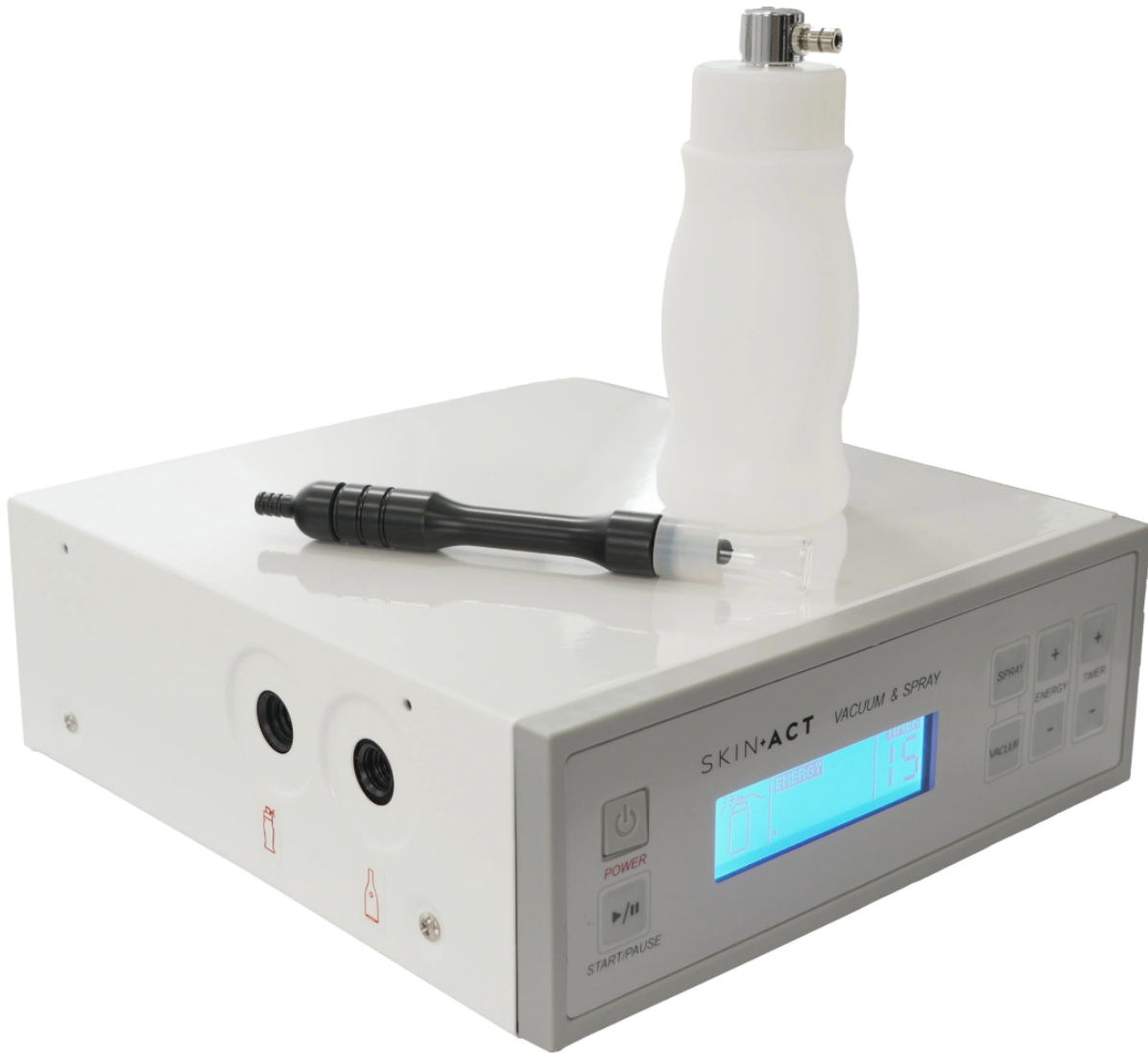


Vacuum And Spray Skin Care Unit

SKU25365


VACUUM & SPRAY

PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

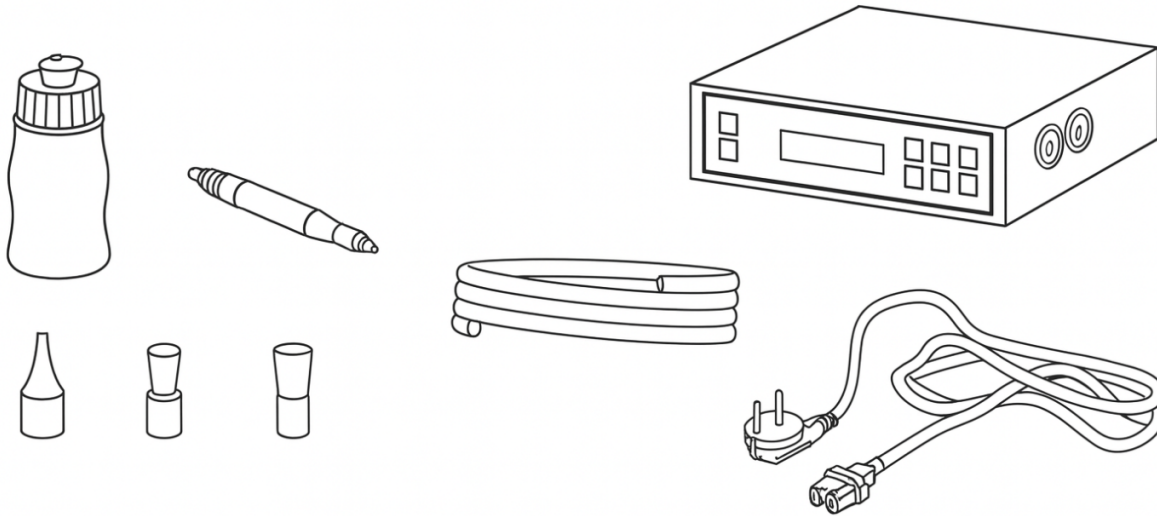


FRONT PANEL

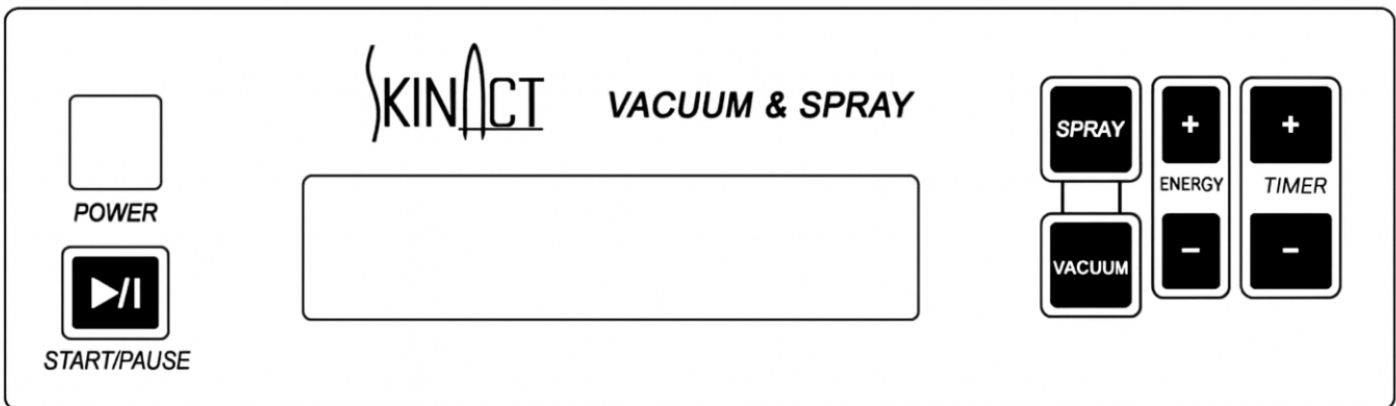


INSTRUCTIONS OF USE

- Connect the power supply via the rear panel, indicated in the sketch as “POWER LINE INPUT.”
- Connect the accessories properly in the front/rear panel as indicated in the sketch.
- Turn on the “POWER” button.

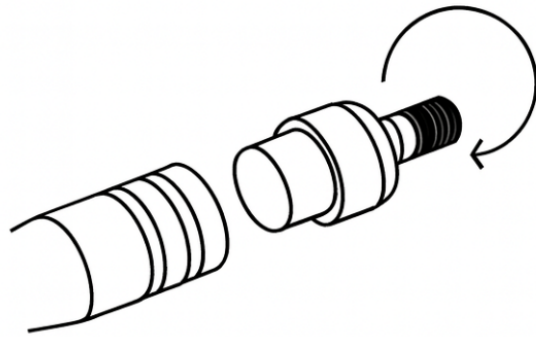

BLACKHEAD VACUUM

- Check that the foam filter is in its proper position in the handle.
- Select the “VACUUM” button to start the vacuum pump.
- Connect the vacuum air pipe to the corresponding vacuum nozzles on the side of the machine and then connect the other pipe ends to the vacuum handle. Three small glass heads are included, giving you the option of performing all types of facial vacuuming operations.

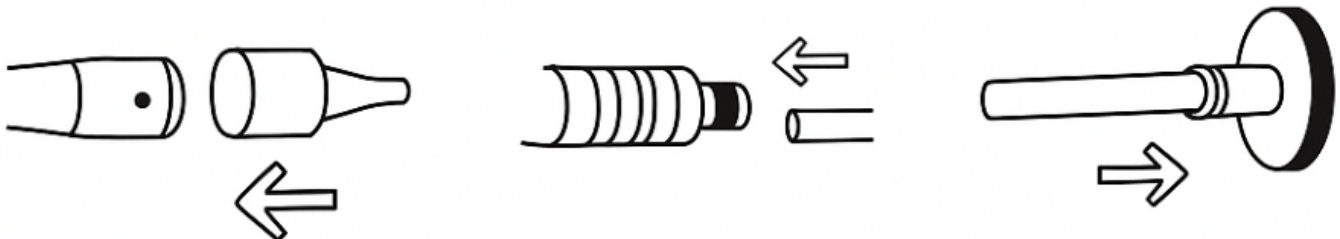



- Adjust the “ENERGY” and “TIMER” settings to your requirements.
- Press the “START/PAUSE” button to start treatment.
- You can adjust the suction effect by covering or uncovering the hole in the side of the handle with your finger.

NOTES

- Normally the operation should not last more than 5 minutes. If the instrument overheats, turn off all the buttons and restart after 10 minutes.

SPRAY

- Connect one end of the spray air pipe to the corresponding spray output nozzle on the side of the machine, and then connect the other end to the small spray bottle inlet nozzle. Two spray bottles are supplied, which can contain different liquids for use in treatments.

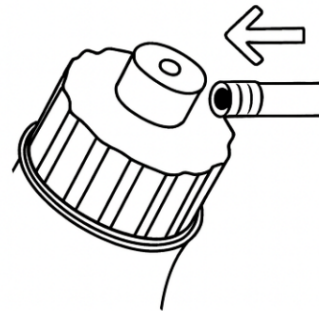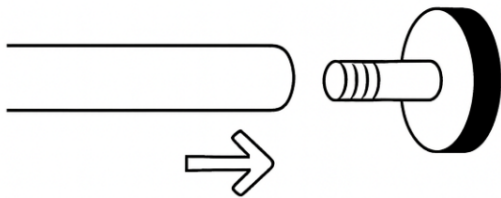

- Adjust the “ENERGY” and “TIMER” settings to your requirements.
- Press the “START/PAUSE” button to start treatment.
- To operate the spray bottle, simply cover the top vacuum nozzle with your index finger, which will allow the desired liquid to be released from the spray canister. To stop the flow of spray, simply remove your finger from the nozzle and the vacuum seal will be broken.

NOTE

- Spray downwards from the forehead during operation. Stop the liquid from going into the nose. Ask the customer to close their eyes and shut their mouth.
- Maintain an adequate distance from the face whilst spraying so that the effect is felt as a gentle mist landing on the skin.
- Normally the operation should not last more than 5 minutes. If the instrument overheats, turn off all the buttons and restart after 10 minutes.

CAUTIONS

- Do not leave the handle on the same part of skin for too long since it can cause injuries.
- Avoid using on irritated skin.
- Always make sure the spray pipe is connected to the right nozzle of the machine, to prevent accidental vacuuming of any spray solution into the pump, which can damage the pump of your machine and void your manufacturer's warranty.
- Do not work the vac/spray pump for extended periods without cool-down time between treatments in order to prevent overheating of the pump and damage to the machine. Damage caused by overheating of the pump is not covered by the manufacturer's warranty.

FAQ

If the suction effect is reduced:
Replace the metal and foam filters.

MAINTENANCE

- The foam filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.

TECHNICAL SHEET

Weights and sizes	Product size (cm)	20.7×22×7
	Gross weight (kg)	3.5
Electrical characteristics	Rated voltage	AC 100-240V
	Frequency	50Hz/60Hz
	Rated power	65 W

SKIN+ACT

Toll Free: 1.800.418.0830	www.skinact.com www.spaandequipment.com	International: 818.834.6640
---------------------------	--	-----------------------------