

Venice Electric Medical Spa Treatment Chair/Table (With Swivel)

User Manual



SKIN+ACT

www.skinact.com | www.spaandequipment.com

Toll Free: 1.800.418.0830

Copyright © 2025 SkinAct, All Rights Reserved

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by SkinAct. SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

SKIN+ACT

www.skinact.com | www.spaandequipment.com


Toll Free: 1.800.418.0830

Product Features

Introducing the Venice Electric Treatment Table—where sleek design meets advanced functionality. Crafted for modern spa and aesthetic settings, this treatment chair features an effortless side-button adjustment system for seamless positioning of the backrest and seat cushion. Its comprehensive structure includes a four-control program controller that connects to four powerful motors, delivering precise control over the backrest, seat, leg-pad angles, and bed height. Built with premium hardware and leather, the Venice ensures both durability and elegance. The user-friendly interface and ultra-stable frame provide a safe, smooth experience at any angle—making it the perfect blend of comfort, performance, and style.

1. **Effortless Positioning:** Easily adjust the backrest and seat cushion using the intuitive side control button, providing smooth and convenient positioning for clients during treatments.
2. **Advanced Construction:** Built with a four-function control system, durable motors, high-quality hardware, and premium leather. The controller seamlessly connects with a manual control panel and four powerful motors.
3. **All-in-One Adjustability:** Features include independent control of backrest angle, seat cushion angle, leg pad angle, and bed height—all integrated into a single streamlined structure.
4. **Modern, User-Friendly Design:** Designed for simplicity and comfort, this chair combines sleek aesthetics with easy operation to elevate both client experience and technician efficiency.
5. **Superior Stability and Safety:** Engineered for maximum stability with zero wobble at any height or angle, ensuring a secure and reliable experience for every use.


Specifications

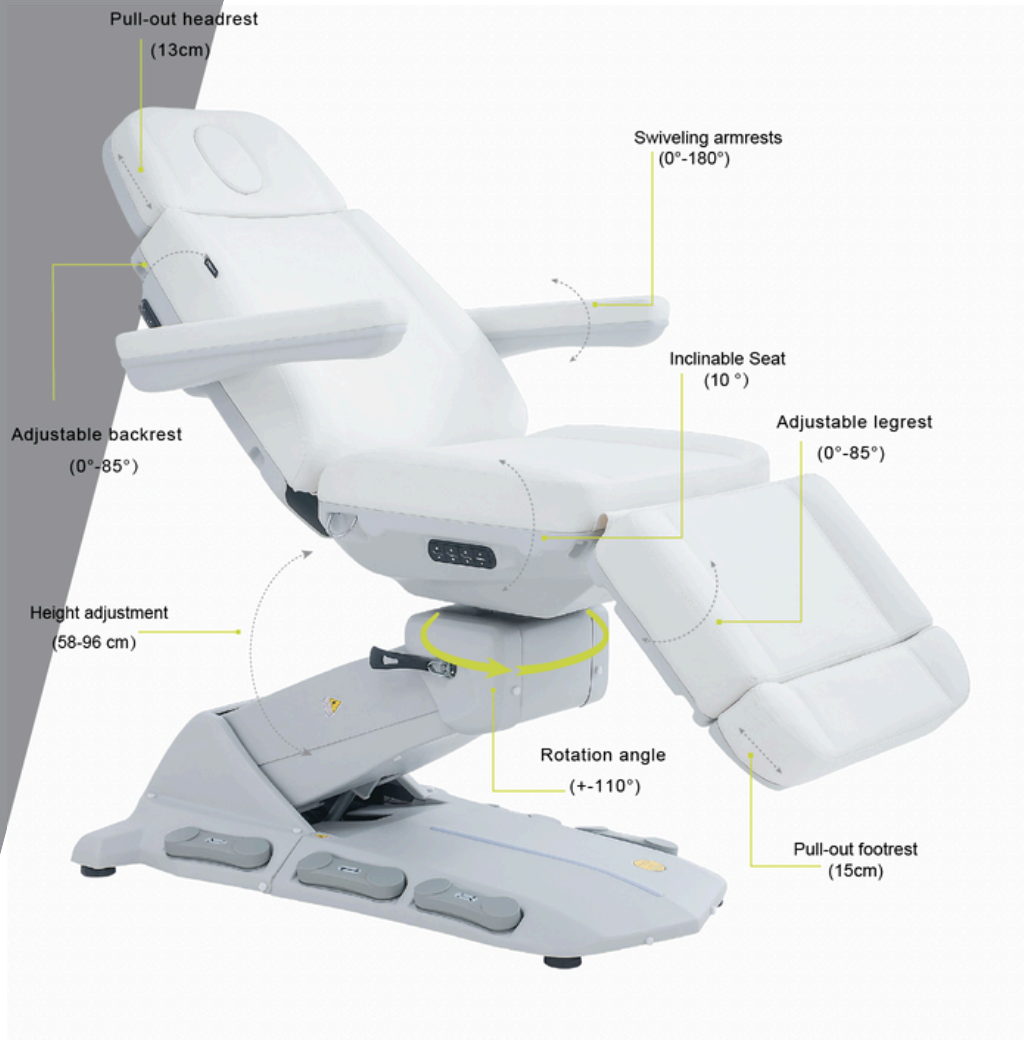


Effortless Button Function

The "Effortless Button" refers to the control buttons located on the backrest and the side of the seat cushion. These buttons allow you to easily adjust the chair to your desired working position with a simple touch. Designed for convenience and precision, this feature supports smooth, efficient adjustments—ideal for enhancing comfort and productivity in professional spa and beauty settings.

1. Cushion cover button:

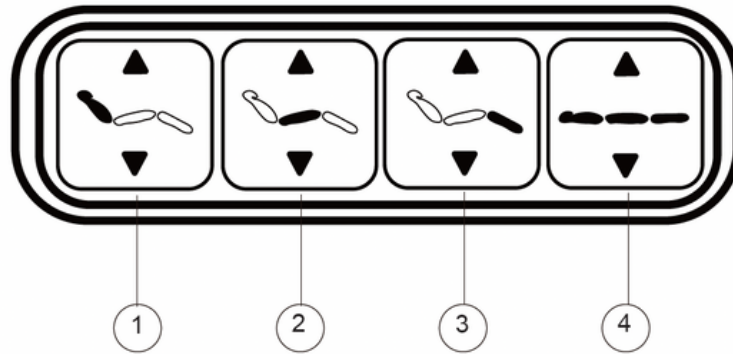


2. Backrest cover buttons:

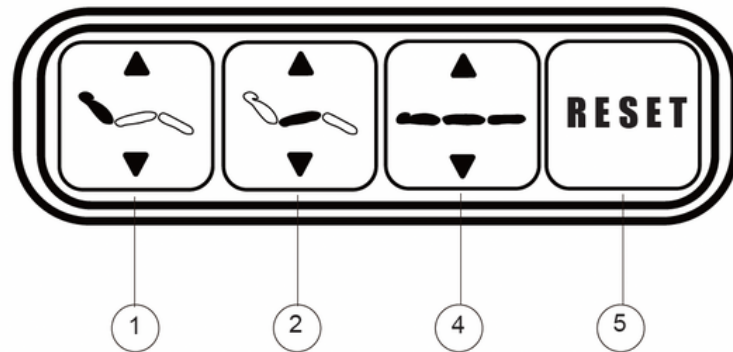


How to use:

1. Backrest Adjustment

Press  to adjust the inclination of backrest, as shown in the figure




2. Seat rest Adjustment

Press  to adjust the inclination of seat rest, as shown in the figure




3. Adjustment of Leg rest

Press  to adjust the inclination of leg pad, as shown in the figure



How to use:

4. Adjustment of Height

Press  to adjust the rise or fall of the bed, as shown in the figure



5. Adjustment of handrail


Move the handrail back and forth through a full 180-degree range





6. Adjust the Bed Rotation


- To rotate the chair, pull out the adjustment handle to enable a 110-degree rotation to the left and right.

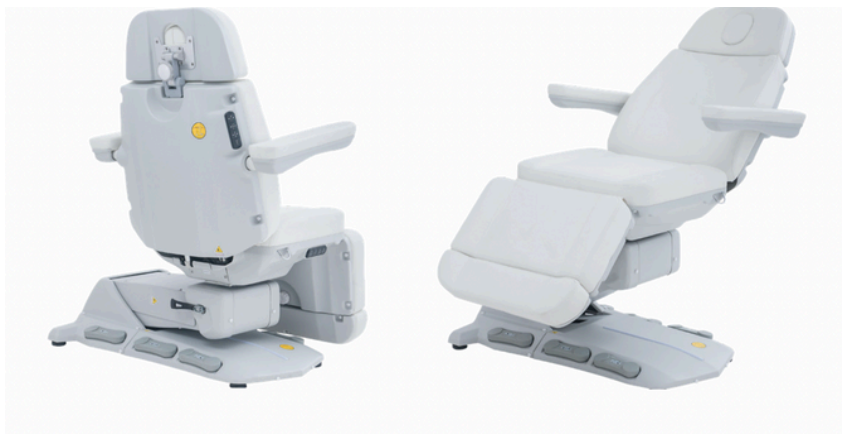- Once the desired angle is reached, push the handle back in to securely lock the bed in place.







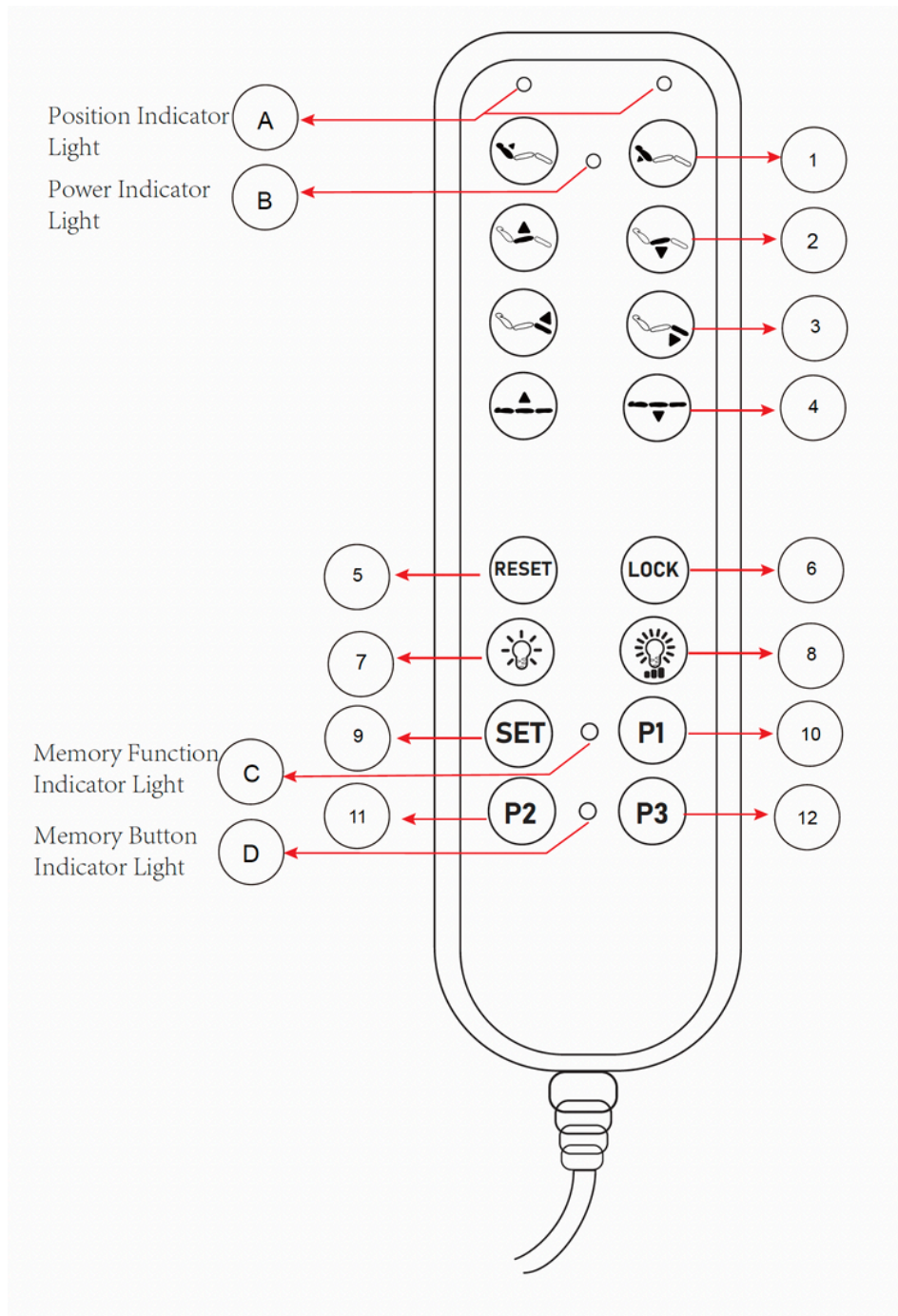





Hand Remote Controls:

Tip: All buttons on the device will flash during initial startup, indicating a locked state. Press the  button to unlock then all buttons will stop flashing and the  light will remain on.



Refer to next page for button descriptions ----->


Hand Remote Controls:

- 1 Press  or  button to control the lifting angle of the backrest of the spa chair.
- 2 Press  or  button to control the tilting angle of the cushion of the spa chair.
- 3 Press  or  button to control the lifting angle of the footrest of the spa chair.
- 4 Press  or  button to control the rising or lowering of the whole spa chair.
- 5 Press and hold the **RESET** button for 3 seconds to reset motor. The backrest will automatically move to the maximum; the height of the whole chair will be lower to the greatest extent; the cushion will automatically move to the maximum; and the leg pad will move to the minimum.
- 6 In the active state, press the **LOCK** button to secure the system, disabling all buttons. To reactivate the buttons, press **LOCK** again while in the locked state.
- 7 Press the  button to active and change the color of the ambient light. It will turn off after the last cycle.
- 8 Press the  button to adjust the brightness of the ambient light.
- 9 Hold the **SET** button 3 seconds to active position recoding and the Memory Function Indicator Light **C** will stay on 3 seconds for recoding. If you press the **P1** key, the current position of the actuator will be memorized as position 1. Other number keys have the same function.
- 10 Hold the **P1** button for 3 seconds, it will automatically move to position 1 which has been memorized, and you can press any button to quit this mode and stop the actuator operation.
- 11 Hold the **P2** button for 3 seconds, it will automatically move to position 2 which has been memorized, and you can press any button to quit this mode and stop the actuator operation.
- 12 Hold the **P3** button for 3 seconds, it will automatically move to position 3 which has been memorized, and you can press any button to quit this mode and stop the actuator operation.

Headrest:


To assemble the headrest, tighten the 4 screws following the drawing instructions below.

All the parts and screws come with the bed packaging.

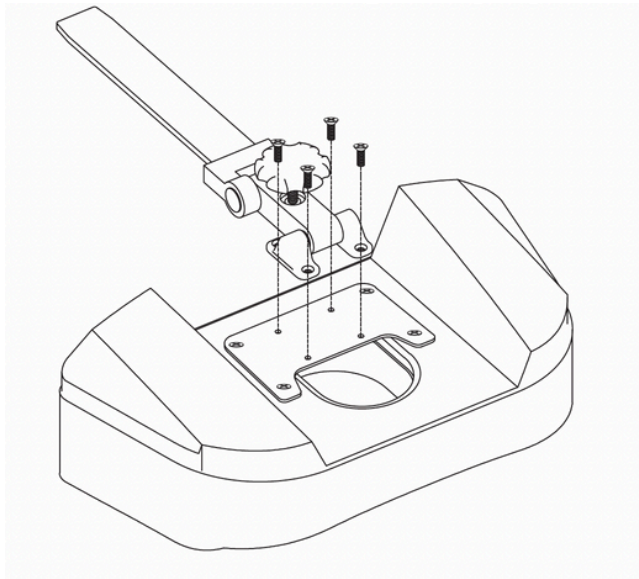



Switch Panel:


1. Ensure the power cord and hand controller are properly inserted in the correct direction and locked as indicated by the arrow.
2. Turn on the power switch; the indicator should be steadily lit.
3. If the device shows no response, turn off the power, open the fuse box, check the fuse, and replace it if necessary.

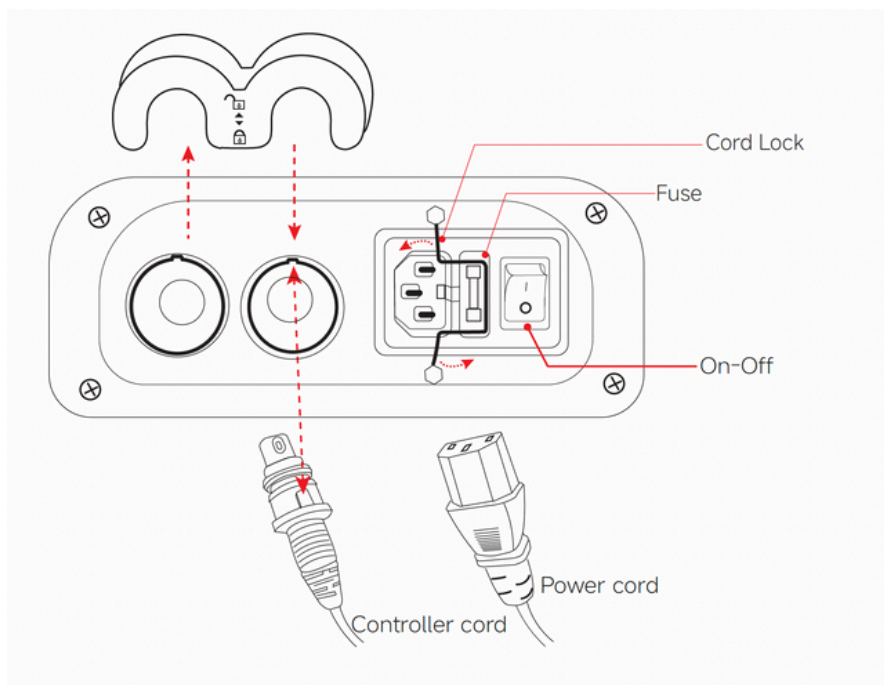


Advanced Foot Pedals:

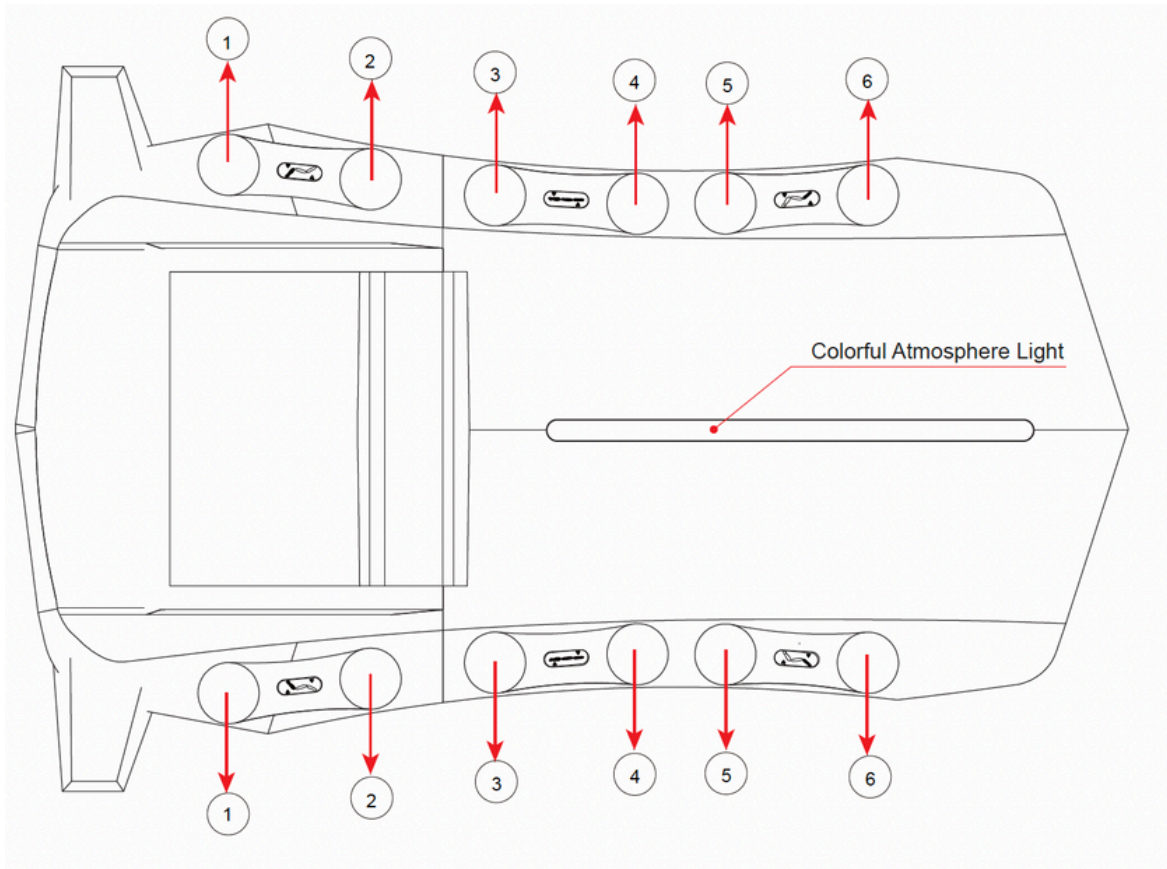

1. Step on foot pedal  to control the lifting angle of the backrest of the spa chair. (1) control the backrest up (2) control the backrest down.

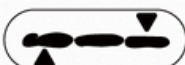2. Step on foot pedal  to the rising or lowering of the whole spa chair. (3) control the chair rising (4) control the chair lowering.

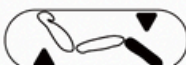3. Step on foot pedal  to control the lifting angle of the footrest of the spa chair. (5) control the footrest up (6) control the footrest down.

Maintenance:

- **Daily Cleaning**

- Use a soft, dry cloth to gently remove dust from the product’s surface.

- **Stain Removal**

- For persistent stains, wipe the affected area with a soft towel dampened with a diluted neutral cleaner. Be sure to dry thoroughly with a clean, dry cloth after cleaning.

- **Do Not Use Harsh Solvents**

- Avoid using benzene, thinners, gasoline, or similar solvents, as these may damage the leather or degrade the product’s surface.

- **Storage and Operating Conditions**

- Store and use the product in a cool, dry environment away from direct sunlight and humidity.

- Recommended operating temperature: -5°C to +50°C

- Recommended relative humidity: 45% to 85%

Routine Maintenance

- Lubricate all moving parts once per year to ensure optimal performance and extend the lifespan of the product.

Please perform the following troubleshooting steps if you encounter any issues while using this product.

Phenomenon	Diagnosis	Eliminating Operations
Press the control key but nothing happens to the bed	<ol style="list-style-type: none"> 1.The function of the pressed control key has been switched to the extremity; 2.Whether the power plug is improperly inserted; 3. Whether all connecting plugs of the transformer are inserted properly; 4.Quality problem of manual control board. 	<ol style="list-style-type: none"> 1.Try to press other control keys; 2.Unplug the power plug and replug it; 3.Unplug and replug; 4.Replace the manual control panel.
Abnormal sound and shaking during the lifting and lowering of the chair	<ol style="list-style-type: none"> 1.The screws are loose 2.All the rotational components are lacking of lubricating oil 3.Quality problem of motor 	<ol style="list-style-type: none"> 1.Tighten all screws 2. Add some lubricant 3.Replace the motor