

Crystal Microdermabrasion + Hot Towel Cabinets (German Motor) + Diamond Microdermabrasion Option



SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

Crystal Peeling

Crystal Peeling Instrument is a strong beauty art that has taken Europe and America by storm leading a fashion and beauty revolution. Natural crystal powder and air currents peel the skin softly. Aged skin is removed through the abrasion of the crystal powder while the vacuum function removes the aged cells and stimulate the skin to grow and renew. Using the special crystals can help smooth out wrinkles and fine lines and can be very beneficial for scars left from wounds or chicken pox. Microdermabrasion works on all skin types. Mature and dull skin greatly benefit from the exfoliative properties of consistent microdermabrasion. Dry and sensitive skin is revitalized as dead skin cells are removed and products used after treatment (face masks, creams, etc.) have deeper penetration. Greasy/ acne prone skin becomes clearer, skin texture is refined, and imperfections are reduced. It also helps to unclog pores, stimulate the circulation of the skin, eliminate dead skin cells from the top layer of the skin, and improves the skin's ability to retain moisture giving the skin a more youthful and healthy appearance. Suction creates micro-drainage of excess lymph fluids and firms the skin creating an inner radiant glow.

Operation:

1. Put the instrument on a clean and flat surface.

2. Put the powder into the bottle and the powder is no more than 2/3 of the bottle.



3. Connect the instruments.

4. Plug the power wire and turn on the machine.



5. Use the handle to test the suction and then adjust the intensity. The entire treatment time should be 15-20 minutes.

6. During the process of treatment, use forefinger and middle finger to stretch the skin for better exfoliation.

7. Plan treatments according to the nature of the skin or client sensitivity.

8. Adjust the intensity according to the different skin types and client needs. Do not direct the spray of crystals at one spot for more than 5 seconds as this could damage the skin.

9.Keep the skin dry throughout the entire treatment. Do.

10.Gently spray across the skin. Pressure and speed will not improve the quality of exfoliation.

11.Redness after the treatment is normal.

Notice:

1.The powder should be no more than 2/3 of the bottle.

2.Please do not use the powder that other suppliers offer. THIS CAN VOID YOUR WARRANTY.

3.The powder should be kept in a clean dry area.

4.After finishing the treatment, dispose of the used crystals. You should not reuse any of the crystal.

5.Do not forget to clean the filter.



To Clean the Filter:

1.Remove the Pot

2.Remove the Screw

3.Clean filter

4.If the instrument is not being used place it in a clean dry area.

Rules

- 1.This instrument is intended for professional use only and therefore should be used only by those licensed/certified to use it.
2. It should not be used on clients with skin problems.
- 3.It should not be used on rosacea or skin with broken capillaries.
- 4.This appliance is not intended for use by people with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- 5.Do not use on children.
- 6.Children should be supervised to ensure that they do not play with the appliance.

Notice:

- 1.Prepare the skin beforehand by removing the makeup with wipes.
- 2.Preform the treatment on dry skin.
- 3.After the treatment use a calming non-alcohol toner to soothe the skin.
- 4.If you choose to do serums you can apply them after the toner.
- 5.Apply a moisturizer and a sunscreen.
- 6.Use only nonirritating products after any microdermabrasion to prevent irritations.

IMPORTANT SAFETY NOTICE:

- 1.Please don't disassemble the equipment. THIS COULD VOID YOUR WARRANTY.
- 2.Do not try to perform any function not indicated in our user manual. All the repair work should be done by our professional personnel.
- 3.Please don't assemble or operate this equipment while you are near the water or your hands are wet. Do not pour liquid on it or submerge it in water.

4. Turn off the unit, disconnect from the power supply, and contact technical support for any of the following:

a) If the unit becomes wet.

b) The equipment has abnormal smell, smoking or a loud and strong noise.

c) A cable is broken.

d) The equipment has fallen down and broken.

5. Don't put anything on the cables and do not place the equipment where someone can step on the cables.

6. Disconnect from the power supply when not in use for your own safety.

7. Don't put any foreign objects into the equipment's outlets or sockets, they may cause fire or cause electrocution.

8. If something enters the equipment do not attempt to remove it yourself. Contact your technical support for assistance.

9. Don't place the equipment on any uneven surfaces.

Our company reserves the right to modify the information without prior notice!
And keep the explanations right for the above.

Diamond Microdermabrasion:

Microdermabrasion is a skin turnover technique that helps repair facial skin damage from the sun and helps minimize the effects of aging. The technician uses a device like a fine sand paper to polish the skin with the diamond tips. The tips glide across the face; mixing gentle abrasion with suction to remove the dead outer layer of skin. As with other skin rejuvenation techniques, more than one treatment may be needed to remove fine wrinkles and unwanted pigmentation. Microdermabrasion is effective in reducing fine lines, "crow's feet", age spots and acne scars. It stimulates the production of skin cells and collagen. It has proven to be a very popular nonsurgical procedure.

Benefits:

- Virtually no side effects
- Provides the skin with a fresh healthy-looking glow
- Nonsurgical, safe, effective "lunch hour" procedure
- Effective on all skin colors and types
- No anesthesia required
- Excellent for skin sensitive to chemical procedures
- Normal activities can be resumed immediately

Operation

1. Sterilize diamond tips
2. Put on disposable gloves and mask.
3. Prepare the skin surface for treatment by first removing any makeup using gentle nonirritating wipes.
4. Make sure the skin is dry to the touch.
5. Make sure the diamond tips is either properly sterilized or replaced.
6. Begin treatment.
7. Adjust the intensity according to client comfort.
8. Once the treatment is completed, the user must cool the treated surface of the skin by applying cold towels.
9. Keep doing this until the skin becomes cool.
10. The skin has just experienced an abrasive treatment. There may be some redness as well as a warm effect.
11. Make sure the clients do not use any other abrasive or irritating products on the skin, as it may cause an adverse reaction.

Note: However, due to the nature of its treatment, it is to be used ONLY by certified personnel. Any violations of the above will void the warranty and the operators should be responsible for any damages to the systems or clients.

PROCESS AND GUIDE

1. Nine different types of diamond tips of various shape and sizes, which can be used in different parts of body, including the eye area. Selecting the appropriate tip and adjusting the vacuum allows for controlled microdermabrasion.



Full body microdermabrasion can be performed before slimming program, on arm acne, and on cellulite etc.



Eye tips are especially for wrinkles, eye area, blackheads, whiteheads, other small areas etc.



Focus on scars, fine lines, and sun damaged skin during the majority of the treatment.

Focus on scars, fine lines, and sun damaged skin during the majority of the treatment.

2.Durability: The diamond cut tips have a long shelf life.

3.Safety

By eliminating aluminum oxide crystals, there is no risk of particles remaining embedded in the skin. Neither patient nor clinician has to breathe aluminum oxide dust, and the treatment room remains clean. This non-invasive skin care treatment can be completed in a short -period of time and without pain. This skin care treatment is ideal for people on the go. There is no downtime from work or social activities.



Hot Towel Cabinet

The Hot Towel Cabinet heats towels for warm soothing treatments. You place damp towels into the Cabinet and out come perfectly toasty towels you can use for any number of treatments. The drip tray catches all the excess water to keep your floors dry. The sturdy frame allows for many years of trouble free and safe operation. Your clients will cherish the soothing comfort each hot towel delivers.

PREPARATION

- 1.Run towels briefly under water.
- 2.Wring out excess water from the towels. Towels should be damp, not soaked. You may need to wring out several times.
- 3.Fold or roll towels and place towels onto wire rack. Do not over fill the cabinet. You must the rack to hold the towels. Failure to do so could damage your unit or towels and could void your warranty.
- 4.Close the door firmly.
- 5.Turn on the power switch on the towel warmer. The power switch will remain illuminated while the switch is in the "ON" position.

UV Sterilizer

- 1.Clean your tools beforehand.
- 2.Place them in the U.V. Sterilizer for 10-15.
- 3.Close the door properly to avoid eye damage.

CAUTION

INTERNAL SURFACES, WIRE BASKETS, TOWELS AND STEAM MAY BE HOT. USE EXTREME CAUTION WHEN REMOVING HOT TOWELS OR BASKETS. ALWAYS USE PROTECTIVE HEAT RESISTANT GLOVES OR MITTS TO AVOID STEAM OR OTHER BURNS. FAILURE TO HEED WARNINGS MAY RESULT IN HARM OR SERIOUS INJURY.

CLEANING & MAINTENANCE

Maintenance should be performed after each day's use.

- 1.Turn OFF the unit by using the power switch.
- 2.Unplug the unit from the power supply.
- 3.Wait for contents to cool before handling. Contents may be hot. Remove the wire racks and towels. Set the wire racks aside to dry.
- 4.With a soft cloth, wipe down the internal surfaces. Do not use harsh chemicals or abrasives. To clean the exterior, use a soft cloth and mild soap.
- 5.Pull the drip tray forward and empty contents. Set the drip try aside to dry.

IMPORTANT SAFEGUARDS

Read All Instructions before Using

Caution must be practiced at all times when using electrical appliances. When using electrical appliances, basic safety precautions should always be followed, including the following:

Do not use the appliance for anything other than its intended purpose.

Do not touch hot surfaces. Use handles or knobs.

To avoid electrical shock: do not immerse cord, plugs, or appliances in water or other liquids.

Unplug from power sources when not in use and before cleaning or maintenance. Allow to cool before taking off parts.

Do not operate any appliance with a damaged cord or plug.

Do not operate after any malfunctions occur or if the appliance has been damaged.

Do not use parts that have not been explicitly recommended by the manufacturer. This could cause injury.

Do not use outdoors. The appliance is recommended for indoor use only.

Do not let the cord touch hot surfaces.

Use close supervision when using the appliance by or near children.

Do not leave the appliance unattended when on.

Do not use near water or other liquids.

Plug the unit into a power supply of appropriate voltage.

Use care when handling the hot towels.

Use care when cleaning. Do not clean or maintain the unit when hot.

Do not attempt to repair the unit yourself. Contact technical support at 1-800-418-0830.

WARNING

TO PREVENT ELECTRICAL SHOCK, UNPLUG BEFORE CLEANING AND BE SURE TO REMOVE ALL PACKING MATERIALS BEFORE USE.

Troubleshooting

1.If the towels are not heating properly:

- a)Make sure the towels are only slightly damp.
- b)If there is excess moisture in the towels, wring them out and place back in the cabinet.

2.If the unit does not turn on:

- a)Check the switch. If the switch is lit there is power in the unit.
- b)If the switch is not lit check the fuse and replace if necessary.

Warranty Statement

The manufacturer (the "warrantor") warrants to the original purchaser that this equipment will be free from defects under normal and proper operation for a period of one (1) year from the date of purchase. This warranty covers all parts and labor for that one (1) year period. This warranty shall not apply to any equipment that has been subjected to abuse, misuse, and alteration of any type or to any defect or damage caused by repair, replacement, substitution, or use with parts other than parts approved by the manufacturer. This warranty also does not cover normal wear and tear. The manufacturer shall not be liable for the loss of use of this equipment, loss of time or business, inconvenience or any other indirect, consequential, special or incidental damages due to defects covered by this warranty or any implied warranty. To implement this warranty, contact the dealer from whom you purchased the equipment. If necessary, you will be asked to return the entire unit or the appropriate part of the equipment. If warranted, the repairs will be performed at no charge and the equipment returned to you. Unit owner must prepay all shipping costs when returning items for warranty service. This limited warranty is the only written or express warranty given by the manufacturer. This warranty gives you specific legal rights but you may also have other rights, which may vary from state to state or province to province.

SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640